

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DRB1 allele ^{1,2,3}																								
*01:01:01-01:01:26, 01:38, 01:41, 01:45, 01:47-01:49, 01:53, 01:56, 01:59-01:60	1		3																					
*01:02:01-01:02:05, 01:02:07-01:02:09, 01:43, 01:46, 01:57		2	3																				23	
*01:02:06, 01:58			3																				23	
*01:03	1			4			7																	
*01:04			3		5				9															
*01:05	1		3			6																		
*01:06			3						9	10														
*01:07	1		3					8																
*01:08	1		3						9															
*01:09	1		3							10														
*01:10	1		3	4								11												
*01:11:01-01:11:02	1		3		5																			
*01:12, 01:37	1		3									12												
*01:13	1		3										13											
*01:14	1		3											14										
*01:15	1						7			10														
*01:16	1		3		5										15									
*01:17	1		3				7																	
*01:18	1		3							11														
*01:19	1		3													16								
*01:20			3						9															
*01:21	1		3												15	16								
*01:22	w		w														17							
*01:23		2											13	14									23	
*01:24:01-01:24:02	1													14		16								
*01:25	1		3															18						
*01:26		2	3									12											23	
*01:27	1		3					8										17						
*01:28	1		3										13		15									
*01:29:01	1		3		5	6																		
*01:29:02	1					6																		
*01:30	1		3															17						
*01:31	1		3			6						12												
*01:32	1		3																			21		
*01:33N																				20				
*01:34		2	3																		21		23	
*01:35			3		5																		23	
*01:36	1		3																			22		
*01:39N	1			4			7													19				
*01:40N	1		3																	19				
*01:42				4			7																23	
*01:44		2	3								11												23	
*01:50	1		3					8											18					
*01:51	1												13	14										
*01:52N	1		3							10												22		
*01:54		2												14		16							23	
*01:55	1				5										15									
*13:165																		17						
*14:112								8																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control