

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	
DRB1 allele ^{1,2,3}																																																	
*13:01:01-13:01:02, 13:01:04-13:01:06, 13:01:08, 13:01:10-13:01:16, 13:117, 13:121, 13:125, 13:127, 13:130-13:131, 13:138, 13:153, 13:173, 13:186-13:187 ⁶	1	2				6		8		10		12				16											27	28		30														44					
*13:01:03	1	2				6		8		10		12				16						23					27	28		30														44					
*13:01:07	1	2				6		8		10		12				16											27	28																					
*13:01:09, 13:177	1	2				6		8		10		12				16												28		30														44					
*13:02:01, 13:02:03-13:02:08, 13:02:11, 13:123, 13:126, 13:143, 13:145, 13:155, 13:168, 13:171:02	1	2				6	7		9			12				16											27																		44				
*13:02:02	1	2				6	7		9			12					17										27					32												44					
*13:02:09	1	2				6	7		9			12				16											27																						
*13:02:10, 13:140	1					6	7		9			12				16											27																						
*13:03:01-13:03:06, 13:194	1	2		4	5				9			12	13												25							32																	
*13:04	1	2		4		6				10		12		14											25		27	28		30																			
*13:05:01-13:05:02	1		3		5		7		9																	26	27																						
*13:06	1	2			5			8		10																	27	28																					
*13:07:01-13:07:02	1		3		5				9				13												25																								
*13:08	1	2				6				10		12	13	14		16									25					30														44					
*13:09, 13:129	1	2				6		8		10	11																27	28																					
*13:10	1	2			5			8		10		12															27	28					32												44				
*13:11:01-13:11:02	1		3		5					10															25		27	28																					
*13:12:01-13:12:04, 13:118	1	2		4	5				9				13												25																								
*13:13	1	2		4	5				9				13					18			?		?		25								?			?													
*13:14:01-13:14:03	1		3		5				9																25		27																						
*13:15	1	2				6		8		10		12			15	16											27	28	29	30																44			
*13:16	1	2				6						12				16			19								27																			44			
*13:17														14					20									28		30																			
*13:18, 13:144	1		3		5			8		10								18								26	27	28																					
*13:19	1	2				6				10		12	13	14	15	16									25				29	30															44				
*13:20	1					6		8		10	11	12				16											27	28		30																			
*13:21:01-13:21:02	1		3	4	5				9																25		27																						
*13:22	1	2				6				10		12		14											25		27	28		30																			
*13:23:01-13:23:02	1	2				6			9			12		14											25		27																						
*13:24	1		3			6				10		12		14											25		27	28		30																			
*13:25	1				5				9		11														25		27																						
*13:26:01	1		3		5		7		9			12	13		15		17																32																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	

Negative Control

*13:26:02	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:27	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:28	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:29	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:30	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:31	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:32	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:33:01	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:33:02-13:33:03	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:34	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:35	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:36	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:37	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:38	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:39	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:40	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:41	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:42	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:43	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:44	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:45	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:46	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:47	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:48	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:49	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:50:01	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:50:02-13:50:03	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:51	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:52	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:53	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:54	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:55	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:56	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:57	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:58	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:59	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:60	1	2	3	4	5	6	7	8	9	10																																						

*13:64	1	2			6			10	12	14	16				23		27	28	30									44																				
*13:65	1	2		4		6	7		9		12	13		16												37			44																			
*13:66:01-13:66:02	1	2		4	5				9		12						25	27			32																											
*13:67						7					16			20															44																			
*13:68	1	2				6			10	12			16			22		27	28	30									44																			
*13:69	1	2				6		8	10	12			16					27	28	30								41																				
*13:70	1	2				6			10	12	13	14	16					25		27	28	30																										
*13:71	1					6		8	10	11	12		16			21		27	28	30																												
*13:72	1	2				6			10	12	13	14	16					25			30	31							44																			
*13:73	1	2				6	7		9		12		16					27											44		47																	
*13:74	1	w				6	7		9		12		16		19			27																														
*13:75	1		3	4		6			10	12	14	16					25	27	28	30																												
*13:76	1	2				6			10	12	13	14	16			23		25		30									44																			
*13:77	1	2			5			8	10	12			17			22	24		27	28		32																										
*13:78	1					6		8	10	11	12		16				24		27	28	30																											
*13:79	1	2				6		8	10	12		16				24		27	28	30									44																			
*13:80	1	2				6		8	10	12		16						27	28	30			34						44																			
*13:81	1	2		4	5				10	12	13							25				32	34																									
*13:82	1	2			5				9									25	27																													
*13:83	1	2				6			10	12	14	16						25	27	28	30								44																			
*13:84	1	2				6			10	12	13		16					25			30								44																			
*13:85	1	2			5		7		9		12	13		15							29		32						44																			
*13:86	1				5				9				15					25	27																													
*13:87	1	2				6		8	10	12		16		18	19				27	28	30								44																			
*13:88	1			4	5				9		11	12	13					25				32																										
*13:89	1	2		4	5				10	12	13							25				32																										
*13:90	1	2		4	5				9		12	13						25				32			36																							
*13:91	1	2				6		8	10	12		16						27	28	30						40			44																			
*13:92, 13:98 ⁵	1	2				6		8	10	12		16						27	28	30				36					44																			
*13:93	1	2		4		6		8	10	12		16						27	28	30					37				44																			
*13:94:01	1	2		4	5				10	12	13		15					25				32				38																						
*13:94:02	1	2		4	5				10	12	13				18			25				32																										
*13:95	1	2		4	5				9		12	13						25				32					39																					
*13:96:01	1	2				6	7		9		12		15	16					27							38			44																			
*13:96:02	1	2				6	7		9		12		16		18				27										44																			
*13:97:01	1	2				6			9		12		16					25	27										44																			
*13:97:02	1	2				6			9		12	14						25	27										44																			
*13:99	1	2				6	7		9		12		16						27								40			44																		
*13:100	1		3		5				9							22	24	25	27																													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:101	1	2		4	5				9		12	13							25				32							40																		

Negative Control

*13:102	1	2			6		8		10	12				16						27	28		30					39			44			Negative Control															
*13:103						7								16			21													44																			
*13:104	1	2			6	7		9		12			15	16						27		29									44																		
*13:105	1	2			6		8		10	12				16			22			27	28		30			35					44																		
*13:106	1	2			6				10	12				16			19				27	28		30							44																		
*13:107	1	2			6			9		12		14		16							25		27								44																		
*13:108	1		3	4	5				10		13										25																												
*13:109	1	2			6					12				16							27										44																		
*13:110	1	2					8		10												27	28		30							44																		
*13:111	1		3		6		8		10	12				16							26	27	28		30																								
*13:112	1	2			6		8		10	12				16							27	28		30						42	44																		
*13:113N	1	2			6		8		10	12				16							27	28		30			36		40		44																		
*13:114	1				6		8		10	12				16							27	28		30																									
*13:115	1	2		4	5		7		9		12	13													32					41																			
*13:116												14					20							30																									
*13:119	1				5		7		9		11					18					27																												
*13:120	1	2		4	5		7		9		12	13													32			37			44																		
*13:122		2		4	5				9		12	13								25					32																								
*13:124	1	2			6	7		9		12				16							27										43	44																	
*13:128	1	2			6	7		9		12				16							27										42	44																	
*13:132	1		3		5				9											25	26	27																											
*13:133	1			4	5				9		12	13								25						32																							
*13:134	1			4	5				9		11		13							25																													
*13:135, 13:182	1	2			6			9		12	13	14		16						25												44																	
*13:136	1		3		5		7		9								23				26	27																											
*13:137N	1	2					8		10	12												27	28								44		46																
*13:139	1	2		4	6	7		9		12				16								27						37			44																		
*13:141	1	2			6		8		10	12				16							27	28		30							44	45																	
*13:142N	1	2			6	7		9		12				16								27									44		46																
*13:146	1	2			5	6		8		10						18						27	28		30						44																		
*13:147	1	2			6	7		9		12				16								27									44	45	46																
*13:148	1	2			6		8		10	12				16								27	28		30						44	45	46																
*13:149	1				6				10	11	12	13	14							25				30																									
*13:150	1		3		5					10										25	26	27	28																										
*13:151	1	2		4	5			9		12	13										25	26	27	28					32																				
*13:152	1	2		4	5			9		12	13									25					31	32																							
*13:154	1	2		4	5			9								18				25		27																											
*13:156	1	2			5		8		10							18						27	28																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33		34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33		34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:157	1	2			5		7		9													27																											
*13:158	1		3		5		7		9								18								26	27																							
*13:159		2			6	7		9		12	13			16																																			

Negative Control

*13:160							8							16				21					28	30								44																
*13:161	1	2		4	5			9											24	25																												
*13:162	1	2				6			10	12		14						22	24	25		27	28		30																							
*13:163	1	2			5		8		10	12					17				23	24			27	28			32		34																			
*13:164	1		3	4	5				10			13			18						25																											
*13:165, 13:171:01, 11:20 ⁴		2				6	7		9		12			16								27										44																
*13:166	1	2				6		8		10	12			16						24			27	28		30				39		44																
*13:167	1	2		4					9		?	13									25					32																						
*13:169	1		3	4	5					10											25		27	28			32																					
*13:170	1	2			5		7		9		12											27				32						44																
*13:172	1	2			5		7		9		12											27				32					41																	
*13:174	1	2		4	5				9		12	13						23								32																						
*13:175													14	16			20						28	30								44																
*13:176	1	2			5		8		10	12							21					27	28			32						44																
*13:178	1				5		8		10	12					17								27	28			32		34																			
*13:179		2				6	7		9		12			16									27					33				44																
*13:180	1				5				9		11	12	13					22		25						32																						
*13:181	1	2			w		8		10	12					17				24			27	28			32																						
*13:183							8							16									28		30							44																
*13:184	1					6	8		10	12				16									27	28																								
*13:185N	1	2				6	8		10	12				16						24	25	27			30							44																
*13:188	1	2		4				9			12	13									25					32																						
*13:189	1		3		5		7		9			13									26									41																		
*13:190	1	2				6	8		10	12				16									27	28		30						43	44															
*13:191		2				6	8		10	12	13			16											30							44																
*13:192	1					6	8		10	11													27	28																								
*13:193	1				5		8		10								23						27	28																								
*13:195													14	16			20									29						44																
*13:196	1				5		7		9		12											27				32																						
*01:42, 01:61																									30																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Negative Control

[illegible]

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
*03:05:03	1						7	8	9	10								18				21	22	23	24			27	28					33	34	35	36	37	38	39	40									
*03:10								8		10						15												27	28																					
*03:12	1			4				8		10						15						?		?				27	28							?		37		?										
*03:13:01	1							8		10						15								24				27	28																					
*03:13:02	1							8		10								18						24				27	28																					
*03:14	1						7		9																			27																						
*03:15, 03:78	1							8		10																		27	28																					
*03:16	1							8		10						15						?		?				27	28							34	?			?										
*03:17, 03:35	1								9				13					18								25																								
*03:19	1							8		10						15												27	28									38												
*03:20	1							8		10						15													27	28																				
*03:26	1							8		10			13			15													27	28																				
*03:31, 03:75	1									10			13			15											25																							
*03:33	1							8		10						15												27	28																					
*03:37, 03:56	1							8		10						15													27	28																				
*03:38	1						7		9				13			15								22		24																							45	
*03:40	1								9							15											25	27																						
*03:41	1								9				13			15											25	25																						
*03:42, 03:69, 03:87, 03:92, 03:111, 15:32								8																																										
*03:43	1									10						15									23			27	28																					
*03:44	1									10						15												27	28																					
*03:52	1									10						15													27	28																				
*03:53	1						7		9							15																																		
*03:57, 03:63	1															15												27																						
*03:71	1							8		10						15								23				27	28																					
*03:73	1							8		10						15		17						23				27	28																					
*03:76	1				5			8		10		12				15							21					27	28							32														
*03:79	1							8		10						15								22		24		27	28																					
*03:81	1						7	8	9	10									18									27																						
*03:83	1							8		10						15									24			27	28																					
*03:85	1							8		10			13			15							22																											
*03:88	1						7		9				13			15									24																									
*03:96	1							8		10						15							22					27	28																					
*03:97	1							8		10														23				27	28																					
*04:02:01-04:02:03, 04:37, 04:124, 04:133-04:134, 04:138, 04:143, 04:161																																																		
*04:14, 04:106, 04:132																																																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
*04:41																																																	41	

*04:77, 04:102, 04:113, 16:08					7																												41															
*08:09, 08:21, 11:22, 14:15, 14:73																																																
*08:20	1		3		5						10		13				18						25																									
*08:32, 12:11, 12:22, 12:38-12:39, 12:53																													29																			
*08:40, DRB3*02:27																	20																															
*11:01:01-11:01:02, 11:01:04, 11:01:06, 11:01:08-11:01:09, 11:01:13-11:01:17, 11:01:20-11:01:25, 11:12:01, 11:29, 11:32-11:33, 11:61-11:62:01, 11:66, 11:72, 11:74:01-11:75, 11:80-11:81, 11:90-11:91, 11:94-11:95, 11:97, 11:99, 11:101, 11:106, 11:108-11:112, 11:117, 11:121, 11:126, 11:128, 11:133-11:134, 11:140-11:142, 11:144, 11:147:01-11:147:02, 11:150, 11:152, 11:154-11:155, 11:157-11:158, 11:162, 11:166-11:167, 11:169N, 11:172, 11:175																													25				27															
*11:01:03, 11:01:11			3		5				9		12						17						25		27		32																					
*11:01:05			3		5				9										23				25		27																							
*11:01:07			3		5				9														25																									
*11:01:10			3		5				9		12				17				23				25		27		32																					
*11:01:12, 11:10:01-11:10:02, 11:12:02, 11:39, 11:62:02			3		5				9														25		26 27																							
*11:01:18-11:01:19, 11:143					5				9														25		27																							
*11:02:01-11:02:02, 11:65:01-11:65:02, 11:70			2				6				10		12		14								25		27 28		30																					
*11:03, 11:63:01, 11:176			3				6				10		12		14								25		27 28		30																					
*11:04:01-11:04:02, 11:04:05-11:04:06, 11:04:08-11:04:12, 11:35, 11:38, 11:46:02, 11:54:02, 11:60, 11:77-11:78, 11:92, 11:102:01-11:102:02, 11:123, 11:135, 11:137, 11:146, 11:156, 11:163, 11:165					3				5								10												25				27 28															
*11:04:03, 11:54:01			3		5						10								23				25		27 28																							
*11:04:04, 11:43, 11:46:01			3		5						10												25		26 27 28																							
*11:04:07			3		5						10		12				17						25		27 28		32																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*11:06:01, 11:06:03, 11:47, 11:129			3		5						10														25		27 28						34															
*11:06:02			3		5						10								23				25		27 28						34																	

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]