

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
DRB1 allele ^{1,2,3}																																																		
*14:01:01	1	2			6							12	13				18				22				26							33						39		42										
*14:01:02-14:01:04, 14:86, 14:153 ⁵	1	2			6							12	13				18				22				26							33						39		?										
*14:02:01			3	4	5																22																													
*14:02:02, 14:47			3	4	5																22																													
*14:03:01-14:03:02				4	5		7														22																													
*14:04:01								8					13							20													34	35						42										
*14:04:02, 14:120								8					13							20													34	35						?										
*14:05:01	1				6				9			12				16	18				22																38		42											
*14:05:02	1				6				9			12				16	18				22																													
*14:05:03	1				6				9			12					18				22																	38		?										
*14:05:04, 14:127:02	1				6				9			12				16	18				22																	38		?										
*14:06:01			3	4	6																22									30																				
*14:06:02-14:06:03			3	4	6																22									30																				
*14:07:01	1	2			5							12	13				18				22				25							33							39		42									
*14:07:02	1	2			5							12	13				18				22				25							33							39		?									
*14:08	1	2			6							12	13			16	18				22											33							39		?									
*14:09			3		5												18				22																													
*14:10										10			13																																					
*14:11																16													28				34	35																
*14:12:01-14:12:02				4		6	7														22									30																				
*14:13			3	4	5																21	22										32																		
*14:14	1				5							12				16	18				22																													
*14:15																		19	20															35																
*14:16		2			6						11	13					18	19			22												33																	
*14:17			3		6												18					23								30																				
*14:18	1			4	6							12				16					22									30						36														
*14:19			3	4	5												17				22												32																	
*14:20				4	6																22																													
*14:21, 03:76, 13:178 ⁴			3		6												17	18				23								30		32																		
*14:22		2			5						11		13	14			18				22				25								33																	
*14:23:01-14:23:04	1				6							12				16	18				22																													
*14:24				4	5																21	22																												
*14:25		2			5						11			14			18	20			22				25								33																	
*14:26, 14:110 ⁵	1	2			6							12	13				18				22			24								33																		
*14:27				4	5		7							14							22																													
*14:28								8					13		15					20														34	35															
*14:29			3	4	6												15				22									30																				
*14:30			3		5													18				23																												
*14:31								8				11	13							20														34	35															
*14:32:01-14:32:02	1	2			6							11	13				18				22												33																	
*14:32:03	1	2			6							11	13				18				22												33																	
*14:33			3		6							12					18					23								30					36															
*14:34	1	2			6							11	13			16	18				22												33																	
*14:35	1	2			6							12	13				18					23											31	33																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
*14:36	1				5							12				16		18				22		24																39		?									
*14:37						6			9									18			21	22																38			?										
*14:38:01-14:38:02, 14:112, 14:114, 14:117-14:118, 14:125, 14:131, 14:139, 14:147, 14:149, 14:151	1	2				6						12	13					18				22												33						39		?									
*14:39	1	2				6				10		12	13					18				22													33								?								
*14:40				4	5		7												19			22																						?							
*14:41				4	5																	22		24																				?							
*14:42	1				5							12				16		18					23																		39		?								
*14:43	1					6			9			12				16	17	18				22																		38			?								
*14:44:01, 14:44:03	1				5							12				16		18				22				25																	38			?					
*14:44:02	1				5							12						18				22				25																		38			?				
*14:45	1					6	7		9			12				16		18				22																					38			?				45	
*14:46			3																								27																								
*14:48			3	4	5																	22							29																?						
*14:49		2		4	5						11			13								22				25									33										?						
*14:50								8					13							20																34	35								?						
*14:51			3	4	5							12										22																								?					
*14:52			3																											30							35								?						
*14:53		2			5						11			14				18		20			23		25							31			33										?						
*14:54:01-14:54:03, 14:54:05	1	2				6						12	13					18				22													33								39			42					
*14:54:04	1	2				6						12	w					w				22													33								39			42					
*14:55	w	2				6	7						13					18	19			22				25																			?						
*14:56	1					6			9			12				16		18				22				25																	38			?					
*14:57										10									19																											42					
*14:58	1	2				6						12						18		20		22													33								39			?					
*14:59	1					6						12				16		18				22					26				30						36								39			?			
*14:60	1	2				6						12	13					18				22							29							33								39			?				
*14:61								8					13																																	?					
*14:62	1	2				6			9			12	13					18				22														33								39			?				
*14:63				4	5		7															21	22													32								?					47		
*14:64	1					6						12				16		18					23													36							39			?					
*14:65	1					6			9									18				21		23																				39			?				
*14:67				4	5		7																22																								?				47
*14:68:01-14:68:02								8					13							20						25											34								?						
*14:69		2			5						11							18		20		22				25										33								?							
*14:70	1	2				6						12	13									22														33								39			?			44	
*14:71								8					13							20																	34	35						40		?					
*14:72	1	2				6						12	13			16		18					23													31								39			?				
*14:73								8					13	14						20																	35							?							
*14:74		2				6					11		13					18				22														33								?							
*14:75, 14:162 ⁴	1	2				6						12	13					18				22														33							39		41		?				
*14:76								8												20																		35							?						
*14:77				4	5		7												19				22		24																			?							
*14:78				4		6	7		9													21	22																												

Negative Control

*14:80			3		6									18			22						30							?																		
*14:81	1			4	6												22						30						39		?																	
*14:82	1	2			6						12	13			18			22							33			39		?	42	43																
*14:83			3	4	6												22						30				33		39		?		44															
*14:84					6	7		9						18	19		22			25										?																		
*14:85				4	5	7											21	22							32					?																		
*14:87	1	2			6							13		18			22								33			39		?			46															
*14:88	1	2			6						12	13		18			22	23							33			39		?			46															
*14:89				4	5	7											22													?			46															
*14:90	1	2			6						12	13		18			22	24							33			39	41	?																		
*14:91	1				6			9			12			16	18		22											38		41	?																	
*14:92N	1	2			6						12	13			18		22								33			39		?			46															
*14:93							8					13				20			25			28				34				?																		
*14:94				4	5												22													41	?																	
*14:95	1				6						12			16	18			23					30				36		39		?	43																
*14:96	1				6			9			12			16	18		22			26				30				36		38		?																
*14:97	1	2			6						12	13		18			22						29			33			39	41	?																	
*14:98				4	5												22													?			45															
*14:99	1	2			6						12	13		15	18		22									33			39		?																	
*14:100	1				6			9						16	18		22											38		?																		
*14:101	1	2			6						12				18		22												39		?	43																
*14:102					5	7										20	22													?																		
*14:103	1				6			9			12				18		22											38		41	?																	
*14:104, 14:111		2			6							13			18		22						29			33			38		?																	
*14:105		2			6					11		13	14		18		22									33				?																		
*14:106			3	4	6						12						22						30				36			?																		
*14:107							8					13				20													40	?																		
*14:108			3	4	6												22						30				35			?																		
*14:109			3	4	5									17			22								32			36			?																	
*14:113	1	2			6						12	13			18		22									33	35		39		?	44																
*14:115				4	5	7											22													?																		
*14:116					5	7									18	20	22													?																		
*14:119		2			5					11		13			18		22			25					33					?																		
*14:121			3		6												22						30						39		?																	
*14:122	1	2			6	7					12	13			18		22									33				?																		
*14:123	1				6									16	18		22												39		?																	
*14:124	1	2			6						12	13					22									33			39		?																	
*14:126:01-14:126:02														16													34	35			?																	
*14:127:01	1				6			9			12				18		22												38		?			45														
*14:128		2			6					11			14		18	20		23						31		33					?																	
*14:129	1	2			6						12	13			18		22									33			39	41	?			46														
*14:130	1	2			6			9			12	13		16	18		22													?																		
*14:132	1	2			6						12	13			18			23						30	31	33		36		39		?	43															
*14:133	1	2			6						12	13			18		22							30		33		36		39		?																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*14:134	1				5						12				16	18			23														36		39		?											
*14:135				4	5	7												22														?																

Negative Control

*14:136						6	7		9							18	19		22		25							38		?																		
*14:137N		2				6					12	13				18			22							31		33			42																	
*14:138								8				13					20									31		34	35		?																	
*14:140		1	2			6					12	13				18			22								33				?																	
*14:141																						27								42																		
*14:142		1	2								12	13				18			22								33			39	?																	
*14:143									10			13									25										?																	
*14:144					5		7									18			22																													
*14:145								8				13					20				26							34	35		?																	
*14:146		2				6					12	13				18			22								33				?																	
*14:148												13										28					34	35		?																		
*14:150		1	2			6					12	13				18											33			39	?																	
*14:152N								8				13					20											35		42																		
*14:154		1		4		6					12			16					22		26		30					36		39	?																	
*14:155		1				6					12			16					22										39	?		44																
*14:156				4		6	7												22												?																	
*14:157		1	2			6			10		12	13				18			22								33			39	?																	
*14:158		1	2			6					12	13				18			22				29			33			39	?			46															
*14:159				4	5														22												?																	
*14:160		1		4		6					12			16					22										39	?																		
*14:163		1	2			6					12	13			17	18			22								33			39	41	?																
*01:02:01-01:02:05, 01:02:07-01:02:11, 01:20:02, 01:23, 01:26, 01:34, 01:43-01:44:02, 01:46, 01:54, 01:57, 04:06:05, 15:01:03, 15:02:06														16																																		
*01:14									11												25																											
*03:01:01-01-03:01:16, 03:01:18-03:01:22, 03:01:25, 03:07, 03:19, 03:22-03:23, 03:28, 03:32, 03:34, 03:36-03:37, 03:39, 03:45-03:48, 03:50-03:51, 03:54-03:56, 03:59, 03:61-03:62, 03:64, 03:66-03:68N, 03:70, 03:72-03:73, 03:77-03:78, 03:83-03:84, 03:86, 03:89, 03:91, 03:93-03:95, 03:98, 03:100:01-03:100:02, 03:104, 03:106-03:110, 03:113-03:114, 03:116-03:117, 03:121, 13:01:07, 13:20, 13:71, 13:111, 13:114, 13:184						6										18			23						30																							
*03:01:17					6									16	18				23							30					43																	
*03:01:23, 03:16, 03:82					6											18										30																						
*03:01:24, 03:112					6											w				23						30																						
*03:02:01-03:02:02, 03:74, 03:102-03:103, 03:115				4	5														22																													
*03:03				4		6													22							30																						
*03:04:01-03:04:02					6											18			23												41																	
*03:05:01-03:05:03, 03:09, 03:40, 13:02:10, 13:14:01-13:14:03, 13:23:01-13:23:02, 13:25, 13:29, 13:56, 13:62-13:63, 13:74, 13:82, 13:97:01-13:97:02, 13:132, 13:140						5										18			23																													
*03:06, 03:80					6											18			22							30																						
*03:08, 11:40, 11:59					6											18			23							30				37																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*03:10		2				6							13					18					23						30	31		33																
*03:11:01						6					12							18					23						30					36														
*03:12						6			9									18			21		23						30		32																	

Negative Control

[illegible]

[illegible]

*11:04:07	1				6									14			18			23									37					
*11:09, 11:28:01-11:28:02, 11:87					5												18			23									37					
*11:13:01-11:13:02	1				6										16		18			23									37	39		42		
*11:17	1				6					12					16		18			22									37	39				
*11:20					5												18			23								36	37					
*11:23:01-11:23:02, 11:64					5		7										18		20	23									37					
*11:25, 11:55					6		7										18		20	23									37					
*11:27:01					5												18			23									37				45	
*11:29:01					5												18			23									37			41		
*11:31, 11:96		2			5												18			23						31			37					
*11:37:01-11:37:02, 11:107, 11:145, 11:170					5												18			22									37					
*11:41, 11:48, 11:56, 11:171					6												18			22									37					
*11:44, 11:85, 11:127					6												18			23						29			37			41		
*11:45					5		7										18		20	23									37					47
*11:47, 11:63:01-11:63:02, 11:165					6												18			23									37			41		
*11:49:01-11:49:02					5												18			23						29			37			41		
*11:50					6												18			22						29			37			41		
*11:52	1	2			6					12	13				16		18			22									37	39				
*11:53, 11:136, 11:150					5					12							18			23									37					
*11:54:01					6					12					16		18			23									37					
*11:54:02					6					12							18			23									37					
*11:58:01, 11:113, 11:115, 11:182					6							14					18			23						30			37					
*11:58:02					6						14						18			23									37					
*11:69, 11:82					5												18		21	23									37					
*11:73, 11:79, 11:93					5											17	18			23									37					
*11:76					6												18			23	24								37					
*11:83					6						14						18			22						30			37					
*11:84:01, 11:124, 11:138					6												18			23									37				45	
*11:86																	18			23									37					
*11:88					6												18												37					
*11:89																																		

[illegible]

*13:60	1			5											18		22											41																				
*13:61:01				6											18		23				30						36					45																
*13:64				6											18	19	23	24																														
*13:65				5											18		21	22				32				36																						
*13:66:01-13:66:02				5											17	18		21	23																													
*13:67, 13:103, DRB3*01:08																											36																					
*13:72				6			9								18	19		22										38																				
*13:76				6				12					16		18	19		22																														
*13:77	1			6											18			23				29	30			36																						
*13:81, 13:89, 13:94:01				6			9								17	18		21	22																													
*13:83				6											18	19		23																														
*13:84				6											18			22											41																			
*13:85		3	4	5											17			22					32			36																						
*13:86				5														23																														
*13:92				6											18			23	24				30			36																						
*13:93				6			9								18		21	23				30	32			36																						
*13:94:02				6			9								17	18		21	22													45																
*13:95				5											17	18		21	22			29										46																
*13:96:02				5											18			23								36						45																
*13:101				5											17			21	22																													
*13:104			4	5														23								36																						
*13:107				5											18	19		23																														
*13:119				5		7									18			23																														
*13:120		3		5											17	18		21	22				32			36																						
*13:135, 13:182				5											18	19		22																														
*13:136				5								14			18			23	24																													
*13:139				5											18		21	23					32			36																						
*13:146, 13:156				6	7										18			23				30				36							47															
*13:154				5	7										18	20	21	23															47															
*13:158				5	7							14			18			23																														
*13:159		2		5					11		13				18			22			25					33	36																					
*13:160, 13:183																						30				36																						
*13:161				5											18		21																															
*13:163	1			6							15	16			18			23				30				36																						
*13:164				6	7		9								18		20	21	22																													
*13:170		3		5											17	18			23				32			36																						
*13:171:01		2		5											18			23								36																						
*13:172				5											17	18		23																														
*13:174				5											17	18		21	22	24																												
*13:175															19						28					35			40																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*13:179		2			5						11		13				18						23		25						31		33		36													
*13:180				5													17	18				22						29																				
*13:181	1				6												18					23						30																				
*13:188				5							12						17	18			21	22																										
*13:189				5									14				18				22																											
*13:191		2			6						11		13				18				22								30			33		36														

Negative Control

pl

[illegible]