

[illegible]

*15:47	1	2	3		5												
*15:48		2	3	4								12					
*15:49	1			4													
*04:73, 11:82, 13:09, 14:37			3														
*11:88												12					
*16:01:01-16:03, 16:08, 16:11, 16:13N-16:14, 16:16					5		7										
*16:04					5		7										
*16:05:01-16:05:02, 16:07					5		7								15		
*16:09-16:10	1				5												
*16:15				4			7										
DRB1 allele																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	

INTERPRETATION TABLE																
DRB1*15 SSP subtyping																
Amplification patterns of the DRB1*15:01 to 15:49 alleles																
Well ⁴																
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
160	205	175	165	165	165	170	200	70	180	170	255	190	165	215	155	Length of spec. PCR product(s)
		225	200		200	220		195					220		200	Length of int. pos. control ¹
430	515	430	430	430	430	430	430	430	430	430	430	430	430	430	430	5'primer(s) ²
30(175)	30(175)	13(126)	13(126)	30(175)	13(126)	13(126)	15(131)	15(130)	13(126)	23(155)	13(126)	13(126)	13(126)	13(126)	13(126)	
5' -gAT ^{3'}	5' -gAT ^{3'}	5' -Agg ^{3'}	5' -Agg ^{3'}	5' -gAC ^{3'}	5' -Aag ^{3'}	5' -Agg ^{3'}	5' -gTA ^{3'}	5' -AgA ^{3'}	5' -Agg ^{3'}	5' -gCA ^{3'}	5' -Agg ^{3'}	5' -Agg ^{3'}	5' -Agg ^{3'}	5' -Agg ^{3'}	5' -Agg ^{3'}	
					15(131)		19(143)	58(259)								
					5' -gTA ^{3'}		5' -CAg ^{3'}	5' -ACA ^{3'}								
					25(161)		21(148)									
					5' -gCA ^{3'}		5' -ggT ^{3'}									
70(295)	84(337)	57(258)	55(251)	71(298)	67(286)	57(257)	71(298)	67(286)	60(266)	67(286)	85(341)	62(273)	55(250)	72(301)	51(239)	3'primer(s) ³
5' -CTC ^{3'}	5' -CCT ^{3'}	5' -gCC ^{3'}	5' -gCA ^{3'}	5' -CgC ^{3'}	5' -gAT ^{3'}	5' -CgA ^{3'}	5' -CgC ^{3'}	5' -gAT ^{3'}	5' -Agg ^{3'}	5' -gAT ^{3'}	5' -CAG ^{3'}	5' -CTC ^{3'}	5' -CCA ^{3'}	5' -CCC ^{3'}	5' -CCC ^{3'}	
		74(307)	67(286)			73(305)							73(305)		67(287)	
		5' -CAg ^{3'}	5' -gAg ^{3'}			5' -ggC ^{3'}							5' -CgA ^{3'}		5' -ggg ^{3'}	
		77(317)														
5' -AgT ^{3'}																
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No. DRB1 allele
160	205	175	165	165	165	170	200	70	180	170	255	190	165	215	155	Length of spec. PCR product(s)
		225	200		200	220		195					220		200	Well No. DRB1 allele
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	*15:01:01:01-15:01:12

[illegible]

															32	*15:47
																*15:48
															32	*15:49
																*04:73, 11:82, 13:09, 14:37
																*11:88
																*16:01:01-16:03, 16:08, 16:11, 16:13N-16:14, 16:16
		19														*16:04
																*16:05:01-16:05:02, 16:07
																*16:09-16:10
																*16:15
																DRB1 allele
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.