

[illegible]

[illegible]

*15:42	1	2	3	4													
*15:43	1	2	3	4													
*15:44	1	2	3		5												
*15:45	1	2	3	4													
*15:46	1	2	3	4													
*15:47	1	2	3		5												
*15:48		2	3	4								12					
*15:49	1			4													
*15:50N	1			4													
*15:54, 15:66	1			4				8									
*15:55	1	2	3	4													
*15:57	1	2		4		6											
*15:58	1	2			5												
*15:59		2	3	4													
*15:63	1				5			8									
*15:65	1	2	3	4													
*15:69			3						9								
*04:73, 04:105, 11:82, 13:09, 13:129, 14:37			3														
*11:88												12					
*16:01:01-16:03, 16:08, 16:11, 16:13N-16:14, 16:16- 16:17, 16:19					5		7										
*16:04, 16:18					5		7										
*16:05:01-16:05:02, 16:07					5		7								15		
*16:09-16:10	1				5												
*16:15				4			7										
DRB1 allele																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	

INTERPRETATION TABLE																	
DRB1*15 SSP subtyping																	
Amplification patterns of the DRB1*15:01 to 15:69 alleles																	
Well ⁶																	
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
160	205	175	165	165	165	125	200	70	180	175	135	200	165	215	155	Length of spec. PCR product(s)	
		225	200		200	170		195			255		220		200		
						220											
430	515	430	430	430	430	430	430	430	430	430	430	430	430	515	430	Length of int. pos. control ¹	
29(175)	29(175)	13(126)	13(126)	29(175)	13(126)	13(126)	15(131)	14(130)	13(126)	20(146)	13(126)	13(126)	13(126)	13(126)	13(126)	5'primer(s) ²	
5'-gAT ^{3'}	5'-gAT ^{3'}	5'-Agg ^{3'}	5'-Agg ^{3'}	5'-gAC ^{3'}	5'-AAg ^{3'}	5'-Agg ^{3'}	5'-gTA ^{3'}	5'-AgA ^{3'}	5'-Agg ^{3'}	5'-TgC ^{3'}	5'-Agg ^{3'}	5'-Agg ^{3'}	5'-Agg ^{3'}	5'-Agg ^{3'}	5'-Agg ^{3'}		
					15(131)		19(143)	57(259)		23(155)							
					5'-gTA ^{3'}		5'-CAg ^{3'}	5'-ACA ^{3'}		5'-gCA ^{3'}						3'primer(s) ³	
					25(161)		20(148)										
					5'-gCA ^{3'}		5'-ggT ^{3'}										
69(295)	83(337)	57(258)	55(251)	70(298)	66(286)	40(208)	70(298)	66(286)	60(266)	66(286)	44(220)	62(273)	54(250)	71(301)	51(239)		
5'-CTC ^{3'}	5'-CCT ^{3'}	5'-gCC ^{3'}	5'-gCA ^{3'}	5'-CgC ^{3'}	5'-gAT ^{3'}	5'-gTg ^{3'}	5'-CgC ^{3'}	5'-gAT ^{3'}	5'-Agg ^{3'}	5'-gAT ^{3'}	5'-CCT ^{3'}	5'-CTC ^{3'}	5'-CCA ^{3'}	5'-CCC ^{3'}	5'-CCC ^{3'}		
		73(307)	66(286)			57(257)					85(341)	69(295)	73(305)		67(287)		
		5'-CAg ^{3'}	5'-gAg ^{3'}			5'-CgA ^{3'}					5'-CAg ^{3'}	5'-Tg ^{3'}	5'-CgA ^{3'}		5'-ggg ^{3'}		
		77(317)				73(305)											
		5'-AgT ^{3'}				5'-ggC ^{3'}											
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.	
160	205	175	165	165	165	125	200	70	180	175	135	200	165	215	155	Length of spec.	
		225	200		200	170		195			255		220		200	PCR product(s)	
						220											
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.	
																DRB1 allele ⁴	

																*15:01:01:01-15:01:17, 15:51-15:53, 15:56, 15:61- 15:62, 15:64, 15:67
																*15:02:01-15:02:08, 15:60, 15:68
				21												*15:03:01:01-15:03:02
																*15:04
																*15:05
																*15:06
																*15:07
																*15:08
																*15:09
																*15:10
																*15:11
																*15:12
																*15:13
										27						*15:14
																*15:15
											28					*15:16, 15:35 ⁵
17																*15:17N
																*15:18
	18															*15:19
		19														*15:20
			20													*15:21
				21												*15:22
					22											*15:23
		19				23										*15:24
							24									*15:25
			20													*15:26
					22											*15:27
						23										*15:28
		19														*15:29
																*15:30
																*15:31
																*15:32
			20					25								*15:33
																*15:34
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
160	205	175	165	165	165	125	200	70	180	175	135	200	165	215	155	Length of spec.
		225	200		200	170		195			255		220		200	PCR product(s)
						220										
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
		19														*15:36
																*15:37
								26								*15:38
								25								*15:39
					22		24									*15:40
												29				*15:41

																*15:42
							24									*15:43
												30				*15:44
												30				*15:45
													31			*15:46
															32	*15:47
																*15:48
															32	*15:49
												29				*15:50N
				20												*15:54, 15:66
							23									*15:55
																*15:57
																*15:58
																*15:59
										26						*15:63
															32	*15:65
																*15:69
																*04:73, 04:105, 11:82, 13:09, 13:129, 14:37 *11:88
																*16:01:01-16:03, 16:08, 16:11, 16:13N-16:14, 16:16- 16:17, 16:19 *16:04, 16:18
				19												*16:05:01-16:05:02, 16:07 *16:09-16:10 *16:15
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	DRB1 allele Well No.