

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
DRB1 allele ^{1,2,3}																																
*15:01:01:01-15:01:35, 15:51-15:53, 15:56, 15:62, 15:71-15:72, 15:76, 15:79, 15:81- 15:83, 15:86-15:87, 15:90, 15:92, 15:95, 15:97-15:98, 15:106- 15:109, 15:111, 15:114, 15:116, 15:121, 15:124, 15:127-15:128, 15:132- 15:133, 15:135, 15:139, 15:141, 15:144-15:146, 15:150 ⁶	1	2	3	4																												
*15:02:01:01-15:02:09, 15:02:11-15:02:18, 15:60, 15:99, 15:101, 15:105:01-15:105:02, 15:119, 15:126, 15:131, 15:140, 15:149	1	2	3		5																											
*15:02:10		2	3		5																											
*15:03:01:01-15:03:02	1	2		4		6															21											
*15:04, 15:147	1			4						10																						
*15:05, 15:73	1			4				8																								
*15:06:01-15:06:03, 15:77, 15:110, 15:125 ⁴	1	2	3	4					9																							
*15:07:01-15:07:03		2	3	4			7																									
*15:08	1	2	3		5											16																
*15:09	1	2	3	4							12																					
*15:10	1			4									13		15																	
*15:11		2	3		5		7																									
*15:12	1	2	3	4						10																						
*15:13	1	2	3	4									14																			
*15:14	1	2	3		5	6																					27					
*15:15:01-15:15:03	1				5					10																						
*15:16, 15:35 ⁴	1	2	3	4																								28				
*15:17N	1			4									14																			
*15:18	1	w	w	4													17															
*15:19	1	2	3		5				9																							
*15:20	1	2	3	4													18															
*15:21	1			4											15			19														
*15:22	1	2	3	4															20													
*15:23	1			4				8													21											
*15:24	1	2	3	4																		22										
*15:25	1			4				8											19	20			23									
*15:26	1	2	3		5																			24								
*15:27	1				5			8							15					20												
*15:28			3																			22										
*15:29	1	2	3		5																		23									
*15:30, 15:104:01- 15:104:02 ⁴	1	2	3		5													19														
*15:31	1				5			8																								
*15:32	1	2	3	4									13																			
*15:33	1	2	3	4																						25						
*15:34					5		7	8							15					20												
*15:36	1	2	3	4				8																								
*15:37:01-15:37:02	1	2	3	4														19														
*15:38	1	2	3		5																						26					
*15:39	1	2	3		5																					25						
*15:40	1	2	3	4																	22		24									
*15:41	1	2	3	4																									29			
*15:42, 15:64 ⁴	1	2	3	4																							27					
*15:43	1	2	3	4																				24								
*15:44	1	2	3		5																									30		

Negative Control

*15:45	1	2	3	4																										30		
*15:46	1	2	3	4																										31		
*15:47	1	2	3		5						11																					
*15:48		2	3	4								12																				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*15:49	1			4							11																					
*15:50N	1			4																									29			
*15:54, 15:66:01-15:66:02	1			4				8							15					20												
*15:55	1	2	3	4																			23									
*15:57	1	2		4		6													19													
*15:58	1	2			5																											
*15:59, 15:143		2	3	4																												
*15:61	1	2	3	4																				24	25							
*15:63	1				5			8																		26						
*15:65	1	2	3	4							11																					
*15:67	1	2	3	4																					25		27					
*15:68	1	2	3		5										15																	
*15:69			3						9																							
*15:70, 15:89, 15:91, 16:33, 16:36 ⁵	1			4																												
*15:74	1	2	3												16																	
*15:75, 15:134N ⁴	1	2	3	4																						26						
*15:78	1	2			5	6															21											
*15:80N	1				5																								29			
*15:84, 15:117	1			4									13																			
*15:85	1	2	3																													
*15:88				4			7			10																						
*15:93	1	2	3	4																			22				27					
*15:94	1	2		4		6			9												21											
*15:96		2	3	4			7																					28				
*15:100	1			4				8												19	20											
*15:102	1	2	3	4											16																	
*15:103	1				5								13																			
*15:112	1	2	3	4																20									29			
*15:113N	1	2	3	4					9																25							
*15:115N					5		7											18														
*15:118	1	2	3		5																							28				
*15:120	1			4				8					13		15					20												
*15:122	1				5										15																	
*15:123	1	2	3	4								12					17															
*15:129N	1			4											16														30			
*15:130		2		4		6															21											
*15:136	1	2	3	4											15																	
*15:137N	1	2		4		6															21								29			
*15:138N	1	2	3	4					9													22										
*15:142	1			4						10											21											
*04:73:01-04:73:02, 04:105:01-04:105:02, 11:82, 13:09, 13:129, 14:37				3																												
*04:162, 11:88, 13:177												12																				
*04:226																	18															
*16:01:01-16:01:06, 16:01:08-16:03, 16:08, 16:11, 16:13N-16:14, 16:16-16:17, 16:19-16:22, 16:24-16:32, 16:35, 16:39-16:43, 16:45, 16:47					5		7																									
*16:01:07							7																									
*16:04:01-16:04:02, 16:18, 16:46					5		7										19															
*16:05:01-16:05:02, 16:07					5		7							15																		
*16:09:01-16:10, 16:37	1				5																											

Negative Control

[illegible]