

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele ^{1,2,3}																																	
*15:01:01:01-15:01:41, 15:51-15:53, 15:62, 15:71-15:72, 15:76, 15:79, 15:81-15:83, 15:86-15:87, 15:90, 15:92, 15:95, 15:97-15:98, 15:106-15:109, 15:111, 15:114, 15:116, 15:121, 15:124, 15:127-15:128, 15:132-15:133, 15:135, 15:139, 15:141, 15:144-15:146, 15:148N, 15:150-15:151, 15:153, 15:160, 15:162, 15:164Q, 15:166, 15:168-15:173, 15:180N, 15:182, 15:184	1	2	3	4																													
*15:02:01:01-15:02:09, 15:02:11-15:02:19, 15:60, 15:99, 15:101, 15:105:01-15:105:02, 15:119, 15:126, 15:131, 15:140, 15:149, 15:152, 15:154N, 15:159N, 15:161, 15:167, 15:179	1	2	3		5																												
*15:02:10		2	3		5																												
*15:03:01:01-15:03:05, 15:163N, 15:165, 15:178	1	2		4		6															21												
*15:04, 15:147	1			4						10																							
*15:05, 15:73	1			4				8																									
*15:06:01-15:06:04, 15:77, 15:110, 15:125 ⁴	1	2	3	4					9																								
*15:07:01-15:07:03		2	3	4			7																										
*15:08	1	2	3		5											16																	
*15:09	1	2	3	4								12																					
*15:10	1			4									13		15																		
*15:11:01-15:11:02		2	3		5		7																										
*15:12	1	2	3	4						10																							
*15:13	1	2	3	4										14																			
*15:14	1	2	3		5	6																					27						
*15:15:01-15:15:03	1				5					10																							
*15:16, 15:35 ⁴	1	2	3	4																								28					
*15:17N	1			4										14																			
*15:18	1	w	w	4													17																
*15:19, 15:155 ⁴	1	2	3		5				9																								
*15:20	1	2	3	4																													
*15:21	1			4											15				18		19												
*15:22	1	2	3	4																	20												
*15:23	1			4				8														21											
*15:24	1	2	3	4																			22										
*15:25	1			4				8												19	20			23									
*15:26	1	2	3		5																				24								
*15:27	1				5			8							15						20												
*15:28			3																				22										
*15:29	1	2	3		5																			23									
*15:30, 15:104:01-15:104:03 ⁴	1	2	3		5															19													
*15:31:01-15:31:02	1				5			8																									
*15:32	1	2	3	4									13																				
*15:33	1	2	3	4																						25							
*15:34					5		7	8							15						20												
*15:36	1	2	3	4				8																									
*15:37:01-15:37:02	1	2	3	4																19													
*15:38	1	2	3		5																						26						
*15:39	1	2	3		5																						25						
*15:40	1	2	3	4																		22		24									
*15:41	1	2	3	4																										29			
*15:42, 15:64 ⁴	1	2	3	4																								27					
*15:43	1	2	3	4																				24									
*15:44	1	2	3		5																										30		
*15:45	1	2	3	4																											30		
*15:46	1	2	3	4																												31	
*15:47	1	2	3		5						11																						
*15:48		2	3	4								12																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*15:49	1			4							11																						
*15:50N	1			4																													

Negative Control

[illegible]