

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
DRB1 allele^{1,2,3}																
*16:01:01:01-16:01:18, 16:24, 16:28, 16:32, 16:42, 16:44-16:45, 16:47, 16:49, 16:51, 16:54-16:56, 16:59Q-16:61, 16:63N, 16:67, 16:69, 16:71-16:73	1		3													
*16:01:19	1		w													
*16:02:01:01-16:02:11, 16:20, 16:29, 16:31, 16:39, 16:43, 16:48, 16:50, 16:52, 16:57, 16:62N, 16:64-16:65, 16:70N, 16:74	1	2														
*16:03, 16:30 ⁴	1		3	4												
*16:04:01-16:04:02	1		3		5											
*16:05:01-16:05:02	1					6										
*16:07	1					6	7									
*16:08	1		3					8								
*16:09:01-16:09:02	1		3						9	10						
*16:10:01-16:10:02	1	2							9	10						
*16:11	1	2									11					
*16:12												12				
*16:13N	1		3										13			
*16:14	1	2						8					13			
*16:15, 16:34			3												15	
*16:16	1	2												14		
*16:17	1	2										12				
*16:18	1	2			5											
*16:19	1	2											13			
*16:21N	1	2									11			14		
*16:22	1	2										12		14		
*16:23, 16:38:01-16:38:02		2													15	
*16:25	1		3							11						
*16:26	1							8								
*16:27	1		3											14		
*16:33			3						9	10					15	
*16:35	1	2			6											
*16:36		2							9	10				14	15	
*16:37	1	2							9	10				14		
*16:40	1								9				13			
*16:41N	1	2			5		8									
*16:46	1	2		4												
*16:53, 16:68		2														
*16:58		2							9	10					15	
*16:66					6				9	10					15	
*01:23, 01:11, 04:53:01, 04:99, 12:11, 12:21, 12:38, 12:43, 12:50, 12:53, 12:57, 12:76, 12:79, 12:88, DRB5*01:31, DRB5*01:44, DRB5*02:26N															15	
*07:140							7									
*11:01:03, 11:01:10-11:01:11, 11:08:03, 11:19:02, DRB5*01:41										10				14		
*11:04:07, 11:42:02, 12:04										10				14	15	
*12:01:01:01-12:01:01:14, 12:01:03-12:02:03, 12:02:05-12:03:03, 12:05-12:06, 12:08-12:10, 12:12-12:15, 12:17-12:20, 12:23-12:37, 12:41-12:42, 12:44-12:48, 12:51-12:52, 12:54-12:56, 12:58-12:65, 12:67-12:72N, 12:74N-12:75, 12:77-12:78, 12:80-12:87, 12:89-12:95, 12:97-12:104, 13:42:02, 13:77, 13:163, 13:181										10					15	
*12:07, 12:16:02-12:16:03, 12:39:01-12:40, 12:96, 13:02:02, 13:289N, DRB5*01:34-01:35										10						
*15:01:01:01-15:01:21, 15:01:23-15:01:51, 15:03:01:01-15:06:04, 15:12-15:13, 15:16-15:18, 15:20, 15:22-15:25, 15:32-15:33, 15:35-15:37:01, 15:40-15:43, 15:45-15:46, 15:49, 15:51-15:56, 15:57:02, 15:61-15:62, 15:64-15:67, 15:70-15:77, 15:79, 15:81-15:87, 15:89-15:95, 15:97-15:98, 15:100, 15:102, 15:106-15:114, 15:116-15:117, 15:120-15:121, 15:123-15:125, 15:127-15:129N, 15:132-15:139, 15:141-15:142, 15:144-15:148N, 15:150-15:151, 15:153, 15:157-15:158, 15:160, 15:162-15:166, 15:168-15:171:02, 15:173-15:178, 15:180N-15:187, 15:190-15:191, 15:193, 15:195-15:197, 15:199-15:205, 15:207N-15:209N, 15:211, 15:215-15:217N									9							
*15:02:01:02-15:02:09, 15:02:11-15:02:20, 15:08, 15:14-15:15:03, 15:19, 15:26-15:27, 15:29-15:31:02, 15:38:01-15:39, 15:44, 15:47, 15:58, 15:60, 15:63, 15:68, 15:78, 15:99, 15:101, 15:103-15:104:01, 15:105:01-15:105:02, 15:118:01, 15:119, 15:122, 15:126, 15:131, 15:140, 15:149, 15:152, 15:154N-15:156, 15:159N, 15:161, 15:167, 15:179, 15:188:01:01-15:189, 15:194, 15:206, 15:210, 15:213	1								9							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*15:02:10, 15:11:01-15:11:02, 15:34, 15:115N, 15:118:02	1															
*15:10						6			9							
*15:21					5	6			9							
*15:37:02, 15:57:01				4					9							
*15:50N									9	10						
*15:80N	1								9	10						
*15:104:02-15:104:03, 15:212	1								9							
*15:172, 15:192, 15:214							8	9								
DRB5*01:13														14		
DRB5*01:20, DRB5*01:47, DRB5*01:64, DRB5*02:25N												12		14		
DRB5*02:08, DRB5*02:12												12		14	15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control

Negative Control