

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
DRB1 allele ^{1,2,3}																																
*03:01:01-03:01:24, 03:56, 03:59, 03:61, 03:66, 03:70, 03:73, 03:78, 03:83, 03:86, 03:91, 03:94, 03:98-03:100:02, 03:104, 03:106-03:109, 03:112, 03:114, 03:116	1			4		6			9									18												30		
*03:02:01-03:02:02, 03:74, 03:102-03:103, 03:115		2	3					8		10														24						30		
*03:03		2		4				8		10														24						30		
*03:04:01-03:04:02	1				5	6			9									18												30		
*03:05:01-03:05:03			3			6			9									18												30		
*03:06, 03:60	1			4					9	10																				30		
*03:07				4				8										18												30		
*03:08	1			4		6	7											18												30		
*03:09			3			6			9		11							18						23						30		
*03:10	1			4		6						12						18												30		
*03:11:01	1			4		6			9				13														27			30		
*03:12	1			4		6							14					18												30		
*03:13:01-03:13:02	1			4		6			w		11							18							25					30		
*03:14			3			6			9							16		18												30		
*03:15	1			4		6			9							16		18												30		
*03:16	?			?		w			9				14					18												30		
*03:17										10			13		15												27			30		
*03:18, 03:48 ⁵	1			4		6			9									18					22							30		
*03:19	1			4		6			9									18					23							30		
*03:20, 03:93 ⁵	1			4		6			9								17	18												30		
*03:21								8										18				21								30		
*03:22	1			4		6			9									18	19											30		
*03:23	1			4		6			9		11							18					23							30		
*03:24				4									13														27			30		
*03:25	1					6			9									18		20										30		
*03:26	1			4					9	10																26				30		
*03:27		2	3							10			13											24			27			30		
*03:28	1			4		6			9									18		20										30		
*03:29		2	3					8										18						24						30		
*03:30	1					6			9									18				21								30		
*03:31	1								9	10										20										30		
*03:32	1			4		6			9									18							25					30		
*03:33	1			4		6			9									18								26				30		
*03:34, 03:55	1			4		6			9									18									27			30		
*03:35										10			13														27			30		
*03:36	1			4		6			9									18										28		30		
*03:37	1			4		6			9					15				18												30		
*03:38		2	3					8		10	11													24						30		
*03:39	1			4		6			9									18											29	30		
*03:40, 03:105								8										18											29	30		
*03:41								8		10																			29	30		
*03:42, 03:87	1			4		6			9									18				21								30		
*03:43	1					6			9									18				21					26	27		30		
*03:44	1					6			9									18									26			30		
*03:45	1			4		6			9									18										28		30	31	
*03:46	1			4		6												18									27					
*03:47	1			4		6			9				13					18												30		
*03:49								8										18												30		
*03:50	1			4		6			9									18												30	31	
*03:51	1			4		6			9			12						18									27			30		
*03:52	1					6			9									18											29	30		
*03:53		2	3					8																24			27			30		
*03:54, 03:113				4														18												30		
*03:57, 03:63						6			9									18												30		
*03:58	1			4	5	6			9									18												30		
*03:62	1			4		6			9				13		15			18												30		
*03:64	1			4		6												18														
*03:65	1			4	5	6	7											18														
*03:67N	1			4		6			w									18								25		28		30		
*03:68N	1			4		6			9				14					18		20										30		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*03:69	1			4		6			9									18							25							

Negative Control

*03:71	1			4		6			9							18										26	27		30			
*03:72	1			4		6	7		9							18													30			
*03:75	1								9	10							20											29	30			
*03:76	1			4		6			9			12																	30			
*03:77, 03:84, 03:89, 03:95, 13:144 ⁴	1			4		6			9																				30			
*03:79	1			4		6					11					18													30			
*03:80	1			4					9	10				15															30			
*03:81	1		3	4		6			9				13													27			30			
*03:82	1			4					9							18													30			
*03:85	1			4							10																		30			
*03:88		2	3					8		10	11										24	25							30			
*03:90		2	3							10				16							24								30			
*03:92	1			4		6			9							18												29	31			
*03:96	1			4		6										18													30			
*03:97, 14:33 ⁴			3	4																									30			
*03:101				4				8								18					24								30			
*03:110	1			4		6						12		14		18													30			
*03:111	1			4		6			9							18																
*03:117	1			4		6			w							18													30			
*03:118	1					6			9							18	20											29	30			
*01:16, 01:55, 04:22, DRB3*01:09, DRB3*02:04, DRB3*02:22, DRB3*03:03								8																								
*04:98:01-04:98:02															16																	
*08:20, 13:03:01-13:03:06, 13:07:01-13:08, 13:12:01-13:13, 13:33:01-13:33:03, 13:37-13:38, 13:47-13:49, 13:55, 13:60, 13:70, 13:72, 13:76, 13:84, 13:88-13:90, 13:94:01-13:95, 13:101, 13:108, 13:115, 13:133-13:134, 13:149, 13:151-13:152, 13:164, 13:167, 13:180, 13:182, 13:188, 13:194, 14:05:01-14:05:04, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36-14:37, 14:43-14:45, 14:56, 14:84, 14:91, 14:100, 14:103, 14:116, 14:123, 14:136, 14:155											10																		30			
*09:01:08, 09:02:01-09:02:02, DRB3*01:07, DRB3*01:15									9																							
*09:06	1																															
*09:08														14																		
*11:06:01-11:06:03, 11:21, 11:47, 11:57, 11:129																17																
*11:07, 11:103, 11:125, 11:173								8								18												29				
*11:09, 11:20, 11:28:02, 11:122, 13:103, 13:165, 13:171:01, 13:179, 16:08, DRB3*01:08, DRB3*02:06			3																													
*11:10:01-11:10:02, 11:12:01-11:13:02, 11:30, 12:05, 12:14-12:15, 13:175, 13:195, 14:35, 14:72, 14:138																			21													
*11:16, 11:40, 11:58:01, 11:113, 13:43, 13:160, 13:183, 15:32				4																												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control

Negative Control

[illegible]

*13:19, 14:20, 14:40, 14:94, 14:102, 14:156, 14:159	2								10																			30				
*13:26:01-13:26:02, 13:85, 14:02:01-14:03:01, 14:13, 14:19, 14:24, 14:27, 14:47, 14:51, 14:63, 14:67, 14:85, 14:89, 14:98, 14:135		2	3						10										24								30					
*13:27, 13:71, 13:129, 13:176	1			4		6			9		w																30					
*13:32, 13:40, 14:80				4					10																		30					
*13:34, 13:136			3																26	27							30					
*13:36, 13:65, 13:120, 13:189, 14:09, 14:144			3						10																		30					
*13:39			3						11																		30					
*13:41			3			6			9		w																30					
*13:53, 14:06:01-14:06:03, 14:12:01-14:12:02, 14:78, 14:81, 14:108		2		4					10										24								30					
*13:54, 13:100, 13:162									11																		30					
*13:57		2																21									30					
*13:58, 13:81									10							17											30					
*13:64																		21			26	27					30					
*13:67			3															21														
*13:77-13:79, 13:181				4					11																		30					
*13:83, 13:107, 14:65																		21									30					
*13:86		2																									30					
*13:104, 14:03:02		2	3																24								30					
*13:106																				26							30					
*13:118, 13:135, 14:96									10					15													30					
*13:125				4										17													30					
*13:131				4										15													30					
*13:150														15													30					
*13:159			3						10																							
*13:163				4						11						17											30					
*13:174									10												26	27					30					
*13:191				4					10																							
*14:18, 14:106		2	3	4					10											24							30					
*14:29		2		4					10							17				24							30					
*14:38:01-14:38:02									10				13																			
*14:41, 14:77		2							10												26	27					30					
*14:46, DRB3*02:20			3							10										24												
*14:48, 14:109, 14:115		2	3						10				11							24							30					
*14:49		2							10																							
*14:52				4															24													
*14:59			3	4					10				15														30					
*14:64			3	4						11																	30					
*14:76, 14:79								8																	29	31						
*14:82	1								10				12																			
*14:83		2		4					10																		30					
*14:95	1		3	4		6			9																	30						
*14:97										10											26											
*14:99										10						17																
*14:110										10											26	27										
*14:121				4						10										24							30					
*14:127:01-14:127:02										10				13													30					
*14:132	1		3	4		6				12																						
*14:133			3	4						10																						
*14:154		2	3	4						10				15							24							30				
*15:25								8							18																	
DRB3*01:01:03																				25												
DRB3*01:14										10				11									26	27								
DRB3*02:19								8													25											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32