

*03:68N	1		4	6		9			14		18	20						30														
*03:69	1		4	6		9					18					25																
*03:71:01-03:71:02	1		4	6		9					18					26	27	30														
*03:72	1		4	6	7	9					18							30														
*03:75	1					9	10					20						29	30													
*03:76, 03:125	1		4	6		9		12										30														
*03:77, 03:84, 03:89, 03:95, 13:144 ⁴	1		4	6		9												30														
*03:79	1		4	6			11				18							30														
*03:80	1		4			9	10			15								30														
*03:81			3		6	9			13								27	30														
*03:82	1		4			9					18							30														
*03:85	1		4				10											30														
*03:88		2	3			8	10	11								24	25	30														
*03:90		2	3				10			16						24		30														
*03:92	1		4	6		9					18							29	31													
*03:96	1		4	6							18							30														
*03:97, 14:33 ⁴			3	4														30														
*03:101			4			8					18					24		30														
*03:110	1		4	6				12	14		18							30														
*03:111, 03:135	1		4	6		9					18																					
*03:112	1		4	6		9		11		15	18							30														
*03:115		2	3			8	10	11	13							24		30														
*03:117	1		4	6		w					18							30														
*03:118	1			6		9					18	20						29	30													
*03:119		2	4			8					18					24		30														
*03:120			4			8					18					w		30														
*03:122			3				9	10										30														
*03:124	1		4	6	7	9					18	19						30														
*03:126, 14:95 ⁴	1		3	4	6	9												30														
*03:130			3	6							18						27															
*03:132	1		4	6	7	9				16	18							30														
*01:16, 01:55, 04:22, DRB3*01:09, DRB3*01:36, DRB3*02:04, DRB3*02:22:01- 02:22:02, DRB3*02:56, DRB3*03:03-03:04						8																										
*04:98:01-04:98:02, 14:107										16																						
*04:211, 11:16, 11:40, 11:58:01, 11:113, 13:43, 13:160, 13:183, 13:214, 13:217, 15:32			4																													
*08:20, 13:03:01-13:03:06, 13:07:01-13:08, 13:12:01-13:13, 13:33:01-13:33:03, 13:37-13:38, 13:47-13:49, 13:55, 13:60, 13:70, 13:72, 13:76, 13:84, 13:88-13:90, 13:94:01-13:95, 13:101, 13:108, 13:115, 13:133-13:134, 13:149, 13:152, 13:164, 13:167, 13:180, 13:182, 13:188, 13:194, 13:219, 14:05:01-14:05:04, 14:14, 14:23:01-14:23:04, 14:36-14:37, 14:43-14:45, 14:56, 14:84, 14:91, 14:100-14:101, 14:103, 14:116, 14:123, 14:136, 14:155, 14:161, 14:167, 14:169, 14:178								10										30														
*09:01:08, 09:02:01-09:02:02, DRB3*01:07, DRB3*01:15, DRB3*01:41							9																									
*09:06, DRB3*01:34	1																															
*09:08										14																						
*11:06:01-11:06:03, 11:21, 11:47, 11:57, 11:129											17																					
*11:07:01-11:07:02, 11:103:01- 11:103:02, 11:125, 11:173						8					18							29														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*11:09, 11:20, 11:28:02, 11:122, 13:103, 13:165, 13:179, 16:08, DRB3*01:08, DRB3*02:06			3																													

Negative Control

[illegible]

[illegible]

*14:48, 14:109, 14:115		2	3							10	11									24							30					
*14:49		2								10																						
*14:52, DRB3*03:10				4																24												
*14:59			3	4						10				15													30					
*14:64			3	4							11																30					
*14:76, 14:79								8																		29		31				
*14:82	1									10		12																				
*14:83		2		4						10																	30					
*14:97										10													26									
*14:99										10						17																
*14:111										10					16																	
*14:121				4						10										24							30					
*14:127:01-14:127:02										10			13														30					
*14:132	1		3	4		6						12																				
*14:133			3	4						10																						
*14:151										10	11		13																			
*14:154		2	3	4						10				15						24							30					
*14:171								8		10																						
*14:177		2		4						10	11									24							30					
*15:25								8								18																
DRB3*01:01:03																					25											
DRB3*01:14										10	11											26	27				30					
DRB3*01:23					6											18			21													
DRB3*01:25											?											?	?									
DRB3*01:31																				23												
DRB3*02:19								8														25										
DRB3*02:54													?	?									?	?								
DRB5*01:04							?																									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control