

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
<b>DRB1 allele<sup>1,2,3</sup></b>																																		
*03:01:01-03:01:26, 03:56, 03:59, 03:61, 03:66, 03:70, 03:73, 03:78, 03:83, 03:86, 03:91, 03:94, 03:98-03:100:02, 03:104, 03:106-03:109, 03:114, 03:116, 03:121, 03:127-03:129, 03:133-03:137, 03:139, 03:141-03:142	1			4		6			9									18												30				
*03:02:01-03:02:03, 03:74, 03:102-03:103, 03:131		2	3					8		10														24							30			
*03:03		2		4				8		10														24							30			
*03:04:01-03:04:02	1				5	6			9									18													30			
*03:05:01-03:05:03			3			6			9									18													30			
*03:06, 03:60	1			4					9	10																					30			
*03:07				4				8										18													30			
*03:08	1			4		6	7											18													30			
*03:09			3			6			9		11							18					23								30			
*03:10	1			4		6						12						18													30			
*03:11:01	1			4		6			9				13																		30			
*03:12	1			4		6								14				18													30			
*03:13:01-03:13:02	1			4		6			w		11							18							25						30			
*03:14			3			6			9							16		18														30		
*03:15:01	1			4		6			9							16		18														30		
*03:15:02	1			4		6			9							16																30		
*03:16	1			4		6			9					14				18														30		
*03:17									10				13	15																		30		
*03:18, 03:48 <sup>5</sup>	1			4		6			9									18				22										30		
*03:19	1			4		6			9									18					23									30		
*03:20, 03:93 <sup>5</sup>	1			4		6			9								17	18														30		
*03:21								8										18			21											30		
*03:22	1			4		6			9									18						25				29			30			
*03:23	1			4		6			9		11							18					23									30		
*03:24				4									13																			30		
*03:25	1					6			9									18		20												30		
*03:26	1			4					9	10															26							30		
*03:27		2	3							10			13											24								30		
*03:28	1			4		6			9									18		20												30		
*03:29		2	3					8										18					24									30		
*03:30	1					6			9									18			21											30		
*03:31	1								9	10										20													30	
*03:32	1			4		6			9									18							25							30		
*03:33	1			4		6			9									18								26						30		
*03:34	1			4		6			9								17	18									27					30		
*03:35, 14:127:01-14:127:02 <sup>4</sup>										10			13																			30		
*03:36	1			4		6			9									18											28			30		
*03:37	1			4		6			9					15				18														30		
*03:38		2	3					8		10	11												24										30	
*03:39	1			4		6			9									18														29	30	
*03:40, 03:105								8										18														29	30	
*03:41:01-03:41:02								8		10																						29	30	
*03:42, 03:87	1			4		6			9									18			21											30		
*03:43	1					6			9									18			21					26	27					30		
*03:44	1					6			9									18							26							30		
*03:45	1			4		6			9									18										28				30	31	
*03:46	1			4		6												18									27					30		
*03:47	1			4		6			9				13					18														30		
*03:49								8										18														30		
*03:50	1			4		6			9									18														30	31	
*03:51	1			4		6			9			12						18														30		
*03:52	1					6			9									18											29			30		
*03:53		2	3					8																24			27					30		
*03:54, 03:113				4														18														30		
*03:55	1			4		6			9									18									27					30		
*03:57, 03:63, 03:123						6			9									18														30		
*03:58	1			4		6			9									18	19													30		
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<																	







