

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele ^{1,2,3}																																	
*03:01:01-03:01:28, 03:56, 03:59, 03:61, 03:66, 03:70, 03:73, 03:78, 03:83, 03:86, 03:91, 03:94, 03:98-03:100:02, 03:104, 03:106-03:109, 03:114, 03:116, 03:121, 03:127-03:129, 03:133-03:137, 03:139, 03:141-03:144, 03:146-03:153, 03:159-03:161, 03:163	1			4		6			9									18												30			
*03:02:01-03:02:04, 03:74, 03:102-03:103, 03:131, 03:154		2	3					8		10														24						30			
*03:03		2		4				8		10														24						30			
*03:04:01-03:04:02	1				5	6			9									18												30			
*03:05:01-03:05:03			3			6			9									18												30			
*03:06, 03:60	1			4					9	10																				30			
*03:07:01-03:07:02				4				8										18												30			
*03:08	1			4		6	7											18												30			
*03:09			3			6			9		11							18					23							30			
*03:10	1			4		6						12						18												30			
*03:11:01	1			4		6			9				13																	30			
*03:12	1			4		6								14				18												30			
*03:13:01-03:13:02	1			4		6			w		11							18							25					30			
*03:14			3			6			9							16		18												30			
*03:15:01	1			4		6			9							16		18												30			
*03:15:02	1			4		6			9							16														30			
*03:16	1			4		6			9					14				18												30			
*03:17										10			13		15															30			
*03:18, 03:48 ⁵	1			4		6			9									18				22								30			
*03:19	1			4		6			9									18					23							30			
*03:20, 03:93 ⁵	1			4		6			9								17	18				21								30			
*03:21								8										18												30			
*03:22	1			4		6			9									18							25				29	30			
*03:23	1			4		6			9		11							18					23							30			
*03:24				4									13																	30			
*03:25:01	1					6			9									18		20										30			
*03:25:02, 03:118	1					6			9									18		20									29	30			
*03:26	1			4					9	10																26				30			
*03:27		2	3							10			13											24						30			
*03:28	1			4		6			9									18		20										30			
*03:29		2	3					8										18						24						30			
*03:30	1					6			9									18				21								30			
*03:31	1								9	10										20										30			
*03:32	1			4		6			9									18							25					30			
*03:33	1			4		6			9									18								26				30			
*03:34	1			4		6			9									17	18								27			30			
*03:35, 14:127:01-14:127:02 ⁴										10			13																	30			
*03:36	1			4		6			9									18										28		30			
*03:37	1			4		6			9						15			18												30			
*03:38		2	3					8		10	11													24						30			
*03:39	1			4		6			9									18												29	30		
*03:40, 03:105								8										18												29	30		
*03:41:01-03:41:02, 03:155								8		10																				29	30		
*03:42, 03:87	1			4		6			9									18				21								30			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*03:43	1					6			9									18			21					26	27			30			
*03:44	1					6			9									18								26				30			
*03:45	1			4		6			9									18										28		30	31		
*03:46	1			4		6												18									27						
*03:47	1			4		6			9				13					18												30			
*03:49								8										18												30			
*03:50	1			4		6			9									18												30	31		
*03:51	1			4		6			9			12						18												30			
*03:52	1					6			9									18												29	30		
*03:53		2	3					8																24			27			30			
*03:54, 03:113				4														18												30			
*03:55	1			4		6			9									18										27		30			
*03:57, 03:63, 03:123						6			9									18												30			

Negative Control

*03:58	1		4		6			9							18	19												30	Negative Control			
*03:62	1			4	6			9				13		15		18												30				
*03:64, 03:156N	1			4	6											18																
*03:65	1			4	6	7										18	19															
*03:67N	1			4	6			w								18						25				28		30				
*03:68N	1			4	6			9					14			18		20										30				
*03:69	1			4	6			9								18						25										
*03:71:01-03:71:02	1			4	6			9															26	27				30				
*03:72	1			4	6	7		9								18												30				
*03:75	1							9	10									20									29	30				
*03:76, 03:125	1			4	6			9			12																	30				
*03:77, 03:84, 03:89, 03:95, 03:157	1			4	6			9																				30				
*03:79	1			4	6					11						18												30				
*03:80	1			4				9	10					15														30				
*03:81			3		6			9				13																30				
*03:82, 03:138	1			4				9								18												30				
*03:85	1			4						10																		30				
*03:88		2	3				8		10	11											24	25						30				
*03:90		2	3						10					16							24							30				
*03:92	1			4	6			9								18										29	31					
*03:96	1			4	6											18												30				
*03:97, 14:33 ⁴			3	4																								30				
*03:101				4			8									18					24							30				
*03:110	1			4	6						12		14			18												30				
*03:111, 03:162	1			4	6			9								18																
*03:112	1			4	6			9		11			15			18												30				
*03:115		2	3				8		10	11		13									24							30				
*03:117	1			4	6			w								18												30				
*03:119		2		4			8									18					24							30				
*03:120				4			8									18					w							30				
*03:122			3					9	10																			30				
*03:124	1			4	6	7		9								18										28		30				
*03:126, 14:95 ⁴	1		3	4	6			9																				30				
*03:130			3		6											18									27							
*03:132	1			4	6	7		9						16		18												30				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*03:140	1			4	6	7								15			18															
*03:145	1			4	6			9								16	17												30			
*03:158	1			4					10				14																30			
*01:16, 01:55, 01:101, 04:22, 04:283, DRB3*01:09, DRB3*01:36, DRB3*02:04, DRB3*02:22:01-02:22:02, DRB3*02:56, DRB3*03:03-03:04:02							8																									
*01:90, 04:98:01-04:98:02, 14:107, DRB3*01:82															16																	

Negative Control

[illegible]

[illegible]

[illegible]

*13:19, 14:20, 14:40, 14:94, 14:102, 14:156, 14:159-14:160, 14:181	2								10																					30			
*13:26:01-13:26:02, 13:85, 13:198, 13:235, 14:02:01:01-14:03:01, 14:13, 14:19, 14:24, 14:27:01- 14:27:02, 14:47, 14:51, 14:63, 14:67, 14:85, 14:89, 14:98, 14:135, 14:174, 14:194, 14:198, 14:209	2		3						10									24													30		
*13:27:01-13:27:02, 13:71, 13:129, 13:176	1			4		6				9		w																	30				
*13:32, 13:40, 14:80				4					10																					30			
*13:34, 13:136			3																			26				27				30			
*13:36, 13:65, 13:120, 13:189, 14:09, 14:144				3					10																					30			
*13:39			3								11																		30				
*13:41, 13:277, 14:179			3			6				9		w																	30				
*13:53, 14:06:01-14:06:04, 14:12:01-14:12:02, 14:78, 14:81, 14:165, 14:170, 14:176, 14:195N, 14:212	2		4						10										24											30			
*13:54									11								19													30			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*13:57		2																			21										30		
*13:58, 13:81									10								17														30		
*13:62																									26				27		30		
*13:64, 13:204, 13:250																					21						26				27		30
*13:67			3																		21										30		
*13:77-13:79, 13:181				4						11																				30			
*13:83, 13:107, 14:65, 14:187																					21										30		
*13:86		2																												30			
*13:100, 13:162										11																				30			
*13:104, 14:03:02		2		3																		24									30		
*13:106																									26					30			
*13:118, 13:135, 14:96									10						15											26					30		
*13:125, 13:242:01-13:242:02				4													17													30			
*13:131				4											15															30			
*13:144	1			4		6			9									19												30			
*13:150														15				19												30			
*13:151, 13:174									10																26				27		30		
*13:159			3						10																								
*13:163				4						11						17														30			
*13:191				4					10																								
*13:216									10																26					30			
*13:221				4																					26				27		30		
*13:262				4					10					15																30			
*13:266				4																					26					30			
*13:284																	17		19											30			
*14:18, 14:106		2	3	4					10														24							30			
*14:29		2		4					10								17						24							30			
*14:38:01-14:38:02									10				13																				
*14:41, 14:77		2							10																26				27		30		
*14:46, DRB3*02:20			3																				24										
*14:48, 14:109, 14:115		2	3						10			11										24								30			
*14:49		2							10																								
*14:52, DRB3*03:10				4																		24											
*14:59			3	4					10					15																30			
*14:64			3	4						11																				30			
*14:76, 14:79							8																					29		31			
*14:82	1								10			12																					
*14:83		2		4					10																					30			
*14:97									10																26								
*14:99									10						17																		
*14:108		2		4					10						19						24									30			
*14:111									10					16																			
*14:121				4					10													24								30			
*14:132	1		3	4		6					12											24											
*14:133			3	4					10																								

Negative Control

Negative Control

[illegible]