

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele^{1,2,3}																																	
*03:01:01:01-03:01:34, 03:52:02, 03:56, 03:59, 03:61, 03:66, 03:70, 03:78, 03:83, 03:86, 03:91, 03:94, 03:98-03:99, 03:104, 03:106-03:109, 03:114, 03:116, 03:121, 03:127-03:129, 03:133-03:137, 03:139, 03:141-03:144, 03:146-03:153, 03:159-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180, 03:182-03:184, 03:186	1			4		6			9									18												30			
*03:02:01:01-03:02:05, 03:74, 03:102-03:103, 03:131, 03:154, 03:176, 03:179		2	3					8		10														24							30		
*03:03		2		4				8		10														24							30		
*03:04:01-03:04:02	1				5	6			9									18												30			
*03:05:01-03:05:03			3			6			9									18													30		
*03:06, 03:60	1			4					9	10																					30		
*03:07:01:01-03:07:02, 03:171				4				8										18													30		
*03:08	1			4		6	7											18													30		
*03:09			3			6			9		11							18						23							30		
*03:10	1			4		6						12						18													30		
*03:11:01, 03:11:03, 03:177	1			4		6			9				13																		30		
*03:12	1			4		6								14				18													30		
*03:13:01-03:13:02	1			4		6			w		11							18							25						30		
*03:14			3			6			9							16		18													30		
*03:15:01:01-03:15:01:02	1			4		6			9							16		18													30		
*03:15:02	1			4		6			9							16															30		
*03:16	1			4		6			9					14				18													30		
*03:17									10				13		15																30		
*03:18, 03:48 ⁵	1			4		6			9									18				22									30		
*03:19	1			4		6			9									18					23								30		
*03:20, 03:93 ⁵	1			4		6			9								17	18													30		
*03:21								8										18				21									30		
*03:22	1			4		6			9									18							25				29	30			
*03:23	1			4		6			9		11							18					23								30		
*03:24				4									13																		30		
*03:25:01	1					6			9									18		20											30		
*03:25:02, 03:118	1					6			9									18		20									29	30			
*03:26	1			4					9	10																26					30		
*03:27		2	3										13											24							30		
*03:28	1			4		6			9									18		20											30		
*03:29		2	3					8										18						24							30		
*03:30	1					6			9									18				21									30		
*03:31	1								9	10										20											30		
*03:32	1			4		6			9									18							25						30		
*03:33	1			4		6			9									18								26					30		
*03:34	1			4		6			9								17	18									27				30		
*03:35, 14:127:01-14:127:02 ⁴									10				13					18											28		30		
*03:36	1			4		6			9									18													30		
*03:37	1			4		6			9						15			18													30		
*03:38		2	3					8		10	11													24							30		
*03:39	1			4		6			9									18												29	30		
*03:40, 03:105								8										18												29	30		
*03:41:01-03:41:02, 03:155								8		10																				29	30		
*03:42, 03:87	1			4		6			9									18				21									30		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
*03:43	1					6			9									18				21				26	27			30			
*03:44	1					6			9									18								26				30			
*03:45	1			4		6			9									18										28		30	31		
*03:46	1			4		6												18										27			30		
*03:47	1			4		6			9				13					18													30		
*03:49								8										18													30		
*03:50	1			4		6			9									18													30	31	
*03:51	1			4		6			9			12						18													30		
*03:52:01	1					6			9									18												29	30		
*03:53		2	3					8																24			27				30		
*03:54, 03:113				4														18													30		
*03:55	1			4		6			9									18									27				30		
*03:57, 03:63, 03:123						6			9									18													30		
*03:58	1			4		6			9									18	19												30		
*03:62	1			4		6			9				13		15			18													30		
*03:64, 03:156N	1			4		6												18															
*03:65	1			4		6	7											18	19														
*03:67N	1			4		6			w									18								25		28		30			

Negative Control

*03:68N	1		4	6				9					14			18	20											30	Negative Control				
*03:69	1		4	6				9								18							25					30					
*03:71:01-03:71:02	1		4	6				9								18							26	27				30					
*03:72	1		4	6	7			9								18												30					
*03:73	1		4	6				9								18	20		22									30					
*03:75	1							9	10								20										29	30					
*03:76, 03:125, 03:174N	1		4	6				9			12																	30					
*03:77, 03:84, 03:89, 03:95, 03:157	1		4	6				9																				30					
*03:79	1		4	6						11						18												30					
*03:80	1		4					9	10				15															30					
*03:81			3		6								13															30					
*03:82, 03:138, 03:181	1		4					9								18												30					
*03:85	1		4						10																			30					
*03:88		2	3				8		10	11										24	25							30					
*03:90		2	3						10						16					24								30					
*03:92	1		4	6				9								18											29	31					
*03:96	1		4	6												18												30					
*03:97, 14:33 ⁴			3	4																								30					
*03:100:01-03:100:02	1		4	6				9					15		17	18												30					
*03:101			4				8									18				24								30					
*03:110	1		4	6							12		14			18												30					
*03:111, 03:162, 03:173	1		4	6				9								18												30					
*03:112	1		4	6				9		11			15			18												30					
*03:115		2	3				8		10	11		13								24								30					
*03:117	1		4	6				w								18												30					
*03:119		2	4				8									18						24						30					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28		29	30	31	32
*03:120				4				8									18						w							30			
*03:122			3						9	10																				30			
*03:124	1		4	6	7			9								18											28			30			
*03:126, 14:95 ⁴	1		3	4	6			9																						30			
*03:130			3		6												18									27							
*03:132	1		4	6	7			9							16		18												30				
*03:140	1		4	6	7									15			18												30				
*03:145	1		4	6				9							16	17													30				
*03:158	1		4						10				14																30				
*03:167		2	3						10	11		13								24									30				
*03:178			3				8										18												30				
*03:185				4											16		18												30				
*01:16, 01:55, 01:101, 04:22, 04:283, DRB3*01:09, DRB3*01:36, DRB3*02:04, DRB3*02:22:01-02:22:02, DRB3*02:56, DRB3*03:03-03:04:02								8																									
*01:90, 04:98:01-04:98:02, 14:107, DRB3*01:82, DRB3*02:137N															16																		

[illegible]

[illegible]

[illegible]

*14:18, 14:106		2	3	4						10										24								30				
*14:29		2		4						10					17					24								30				
*14:38:01-14:38:02										10			13																			
*14:41, 14:77		2								10												26	27					30				
*14:46, DRB3*02:20			3																	24												
*14:48, 14:109, 14:115		2	3							10	11									24								30				
*14:49		2								10																						
*14:52, DRB3*03:10				4																24												
*14:59			3	4						10				15														30				
*14:64			3	4						11																		30				
*14:76, 14:79							8																				29	31				
*14:82	1									10		12																				
*14:83		2		4						10																		30				
*14:99										10						17																
*14:108		2		4						10							19			24								30				
*14:111										10					16																	
*14:121				4						10										24								30				
*14:132	1		3	4		6						12																				
*14:133			3	4						10																						
*14:150																									27							
*14:151										10	11		13																			
*14:154		2	3	4						10				15						24								30				
*14:171							8			10																						
*14:177		2		4						10	11									24								30				
*14:200, 14:222N		2	3							10										24												
*14:218		2	3							10												26	27					30				
*15:25							8									18																
DRB3*01:01:03																					25											
DRB3*01:14										10	11											26	27					30				
DRB3*01:23						6										18			21													
DRB3*01:31																				23												
DRB3*01:34, DRB3*03:33	1																															
DRB3*01:42							7																									
DRB3*01:46			3			6										18			21													
DRB3*01:71			3				8													24												
DRB3*01:75								w													25											
DRB3*01:77N													14					20														
DRB3*02:19							8														25											
DRB3*02:146							8									18			21													
DRB5*01:04							?																			?						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Negative Control																																

Negative Control