

| Well No.                                                                                                                                                                                                                                                                                                               | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9  | 10 | 11 | 12 | 13 | 14 | 15 | 16 | 17 | 18 | 19 | 20 | 21 | 22 | 23 | 24 | 25 | 26 | 27 | 28 | 29 | 30 | 31 | 32 | 33 | 34 | 35 | 36 | 37 | 38 | 39 | 40 | 41 | 42 | 43 | 44 | 45 | 46 | 47 | 48 |  |  |    |
|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---|---|---|---|---|---|---|---|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|--|--|----|
| DRB1 allele <sup>1,2,3</sup>                                                                                                                                                                                                                                                                                           |   |   |   |   |   |   |   |   |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:01:01-04:01:22, 04:111-04:112, 04:115, 04:130:01-04:130:02, 04:135, 04:153, 04:159, 04:171, 04:174, 04:187, 04:190, 04:194, 04:196, 04:200, 04:208, 04:216, 04:233, 04:236, 04:239, 04:241-04:243, 04:248, 04:254, 04:257, 04:261, 04:266N, 04:268, 04:270-04:271, 04:273, 04:280N, 04:281, 04:285, 04:296, 04:304 | 1 |   |   | 4 |   |   | 7 |   | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 24 |    | 26 |    |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:02:01-04:02:06, 04:124, 04:134, 04:204, 04:237, 04:259, 04:287                                                                                                                                                                                                                                                     |   | 2 |   | 4 |   |   | 8 |   | 10 |    |    |    | 14 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:03:01:01-04:03:07, 04:03:09-04:03:16, 04:109, 04:163, 04:181, 04:183, 04:198, 04:226:01-04:226:02, 04:234, 04:279, 04:294-04:295, 04:300N                                                                                                                                                                          |   |   | 3 |   |   |   | 8 | 9 | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:03:08                                                                                                                                                                                                                                                                                                              |   |   | 3 |   |   |   | 8 | 9 | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  | 45 |
| *04:04:01-04:04:03, 04:04:06-04:04:16, 04:118, 04:121, 04:149, 04:177, 04:193, 04:220-04:221, 04:223, 04:240, 04:247N, 04:258, 04:272, 04:275-04:276, 04:282, 04:288, 04:290, 04:302                                                                                                                                   |   |   |   | 4 |   |   | 8 | 9 | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:04:04                                                                                                                                                                                                                                                                                                              |   |   |   | 4 |   |   | 8 | 9 | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  | 45 |
| *04:04:05                                                                                                                                                                                                                                                                                                              |   |   |   | 4 |   |   | 8 | w | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:04:17, 04:167                                                                                                                                                                                                                                                                                                      |   |   |   |   |   |   | 8 | 9 | 10 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:05:01:01-04:05:01:04, 04:05:03-04:05:07, 04:05:09-04:05:11, 04:05:13-04:05:14, 04:05:16-04:05:17, 04:05:19-04:05:22, 04:104, 04:162, 04:202, 04:227, 04:232, 04:245, 04:255, 04:286N, 04:289, 04:292                                                                                                               |   |   |   | 4 | 5 |   | 7 |   | 9  |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:05:02, 04:05:15                                                                                                                                                                                                                                                                                                    |   |   |   | 4 | 5 |   | 7 |   | 9  |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:05:08                                                                                                                                                                                                                                                                                                              |   |   |   | 4 | 5 |   | 7 |   | w  |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    | 26 |    |    |    |    | 32 |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |
| *04:05:18                                                                                                                                                                                                                                                                                                              |   |   |   | 4 | 5 |   | 7 |   | 9  | 10 |    | </ |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |    |  |  |    |

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]