

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48					
DRB1 allele ^{1,2,3}																																																					
*04:01:01:01-04:01:23, 04:111-04:112, 04:115, 04:130:01-04:130:02, 04:135, 04:153, 04:159, 04:171, 04:174, 04:187, 04:190, 04:194, 04:196, 04:200, 04:208, 04:216, 04:233, 04:236, 04:239, 04:241-04:243, 04:248, 04:254, 04:257, 04:261, 04:266N, 04:268, 04:270-04:271, 04:273, 04:280N-04:281, 04:285, 04:296, 04:304, 04:317, 04:321, 04:326, 04:331, 04:334-04:335, 04:338, 04:341, 04:343, 04:346	1			4			7			10														24		26						32																					
*04:02:01-04:02:07, 04:124, 04:134, 04:204, 04:237, 04:259, 04:287, 04:336, 04:348		2		4				8		10				14																			32																				
*04:03:01:01-04:03:07, 04:03:09-04:03:21, 04:109, 04:163, 04:181, 04:183, 04:198, 04:226:01-04:226:02, 04:234, 04:279, 04:294-04:295, 04:300N, 04:307, 04:311, 04:316, 04:329N-04:330, 04:333, 04:340, 04:349-04:350, 04:352			3					8	9	10																26							32																				
*04:03:08			3					8	9	10																26							32													45							
*04:04:01:01-04:04:03, 04:04:06-04:04:16, 04:04:18-04:04:19, 04:118, 04:121, 04:149, 04:177, 04:193, 04:220-04:221, 04:223, 04:240, 04:247N, 04:258, 04:272, 04:275-04:276, 04:282, 04:288, 04:290, 04:302, 04:306, 04:308-04:309, 04:312N-04:313, 04:318, 04:322, 04:327, 04:332, 04:345				4				8	9	10																26							32																				
*04:04:04				4				8	9	10																26							32													45							
*04:04:05				4				8	w	10																26							32																				
*04:04:17, 04:167								8	9	10																26							32																				
*04:05:01:01-04:05:01:10, 04:05:03-04:05:07, 04:05:09-04:05:11, 04:05:13-04:05:14, 04:05:16-04:05:17, 04:05:19-04:05:23, 04:104, 04:162, 04:202, 04:227, 04:232, 04:245, 04:255, 04:286N, 04:289, 04:292, 04:305, 04:319-04:320, 04:328				4	5		7			9																26								32																			
*04:05:02, 04:05:15				4	5		7			9																26																											
*04:05:08				4	5		7		w																	26							32																				
*04:05:18				4	5		7		9	10																26							32																				
*04:06:01-04:06:07, 04:144, 04:176, 04:277			3			6		8	9	10																26																											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48					
*04:07:01:01-04:07:06, 04:172, 04:197, 04:222, 04:246, 04:249, 04:256, 04:262-04:263, 04:269, 04:278, 04:337, 04:351			3				7		9	10																26				29			32																				

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]