

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DRB1 allele^{1,2,3}																								
*08:01:01-08:01:06, 08:50	1		3		5		7																	
*08:02:01-08:02:04, 08:42		2	3		5		7																	
*08:03:02-08:03:04, 08:46, 08:51, 08:53	1			4	5		7									16								
*08:04:01, 08:04:04-08:04:07		2	3		5			8									17						23	
*08:04:02-08:04:03		2	3		5			8									w							
*08:05	1		3			6	7																	
*08:06, 08:54	1		3		5			8									17			20			23	
*08:07			3		5		7											19						
*08:08			3		5		7			11														
*08:09		2	3		5		7											18						
*08:10	1			4	5			8								16	17			20			23	
*08:11			3		5		7						13											
*08:12	1			4	5			8		10						16	17			20				
*08:13		2			5		7					12												
*08:14									9							16	17							
*08:15				4	5		7				11													
*08:16	1		3		5		7											19						
*08:17	1		3		5		7							14							21			
*08:18, 08:47	1			4		6	7									16								
*08:19				4	5		7								15	16						22		
*08:20, 14:12:01-14:12:02, 14:84 ⁴					5												17	18						
*08:21		2	3				7											18						
*08:22	1		3		5			8		10							17			20				
*08:23	1			4	5		7									16						22		
*08:24		2	3			6	7																	
*08:25				4		6	7									15								
*08:26	1		3		5		7		9															
*08:27	1			4	5		7					12				16								
*08:28		2	3		5			8						14			17				21		23	
*08:29	1			4			7									16						22		
*08:30:01-08:30:03		2		4	5		7																	
*08:31			3			6		8			11												23	
*08:32	1			4	5		7									16		18						
*08:33	1			4	5		7						13			16								
*08:34				4	5		7								15									
*08:35	1			4	5		7		9							16		18						
*08:36:01-08:36:02	1			4	5		7		9							16								
*08:37	1			4	5		7							14		16					21			
*08:38	1			4	5		7									16			19					
*08:39	1		3		5		7						13											
*08:40	1			4		6	7			10						16								
*08:41			3			6	7				11													
*08:43	1		3				7					12											23	
*08:44		2	3		5		7					12												
*08:45		2		4	5		7							14							21			
*08:48	1				5		7					12												
*08:49	1			4	5		7								15	16			19					
*08:52			3		5		7								15									
*08:55	1		3		5																			
*08:56	1			4			7									16								
*03:12, 14:65									9											20				
*03:92		2						8						14									23	
*04:10:01-04:11:03, 04:67, 04:91, 04:126, 04:147, 13:75, 13:108, 13:169, 15:12																				20				
*04:12																16	17			20				
*04:18, 04:25, 04:58																	17							
*04:86, 04:106, 07:12, 13:03:01-13:03:06, 13:12:01-13:12:03, 13:30, 13:33:01-13:33:03, 13:38, 13:66:01-13:66:02, 13:95, 13:101, 13:115, 13:118, 13:122, 13:151-13:152, 13:161, 13:167, 13:174																16								
*04:136, 13:04, 13:48, 13:58, 13:81, 13:89, 13:94:01-13:94:02																16				20				
*11:23:01-11:23:02, 11:45, 11:55, 11:64, 13:119, 13:146, 13:158					5													18			21			
*11:25, 11:119, 13:18, 13:144, 13:156					5												17	18			21			
*11:67			3		5			8			11			14			17				21		23	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

[illegible]