

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DRB1 allele^{1,2,3}																								
*08:01:01-08:01:06, 08:50	1		3		5		7																	
*08:02:01-08:02:04, 08:42, 08:72-08:73		2	3		5		7																	
*08:03:02-08:03:05, 08:46, 08:51, 08:53, 08:58, 08:61-08:62, 08:66, 08:68-08:69, 08:71, 08:74, 08:76	1			4	5		7									16								
*08:04:01, 08:04:04-08:04:07, 08:59		2	3		5			8									17						23	
*08:04:02-08:04:03		2	3		5			8									w							
*08:05	1		3			6	7																	
*08:06, 08:54, 08:70	1		3		5			8									17			20			23	
*08:07			3		5		7											19						
*08:08			3		5		7			11														
*08:09		2	3		5		7											18						
*08:10	1			4	5			8								16	17			20			23	
*08:11			3		5		7						13											
*08:12	1			4	5			8		10						16	17			20				
*08:13		2			5		7					12												
*08:14	1			4			7		9							16	17							
*08:15				4	5		7				11													
*08:16	1		3		5		7												19					
*08:17	1		3		5		7							14							21			
*08:18, 08:47	1			4		6	7									16								
*08:19				4	5		7								15	16						22		
*08:20, 14:12:01-14:12:02, 14:84, 14:156 ⁴					5												17	18						
*08:21		2	3				7											18						
*08:22	1		3		5			8		10							17			20				
*08:23	1			4	5		7									16						22		
*08:24		2	3			6	7																	
*08:25				4		6	7								15									
*08:26	1		3		5		7		9															
*08:27	1			4	5		7					12				16								
*08:28		2	3		5			8						14			17				21		23	
*08:29	1			4			7									16					22			
*08:30:01-08:30:03		2		4	5		7																	
*08:31			3			6		8			11												23	
*08:32	1			4	5		7									16		18						
*08:33	1			4	5		7						13			16								
*08:34				4	5		7								15									
*08:35	1			4	5		7		9							16		18						
*08:36:01-08:36:02	1			4	5		7		9							16								
*08:37	1			4	5		7							14		16					21			
*08:38	1			4	5		7									16			19					
*08:39	1		3		5		7						13											
*08:40	1			4		6	7			10						16								
*08:41			3			6	7				11													
*08:43	1		3				7					12											23	
*08:44		2	3		5		7					12												
*08:45		2		4	5		7							14							21			
*08:48	1				5		7					12												
*08:49	1			4	5		7								15	16			19					
*08:52			3		5		7								15									
*08:55, 08:64	1		3		5																			
*08:56	1			4			7									16								
*08:57					5																			
*08:60N	1			4	5		7							14		16								
*08:63				4	5		7			11				14							21			
*08:65	1			4	5											16								
*08:67		2	3					8									17						23	
*08:75			3		5			8			11						17						23	
*03:12, 14:65									9											20				
*03:92		2						8						14									23	
*04:10:01-04:11:04, 04:67, 04:91, 04:126, 04:147, 13:75, 13:108, 13:169, 15:12																				20				
*04:12																16	17			20				
*04:18, 04:25, 04:58																	17							
*04:86, 04:106, 07:12, 13:03:01-13:03:06, 13:12:01-13:12:04, 13:30, 13:33:01-13:33:03, 13:38, 13:66:01-13:66:02, 13:95, 13:101, 13:115, 13:118, 13:122, 13:151-13:152, 13:161, 13:167, 13:174, 13:188, 13:194																16								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*04:136, 13:04, 13:48, 13:58, 13:81, 13:89, 13:94:01-13:94:02																16			20					
*11:23:01-11:23:02, 11:45, 11:55, 11:64, 13:119, 13:146, 13:158					5												18			21				
*11:25, 11:119, 13:18, 13:144, 13:156					5												17	18			21			
*11:67			3		5			8			11				14					21		23		
*12:01:01-12:01:07, 12:01:09, 12:05-12:06, 12:08, 12:10-12:11, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36, 12:40-12:41, 12:54, 12:58				4		6		8			10				14	15								
*12:01:08, 12:47				4		6		8			10				14									
*12:02:01-12:02:05, 12:13, 12:15, 12:20, 12:23, 12:26-12:27, 12:31N-12:32, 12:37, 12:44-12:45, 12:50-12:52, 12:55-12:56				3		6		8			10				14	15								
*12:02:06, 12:18, 12:33				3		6		8			10				14									
*12:03:02-12:03:03				4		6		8							14	15							23	
*12:04				4		6		8			10	11			14									
*12:07						6		8			10				14	15								
*12:09			2	4		6		8			10				14									
*12:12						6		8			10		12		14	15								
*12:16:01-12:16:03				3		6	7								14	15								
*12:19				3		6		8							14	15							23	
*12:21				3		6		8			10				15									
*12:22				4			7								14	15								
*12:38, 12:53				4		6		8			10				15									
*12:39				4		6	7								14	15								
*12:43				3				8			10				14	15								
*12:46	1			4		6		8			10				14	16			20					
*12:48		w		4		6		8			10				14									
*12:49			3			6		8			10	11			14									
*13:13					5											16		18						
*13:17, 13:175		2		4		6		8			10				14								23	
*13:32, 13:93										9						16			20					
*13:47, 13:55, 14:03:01-14:03:02, 14:27, 14:40, 14:55, 14:67, 14:77, 14:89, 14:102, 14:115-14:116, 14:135-14:136, 14:144					5												18							
*13:65, 13:120, 13:139										9						16								
*13:116		2		4		6		8			10												23	
*13:154					5											16		18			21			
*13:164					5												17	18		20				
*14:04:01-14:04:02, 14:11, 14:120, 14:145, 14:148, 14:152N								8			11											22	23	
*14:13										9														
*14:15		2	3		5			8									17	18					23	
*14:28								8			10	11										22		
*14:31						6		8				11											23	
*14:50, 14:76, 14:79, 14:107								8			11												23	
*14:52		2				6		8															23	
*14:63					5					9						16		18						
*14:68:01-14:68:02, 14:93							7					11										22		
*14:71								8				11										22		
*14:73			3					8				11										22	23	
*14:78					5					9						16	17	18		20				
*14:85					5					9								18						
*14:126:01-14:126:02		2						8							14							22	23	
*14:138								8				11										22	23	
*15:21																	17				21			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control