

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
DRB3 allele ^{1,2,3}																																
*01:01:02:01-01:01:05	1	2	3										13	14						20												
*01:02	1	2	3										13	14						20		22	23									
*01:03	1	2			5								13							20												
*01:04	1	2	3			6							13							20												
*01:05	1	2	3				7						13							20												
*01:06	1	2	3					8					13							20												
*01:07	1		3				7		9				13	14						20												
*01:08	1	2	3					8					13				17			20												
*01:09										10			13	14		16		18		20												
*01:10	1	2	3		5								13							20												
*01:11	1	2	3										13							20		22										
*01:12	1	2	3										13							20					25							
*01:13	1	2	3										13							20	21											
*01:14		2																							26							
*01:15	1		3				7						13							20												
*02:01									9	10	11	12															27					
*02:02:01:01-02:02:05									9	10			13	14																		
*02:03									9	10			13		15																	
*02:04									9	10	11	12				16																
*02:05	1								9				13	14																		
*02:06									9	10			13	14			17															
*02:07									9	10			13	14				18														
*02:08									9	10			13	14					19													
*02:09										10			13	14				18		20				24								
*02:10										10			13	14						20												
*02:11									9	10			13	14							21											
*02:12									9	10			13	14								22										
*02:13									9	10			13	14									23									
*02:14									9	10								19														
*02:15									9	w			13	14																29		
*02:16									9	10			13	14	15		17			20												
*02:17									9	10		12	13	14	15																	
*02:18									9	10			13	14				19					24									
*02:19									9	10			13	14		16							24									31
*02:20									9	10			13				17															
*02:21									9	10			13	14				18					24									31
*02:22									9	10			13	14		16																
*02:23									9	10			13	14			17		19													
*02:24									9	10	11	12																				
*02:25									9	10			13	14									24									31
*02:26									9	10			13	14										25								
*02:27									9	w			13	14														28	29			
*02:28									9	10			13	14																30		
*02:29N									9	10			13	14																	31	
*03:01:01-03:01:03	1	2									11				15					20				24								
*03:02										10	11				15					20				24								
*03:03	1	2											13	15						20				24								
DRB1*03:01:02, DRB1*03:02:02, DRB1*03:05:03, DRB1*03:13:02, DRB1*03:17, DRB1*03:24, DRB1*03:27, DRB1*03:35, DRB1*03:81, DRB1*11:27:01, DRB1*11:84:01, DRB1*11:124, DRB1*11:136, DRB1*11:138, DRB1*13:33:02-13:33:03, DRB1*13:61:01, DRB1*13:94:02, DRB1*13:96:02																									26							
DRB1*03:37, DRB1*03:80, DRB1*11:43, DRB1*11:50, DRB1*11:115, DRB1*11:151, DRB1*13:131, DRB1*13:150, DRB1*15:65												12																				
DRB1*03:38, DRB1*03:79, DRB1*03:85, DRB1*03:96		2																														
DRB1*03:42	1		3				7	8			11						17			20												
DRB1*03:44																16																
DRB1*03:87	1										11						17			20												
DRB1*03:101									10																							
DRB1*04:82, DRB1*04:99, DRB1*08:07, DRB1*08:19, DRB1*08:25, DRB1*08:34, DRB1*08:52, DRB1*12:27, DRB1*13:31, DRB1*13:46, DRB1*13:54, DRB1*13:77, DRB1*13:100, DRB1*13:162, DRB1*13:180		w																														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control

[illegible]