

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		
DRB3 allele^{1,2,3}																																																		
*01:01:02:01 -01:01:07, 01:28, 01:33, 01:36, 01:38, 01:44, 01:47- 01:48	1	2	3										13							20																														
*01:01:08	1		3										13							20																														
*01:02	1	2	3										13	14						20		22	23																											
*01:03	1	2			5								13							20																														
*01:04	1	2	3			6							13							20																														
*01:05	1	2	3				7						13							20																														
*01:06	1	2	3					8					13							20																														
*01:07	1		3				7		9				13	14																																				
*01:08	1	2	3					8					13				17			20																														
*01:09										10			13	14		16		18		20																														
*01:10	1	2	3		5								13							20																														
*01:11	1	2	3										13							20		22																												
*01:12	1	2	3										13							20					25		28																							
*01:13	1	2	3										13							20	21																													
*01:14		2																							26																									
*01:15	1		3				7																																											

[illegible]

[illegible]

*03:08	1	2						11		15		20		24			33			42		
*03:09, 03:18	1							11		15		20		24			33					
*03:10	1	2						11			17	20		24								
*03:12	1	2						11				20		24			33					
*03:13	1	2						11		15		20		24			33		40			
*03:15	1	2						11		15		20	21	24			33		37			
*03:16	1	2						11		15		20					33					
*03:17	1	2					9	11	12	15				24			31	33				
DRB1*01:77																w						
DRB1*03:01:02,																						
DRB1*03:02:02,																						
DRB1*03:05:03,																						
DRB1*03:13:02,																						
DRB1*03:17,																						
DRB1*03:24,																						
DRB1*03:27,																						
DRB1*03:81,																						
DRB1*03:130,																						
DRB1*08:67,																						
DRB1*11:07:02,																						
DRB1*11:27:01,																						
DRB1*11:84:01,																						
DRB1*11:124,																						
DRB1*11:136,																						
DRB1*11:138,																						
DRB1*13:33:02-																						
13:33:03,																						
DRB1*13:61:01,																						
DRB1*13:94:02,																						
DRB1*13:96:02,																						
DRB1*14:47																						
DRB1*03:35														26			33					

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]