

DRB3 2019 6K0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
DRB3 allele ^{1,2,3}																																																			
*01:01:02:01-01:01:07, 01:01:09-01:01:10, 01:28, 01:33, 01:36, 01:38, 01:44, 01:47-01:48, 01:51-01:54, 01:57-01:66, 01:68-01:70, 01:72-01:74, 01:77N-01:83	1	2	3										13							20																															
*01:01:08, 01:55	1		3										13							20																															
*01:02	1	2	3										13	14						20		22	23																												
*01:03	1	2			5								13							20																															
*01:04	1	2	3			6							13							20																															
*01:05	1	2	3				7						13							20																															
*01:06	1	2	3					8					13							20																															
*01:07	1		3				7		9				13	14																																					
*01:08	1	2	3					8					13					17		20																															
*01:09										10			13	14		16		18		20																															
*01:10	1	2	3		5								13							20																															
*01:11	1	2	3										13							20		22																													
*01:12	1	2	3										13							20					25		28																								
*01:13	1	2	3										13							20	21																														
*01:14		2																							26																										
*01:15	1		3				7						13							20																															
*01:16	1	2	3										13							20																															
*01:17	1	2	3										13							20																															
*01:18	1	2	3										13					18		20																															
*01:19	1	2	3										13							20	21				25																										
*01:20, 01:32	1	2	3										13							20																															
*01:21	1	2	3						9				13																																						
*01:22		2											13							20																															
*01:23	1	2	3				7	8					13							20																															
*01:24		2											13							20																															
*01:25	1	2											13							20																															
*01:26N		2											13							20																															
*01:27	1	2	w										13							20																															
*01:29	1	2	3										13																																						
*01:30, 01:49	1	2	3										13							20																															
*01:31	1	2	3										13							20	21																														
*01:34	1	2	3								11									20																															
*01:35	1	2	3										13																																						
*01:37	1		3										13							20																															
*01:39	1	2											13							20																															
*01:40:01N	1	2	3										13							20																															

Negative Control

Negative Control

*01:40:02N	1	2	3										13				20				28										44																	
*01:41, 02:112	1								9				13												33																							
*01:42	1		3										13				19	20											40																			
*01:43	1	2	3										13				20									34																						
*01:45	1	2	3										13				20											37																				
*01:46	1	2	3				7	8					13				17	20																														
*01:50	1		3				7		9				13												33					40																		
*01:56													13				20																															
*01:67	1	2											13				20			24					32																							
*01:71	1	2			5			8					13				17	20																														
*01:75	1		3				w						13					20																														
*01:76	1		3				7		9				13																	40																		
*02:01									9	10	11	12									27																											
*02:02:01:01-02:02:18, 02:02:20-02:02:22, 02:40- 02:41, 02:45-02:46, 02:49, 02:53-02:54, 02:62-02:66, 02:68-02:69, 02:71-02:72, 02:74-02:77, 02:79, 02:82, 02:84-02:88, 02:90-02:92, 02:94-02:96, 02:98-02:99, 02:101-02:102, 02:105, 02:108-02:110, 02:113- 02:114, 02:116-02:118, 02:121N-02:126, 02:128													9	10			13	14																														
*02:02:19, 02:52, 02:59, 02:120									9	10			13																																			
*02:03									9	10			13		15																																	
*02:04									9	10	11	12				16									32																							
*02:05:01	1								9				13	14														?																				
*02:05:02	1								9				13	14																																		
*02:06									9	10			13	14			17																															
*02:07									9	10			13	14			18											?																				
*02:08									9	10			13	14				19										?																				
*02:09										10			13	14			18	20				24																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*02:10, 02:106										10			13	14						20																												
*02:11									9	10			13	14							21														37													
*02:12, 02:67N ⁴									9	10			13	14								22																										
*02:13									9	10			13	14								23																										
*02:14									9	10							19																															
*02:15									9				13	14											29																							
*02:16									9	10			13	14	15		17		20																													

*02:17				9	10	12	13	14	15										
*02:18				9	10		13	14		19		24							
*02:19				9	10		13	14	16			24			31	32			
*02:20				9	10		13			17									
*02:21				9	10		13	14		18		24			31				
*02:22:01-02:22:02				9	10		13	14	16						32				
*02:23				9	10		13	14		17	19								
*02:24, 02:103				9	10	11	12												
*02:25				9	10		13	14				24			31				
*02:26				9	10		13	14				25							
*02:27, 02:111				9			13	14					28	29					
*02:28				9	10		13	14						30					
*02:29N				9	10		13	14						31					
*02:30, 02:97				9	10		13	14								37			
*02:31:01				9	10		13	14				25	28						
*02:31:02				9	10		13	14										45	
*02:32, 02:34 ⁴				9	10		13	14								38			
*02:33, 02:127				9	10		13	14							35				
*02:35				9	10		13	14							36				
*02:36				9	10		13	14				28					41		
*02:37, 02:107				9	10		13							33					
*02:38	1						13	15		20					33				
*02:39				9	10		13	14								39			
*02:42			7	9	10		13	14								40			
*02:43				9	10		13	14			21				36				
*02:44					10		13	14		18	20								
*02:47				9	10		13	14								37		43	
*02:48				9	10	11	12					24			31				
*02:50				9	10		13	15							33				
*02:51				9	10		13								35				
*02:55N				9	10		13	14					28					44	
*02:56				9	10		13	14	16	18		24			31	32			
*02:57				9	10		13	14									41		
*02:58				9		11	12							29					
*02:60				9	10		13	14				25					41		
*02:61Q				9	10		13										42		
*02:70				9	10		13												46
*02:73				9	10		13	14									42		
*02:78					10		14												47
*02:80N				9	10		13	14										44	
*02:81					10		13	14		20						38			
*02:83					10		13			20					33				
*02:89				9	10	11	12			18		24			31				
*02:93					10		13	14											

Negative Control

DRB3 2019 6K0

[illegible]

[illegible]

DRB3 2019 6K0

[illegible]

[illegible]

[illegible]

DRB3 2019 6K0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
DRB1*14:38:01, DRB1*14:50, DRB1*14:127:01				?																						26								33														
DRB1*14:38:02, DRB1*14:93, DRB1*14:127:02				?																															33													
DRB1*14:46	1				5		7	8					13				17			20								28										40										
DRB1*14:48		w		?																																												
DRB1*14:59, DRB1*14:86, DRB1*14:96, DRB1*14:145, DRB1*14:153-14:154				?								12																																				
DRB1*14:98, DRB1*14:165				?																						26																						
DRB1*14:141				4					9	10			13	14																																		
DRB1*14:174		2		?																																												
DRB4*01:105											11									20	21														36													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Negative Control																																																

Negative Control