

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB4 allele^{1,2,3}																																	
*01:01:01:01, 01:01:01:03, 01:01:03-01:01:04, 01:01:06-01:01:08, 01:75, 01:78-01:79, 01:95	1		3	4																													
*01:01:02, 01:01:09, 01:125	1		?	4																													
*01:01:05, 01:70			3	4																													
*01:02		2		4			7								?	?																	
*01:03:01:01, 01:03:01:03-01:03:01:12, 01:03:02-01:03:14, 01:03:16-01:03:18, 01:63, 01:72, 01:74, 01:76-01:77, 01:80N-01:81, 01:84N-01:89, 01:94, 01:96-01:97, 01:103-01:105, 01:107, 01:112	1			4			7																										
*01:03:01:02N	1				5		7																										
*01:03:01:13N	1				5		?																										
*01:03:15, 01:90	1			4																													
*01:04			?	4		6	?																										
*01:05	1		?	?			?		9												?												
*01:06	1		3	4							11																						
*01:07:01	1		?	4			?					12									?												
*01:07:02	1		?	4			?					12																					
*01:08, 01:39 ⁴	1		?	4			?						13																				
*01:09	1			4			7							14																			
*01:10	1			4			7								15																		
*01:11	1		?	4			?																						29				
*01:12	1		?	4		6	?						13																				
*01:13	1		?	4			?									16																	
*01:14N	1				5	6	7					12																					
*01:15	1		?	4			?																		25								
*01:16N	1		?	4			?																					28					
*01:17	1		?	4			?								16	17																	
*01:18	1		?	4			?																			26							
*01:19	1		?	4			?																				27						
*01:20, 01:36 ⁴	1		?	4			?													21													
*01:21	1		?	4			?							15		17																	
*01:22	1		?	4			?									17				20													
*01:23	1		?	4			?													20													
*01:24, 01:37 ⁴	1		?	4			?															23											
*01:25	1		?	4			?																			26		28					
*01:26	1		?	4			?										17																

Negative Control

[illegible]

[illegible]