

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
DRB5 allele^{1,2,3}																						
*01:01:01:01:01:01:06, 01:01:04:01:01:07, 01:01:09, 01:01:11:01:01:12, 01:19, 01:22:01:01:24, 01:26, 01:29:01:30, 01:33:01:34, 01:36:01:38, 01:40, 01:42, 01:45, 01:50, 01:53N-01:54, 01:57:01:58N, 01:62:01:63:01, 01:65, 01:80:01:81N, 01:83N-01:85, 01:88:01:89, 01:91:01:96, 01:99, 01:101N, 01:104:01:109, 01:111:01:112, 01:114:01:119, 01:121N-01:122, 01:124, 01:126	1	2	3	4	5																21	
*01:01:02	1	2	3	4									?									
*01:01:03, 01:66, 01:75, 01:86	1	2	3	4																	21	
*01:01:08	1	2	3	4																		
*01:01:10, 01:51, 01:63:02, 01:73, 01:78, 01:87, 01:123	1	2	3		5																21	
*01:02:01:01:02:02, 01:25, 01:28, 01:32, 01:35, 01:59:01:60, 01:64, 01:67N-01:68N, 01:70, 01:72, 01:76, 01:90, 01:102-01:103, 01:110, 01:113, 01:120N, 01:125N	1	2	3			6	7														21	
*01:03	1	2				6	7	8													21	
*01:04	1	2	3	4					9													
*01:05	1	2	3		5	6															21	
*01:06, 01:100				4				8			11											
*01:07	1	2	3	4	5				10				?								21	
*01:08:01N-01:08:02N, 01:56	1	2	3			6	7						13								21	
*01:09	1	2	3	4	5			8					?								21	
*01:10N	1	2	3			6	7								15						21	
*01:11	1			4				8			11											
*01:12	1	2	3							10					15							
*01:13, 01:41	1	2	3		5				9												21	
*01:14	1	2	3								11										21	
*01:15	1	2	3	4						10					15							
*01:16	1	2	3	4	5				9												21	
*01:17	1	2	3	4	5		7														21	
*01:18	1	2	3	4	5								13								21	
*01:20	1	2	3			6	7								16						21	
*01:21				4				8			11				16							
*01:27N, 01:46	1	2	3			6	7										17				21	
*01:31, 01:44		2	3	4	5																21	
*01:39						6	7														21	
*01:43		2		4				8								16						
*01:47	1	2	3	4	5										16						21	
*01:48N	1	2	3	4	5					10										20	21	
*01:49N	1	2	3	4	5														19		21	
*01:52N, 01:77	1	2	3																		21	
*01:55		2		4				8														
*01:61	1	2	3				7														21	
*01:69	1	2				6	7	8									17				21	
*01:71N	1			4																		
*01:74	1	2	3			6	7															
*01:79Q, 01:97:01:98				4	5																21	
*01:82	1	2		4	5																21	
*02:02:01:02:02:04, 02:09, 02:11, 02:14:02:16, 02:18, 02:20-02:23, 02:27:02:30, 02:33								8			11	12										
*02:03	1							8			11											
*02:04		2						8				12	?									
*02:05			3				7					12										
*02:06				4				8			11	12										
*02:07								8			11	12		14								
*02:08		2	3			6	7				12					16					21	
*02:10								8			11	12				16						
*02:12		2	3			6	7				12						17				21	
*02:13								8			11							18				
*02:17, 02:32								8			11											
*02:19N								8			11	12					17	18				
*02:24								8			12											
*02:25N		2	3			6	7				12										21	
*02:26N		2	3			6	7				12	13									21	
*02:31	1		3					8			11				15							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
DRB5 allele^{1,2,3}																						
DRB1*03:112, DRB1*09:06, DRB4*01:117																			18			
DRB1*08:59, DRB1*11:210																16						
DRB1*09:07	1	2	3												15							
DRB1*15:02:03, DRB1*16:01:02, DRB1*16:02:02, DRB4*01:05, DRB4*01:07:01													?									
DRB1*15:86													13									
DRB1*16:60				4	5																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22

Negative Control

Negative Control