

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-A allele ^{1,2}	ser ³																																
*01:01:01:01:01:01:22, 01:01:24-01:01:31, 01:01:33-01:01:47, 01:01:49-01:01:55, 01:01:57-01:02, 01:04N, 01:07, 01:09-01:11N, 01:16N, 01:18N, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N, 01:54-01:62, 01:64, 01:67:01-01:68, 01:70-01:72, 01:74-01:78, 01:80-01:94, 01:96-01:97, 01:100-01:120, 01:122-01:125, 01:128-01:129, 01:131-01:133, 01:135, 01:137-01:142, 01:144, 01:146	A1, Null, -	1			4																					26							
*01:01:23, 01:01:48, 01:08, 01:30, 01:79, 01:99, 01:145	A1, -	1																								26							
*01:01:32, 01:69:01-01:69:02, 01:143	-				4																					26							
*01:01:56	-	1			4									w												26					30		
*01:03, 01:06, 01:20, 01:53N, 01:63, 01:65-01:66, 01:73, 01:98, 01:127, 01:130	A1, Null, -	1			4																												
*01:12, 01:19	A1, -	1																										27					
*01:13	A1	1			4							11		13		15										26		28					
*01:14-01:15N	A1, Null	1																															
*01:17	A1	1			4																					26		28					
*01:21, 01:126	A1, -	1			4																					26	27						
*01:28	-	1			4							11				15										26							
*01:43	-	1			4							10														26							
*01:51	-	1			4				8			10														26							
*01:95	-	1			4														17							26							
*01:121	-	1			4															19						26							
*01:134	-	1			4																					26				29			
*01:136	-	1			4							11																					
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24-01:02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79-01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:155, 02:157-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:478	A2, Low A2, A203/ A2, A210/ A2, A2/bla nk, Null, -	2																															
*02:17:01-02:17:03	A2, -	2						w																									
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2	2												13						20		22						28					
*02:35:02	A2													13						20		22						28					
*02:46, 02:70	A2	2																		20		22											
*02:48, 02:129	A2, -																			20		22											
*02:55	A2/A2 8	2							8													21		23									
*02:65, 02:407, 02:449	A2/A1 9, -	2																	19														
*02:78	-	w			4															20		22						28					
*02:135	-	2												13											24						30		
*02:149	-	2																				21											
*02:156, 02:338	-	2																										27					
*02:169	-	2			4																												
*02:237	-	2													14	16								23									
*02:243:01-02:243:02	-	2																	18			21											
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*02:309	-	2											12		14							21			24								
*02:454	-	2											12		14										24								

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]