

INTERPRETATION TABLE																									
HLA-B low resolution SSP																									
Amplification patterns of the B*07:02 to B*83:01 alleles ¹²																									
		Well																							
		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.		110	215	140	130	185	190	290	165	165	180	105	280	195	105	115	80	150	135	60	210	170	110	395	160
PCR product(s)				235	265	235			330	220	390	190			435	395	150			400	245				425
Length of int. pos. control ¹		800	1070	1070	800	800	800	1070	1070	1070	800	1070	1070	1070	1070	1070	1070	800	1070	1070	800	800	1070	1070	1070
5'-primer(s) ²		527	97	209	103	103	103	45	45	45	420	206	419	142	45	161	167	355	206	45	142	409	246	44	44
		5'-TgA ³	5'-TCg ³	5'-ggC ³	5'-CCg ³	5'-CCT ³	5'-CCg ³	5'-ggA ³	5'-ggA ³	5'-ggA ³	5'-TTA ³	5'-AgA ³	5'-gTC ³	5'-TCT ³	5'-ggA ³	5'-Cgg ³	5'-gCT ³	5'-TCA ³	5'-gAC ³	5'-ggA ³	5'-TCT ³	5'-ggC ³	5'-gAA ³	5'-ggC ³	5'-ggC ³
		540		363	103	363	418		357	435		420		419	540					368	368	420			357
		5'-gAC ³		5'-AgC ³	5'-CCg ³	5'-AAT ³	5'-Agg ³		5'-Tgg ³	5'-AAA ³		5'-TTA ³		5'-gTC ³	5'-gAC ³					5'-gTT ³	5'-gTC ³	5'-TTA ³		5'-Tgg ³	
				363	361				412											557					
				5'-AgC ³	5'-AgT ³				5'-ATA ³											5'-ggA ³					
3'-primer(s) ³		605	272	309	193	246	246	165	206	266	559	272	3 ^d I	301	272	234	204	463	302	259	311	544	317	272	302
		5'-gCT ³	5'-Tgg ³	5'-gTg ³	5'-CgT ³	5'-TAT ³	5'-TAT ³	5'-Tgg ³	5'-CCT ³	5'-TCC ³	5'-CTC ³	5'-Tgg ³	5'-TAT ³	5'-gTC ³	5'-TgC ³	5'-TCT ³	5'-TCT ³	5'-gCT ³	5'-ggC ³	5'-gTT ³	5'-ggg ³	5'-ggT ³	5'-ggA ³	5'-TgC ³	5'-ggC ³
			272	559	193	246	572		538	559		572		301	309	272				259	311				302
			5'-TgA ³	5'-Cag ³	5'-CgT ³	5'-TAT ³	5'-gCC ³		5'-gTC ³	5'-Cag ³		5'-gCg ³		5'-gTC ³	5'-ATC ³	5'-TgA ³				5'-gTT ³	5'-ggg ³				
					583	559				583				570	605	272				266	538				477
					5'-gTg ³	5'-CTC ³				5'-gTg ³				5'-CCg ³	5'-gCT ³	5'-Tgg ³				5'-TCC ³	5'-gTC ³			5'-gCg ³	
														572						272					
														5'-gCg ³						5'-Tgg ³					
																				5'-gTA ³					
																				572					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ⁴	ser. ⁵																								
*07:02:01-07:02:20, 07:05:01-07:07, 07:10, 07:12-07:15, 07:18:01- 07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34- 07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55- 07:59, 07:61-07:64, 07:66- 07:68:02, 07:70-07:72, 07:74-07:77, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:117	B7, B42, Null, -	1																							24
*07:03, 07:08, 07:16, 07:32, 07:37	B7, B703, -	1																							24
*07:04, 07:25	B7	1													14										24
*07:09, 07:11, 07:17	B7	1											12												24
*07:19, 07:33, 07:40, 07:48, 07:51, 07:53	B7, B42																								24
*07:20, 07:24	B7, -	1		3																					24
*07:27	-	1										11								20					
*07:28	B7	1				6																			24
*07:36, 07:81	B7, -	1																							
*07:38	-	1																							
*07:47	B7	1																							24
*07:50	-	1										11													24
*07:54	B7	1																							24
*07:60	-			3																					24
*07:65	-	1																	18						
*07:69, 07:85	-	1																					23	24	
*07:73	-	1														16									24
*07:78	-	1									10														24
*07:84	-	1																							24
*07:100	-			3																					24
*08:01:01-08:01:13, 08:05, 08:08N, 08:10- 08:12:03, 08:15-08:16, 08:18-08:19N, 08:22- 08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41- 08:48, 08:51, 08:53, 08:56- 08:59, 08:61, 08:63-08:64, 08:66	B8, Null, -		2						8																24
*08:02-08:03, 08:52	B8, -		2						8																
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:04	B8, –		2						8			11													24
*08:07, 08:14	B8		2																						24
*08:09	B8		2						8																24
*08:13, 08:20, 08:40	B8, –		2																						24
*08:17, 08:54	–		2						8			11													24
*08:21	B8		2	3					8																24
*08:25	B8		2	3																					24
*08:26, 08:50, 08:62, 42:07 ⁶	B8, –								8																24
*08:28, 08:37	B8, –		2										12												24
*08:32	B8		2						8										18						24
*08:35	–		2						8				12												24
*08:36	–		2						8																
*08:38	–		2						8																24
*08:49	–		2															17			20				24
*08:55	–		2																			21			24
*08:60	–		2						8																24
*08:65	–								8															23	24
*13:01:01-13:01:05, 13:17, 13:20, 13:22:01-13:23, 13:25, 13:28-13:29	B13, –			3	4						10							17							
*13:02:01-13:02:09, 13:08Q, 13:15, 13:18-13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40	B13, –			3	4						10														
*13:03	B49, B15			3	4																				
*13:04, 13:10	B49, B15, B13			3	4								12												
*13:06	B13			3	4													17							
*13:07N	Null			3	4						10							17							
*13:09	–				4						10														
*13:11	B13			3	4						10														
*13:12-13:13	B13			3	4													17							
*13:14	B13			3							10														
*13:16	B13			3	4						10														
*13:21, 13:35	–			3	4																				
*13:26	–			3	4								12					17							
*13:31	–			3	4						10														
*13:36	–			3	4						10							17							
*13:39	–				4						10							17							
*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19	B64, Null, –				4	5																		23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

*14:02:01-14:02:02, 14:02:04, 14:03-14:04, 14:09, 14:11, 14:15-14:18	B65, B14, –				4			6																	23	24
*14:02:03, 14:02:06	B65, –				4																				23	24
*14:02:05	B65				4			6																	23	
*14:05-14:06:02, 14:13	B14, –							6																	23	24
*14:08, 39:25N, 39:30, 39:32-39:34, 39:47, 39:50 ⁷	B39, B16, Null, –							5																	23	24
*14:10	–							5									17								23	24
*15:01:01:01- 15:01:01:02N, 15:01:03, 15:01:06-15:01:16, 15:01:18, 15:01:20, 15:26N-15:27:03, 15:32, 15:34-15:35, 15:38:01- 15:38:02, 15:50, 15:56, 15:60, 15:70-15:71, 15:75, 15:79N, 15:81-15:82, 15:92, 15:94N, 15:96- 15:97, 15:102, 15:104- 15:105, 15:109-15:111N, 15:113, 15:117-15:118, 15:122, 15:125, 15:128- 15:129, 15:135, 15:140, 15:145-15:147, 15:152, 15:159-15:160, 15:163- 15:164, 15:166-15:167, 15:169, 15:171-15:172, 15:174-15:175, 15:178, 15:181N-15:184, 15:187, 15:190N, 15:192-15:193, 15:201, 15:203, 15:206	B62, Null, –							7				12						19								
*15:01:02	B62							7		9		12							19							
*15:01:04	B62											12							19							
*15:01:17, 15:05:01- 15:06, 15:33, 15:39:01- 15:40, 15:65, 15:78:01- 15:78:02, 15:84, 15:107, 15:136, 15:141, 15:155, 15:179, 15:185, 15:195, 15:199	B62, –							7				12							19							
*15:01:19, 15:28, 15:120, 15:149N	B62, Null, –							7				12							19							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

*15:02:01-15:02:05, 15:88, 15:112, 15:121, 15:139, 15:170, 15:194	B75, –							7				12					17		19						
*15:03:01-15:03:02, 15:47, 15:49, 15:61, 15:98, 15:103, 15:123, 15:127, 15:132, 15:151, 15:156, 15:158, 15:173	B72, B70, –							8				11	12						19						
*15:03:03	B72							7	8			11	12						19						
*15:04	B62							7					12						19						
*15:07:01-15:07:02, 15:126, 15:205, 15:207	B62, –			3				7					12						19						
*15:08, 15:11:01- 15:11:03, 15:11:05, 15:15, 15:148, 15:191, 15:209N	B75, B62, Null, –							7					12						19						
*15:09	B70							8	9					14					19						
*15:10:01, 15:37, 15:90, 15:99, 15:133	B71, B70, –							8						14					19						
*15:10:02	B71							7	8					14					19						
*15:11:04	B75												12						19						
*15:12, 15:14, 15:19	B76							7					12	13					19						
*15:13	B77							7					12	14			17		19						
*15:16, 15:67, 15:95	B63								9				12	14					19						
*15:17:01:01-15:17:02, 15:162, 15:168, 15:177, 15:196, 15:208	B63, –							7		9				14					19						
*15:18:01-15:18:02, 15:18:04, 15:51-15:52, 15:72, 15:93, 15:108, 15:114, 15:119, 15:134, 15:153, 15:176, 15:197- 15:198, 15:200	B71, B70, –							8					12	14					19						
*15:18:03	B71							8					12	14					19						
*15:20, 15:25:01- 15:25:03, 15:204	B62, –							7					12				17		19						
*15:21, 15:44	B75							7					12	14			17		19						
*15:23, 15:115	B70, B5, –								8				12	14					19						
*15:24, 15:87, 15:157	B62, B15, –							7					12	14					19						
*15:29	B70							8					12						19						
*15:30, 15:58, 15:63	B62							7											19						
*15:31	B75							7					12						19						
*15:36	–			w				7					12				17		19						
*15:42	B15							7											19						
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*15:43	B15							7					12							19					
*15:45	B62			3				7												19					
*15:46	–				4			7	8				12							19					
*15:48, 15:150, 15:188	B62, –							7												19					
*15:53	–				4				8				12							19					
*15:54	B72								8			11	12							19					
*15:55	B15			3				7					12							19					
*15:57	B62							7					12							19					
*15:62	–								8			11	12					17		19					
*15:64	B71								8			11	12							19					
*15:66	B62							7					12		14					19					
*15:68	–			3								11	12												
*15:69	B72								8			11	12							19		21			
*15:73	B62							7												19					
*15:74	B72								8				12							19					
*15:76	–							7					12							19					
*15:77	B62						6	7										17		19					
*15:80	B70								8				12		14			17		19					
*15:83	B62							7	8											19					
*15:85, 15:154, 15:165	B62, –							7					12					17		19					
*15:86	–							7												19					
*15:89	B77			w				7					12					17		19					
*15:91, 15:131	–								8			11	12	13						19					
*15:101	–							7					12							19					
*15:106	–				4			7	8				12					17		19					
*15:116	B62							7					12							19					
*15:124	B71								8				12		14					19					
*15:137	–							7												19					
*15:138	–	1						7					12							19					
*15:142	–		2					7					12							19					
*15:143	–							7	8				12							19					
*15:144	–							7					12					17		19					
*15:161	–								8				12	13	14					19					
*15:180	–		2						8				12							19					
*15:186	–								8				12		14					19		21			
*15:189	–						6	7					12		14					19					
*15:202	–							7					12						18	19					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*18:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:52	B18, Null, –												12			15			18						

*18:04	B18													12			15			18					
*18:09	B18													12			15								
*18:12	B18													12			15			18					
*18:19	B18													12			15			18					
*18:22	B18													12			15	17	18						
*18:26, 18:41	B18, –																15			18					
*18:29, 35:32, 35:37, 35:53N, 35:64, 35:66, 35:68:01-35:68:02, 35:99, 35:118-35:119 ⁸	B35, Null, –													12						18					
*18:37	–												11	12			15								
*18:44	–				4									12			15			18					
*18:48	–													12			15								
*18:49	–													12			15								
*27:01	B27					5											16								
*27:02, 27:53, 27:57, 27:65N	B27, Null, –					5											16								
*27:03-27:06, 27:09-27:10, 27:13, 27:17, 27:20, 27:27, 27:31, 27:35-27:37, 27:45-27:46, 27:48-27:51, 27:54-27:56, 27:58, 27:60-27:61, 27:64N, 27:66N, 27:68-27:69, 27:72-27:73	B27, Null, –					5											16								
*27:07, 27:11, 27:15, 27:21, 27:28, 27:32, 27:34, 27:38, 27:43, 27:47, 27:63, 27:70-27:71	B27, –																16								
*27:08, 27:26, 27:40, 27:42, 27:44	B2708, B27, B7, –					5											16								
*27:12	–					5											16								
*27:14	B27																16						24		
*27:16, 27:39, 27:59N	B27, Null, –					5											16								
*27:18	–					5						11					16								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*27:19	B27																16	17							
*27:23	–					5											16								
*27:24	B27																16								
*27:25	B27																16								
*27:29	B27					5						11					16								
*27:30	B27																16	17							
*27:33	–																16								
*27:41	–					5						12					16								
*27:52, 27:67	–					5																			
*27:62	–																16								

[illegible]

*35:26	B35					6						12					17								
*35:27	B35											12					17								
*35:35	B35											12					17	18			21				
*35:38	B35																17	18							
*35:42:02	-											12					17	18							
*35:43:01, 35:67, 35:79, 35:86, 35:102, 35:117, 35:135	B35, -											12						18							
*35:43:02	-											12						18							
*35:44	-																	18							
*35:45, 35:71	B35											12	13				17	18							
*35:46	B35				4							12					17								
*35:47	B35				4							12					17								
*35:60	B35																	18							
*35:63	-				4							12					17								
*35:76	B35, 22											12					17	18							
*35:81, 35:96, 35:109	-																17	18							
*35:87	-							8										18							
*35:115	-											12					17	18							
*37:01:01-37:01:07, 37:03N, 37:06, 37:13, 37:15-37:21, 37:23-37:25	B37, Null, -													13			17			20					
*37:02	B37, B27				5									13						20					
*37:04:01-37:04:02	B37, -													13			17			20					
*37:05	-													13			17			20					
*37:07	-	1												13						20					
*37:08	B37													13			17			20					
*37:09	B37													13						20					
*37:10, 37:22	B37, -																17			20					
*37:11	-																17	18		20					
*37:12	B37							8						13						20					
*37:14	-																17			20					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*38:01:01-38:02:02, 38:08- 38:09, 38:11-38:16, 38:18- 38:19, 38:21, 38:23-38:24	B38, -					5																21	22	23	
*38:02:03	-																					21		23	
*38:03	B38					5						11										21	22		
*38:04, 38:25	B38, -																					21		23	
*38:05	B38						6															21	22	23	
*38:06	B38																					21			
*38:07	B38					5																21			
*38:10	B38					5																	22	23	
*38:17	-					5								13						20		21	22	23	
*38:20	-					5												17				21	22	23	
*38:22	-					5											16						22	23	

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
																								ser. ⁵	HLA-B allele ⁴
			28																			47		B7, B42, Null, –	*07:02:01-07:02:20, 07:05:01-07:07, 07:10, 07:12-07:15, 07:18:01-07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34-07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55-07:59, 07:61-07:64, 07:66-07:68:02, 07:70-07:72, 07:74-07:77, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:117
																						47		B7, B703, –	*07:03, 07:08, 07:16, 07:32, 07:37
		27	28																			47		B7	*07:04, 07:25
			28																			47		B7	*07:09, 07:11, 07:17
			28																			47		B7, B42	*07:19, 07:33, 07:40, 07:48, 07:51, 07:53
			28																			47		B7, –	*07:20, 07:24
																					46			–	*07:27
			28																			47		B7	*07:28
			28																			46		B7, –	*07:36, 07:81
																					46			–	*07:38
			w																			47		B7	*07:47
																						47		–	*07:50
			28										38									47		B7	*07:54
			28																			47		–	*07:60
			28				32								40				w			47		–	*07:65
																						47		–	*07:69, 07:85
			28																			47		–	*07:73
			28							35												47		–	*07:78
			28							35												47		–	*07:84
			28																		45	47		–	*07:100
			27																			47		B8, Null, –	*08:01:01-08:01:13, 08:05, 08:08N, 08:10-08:12:03, 08:15-08:16, 08:18-08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41-08:48, 08:51, 08:53, 08:56-08:59, 08:61, 08:63-08:64, 08:66
		27																				46		B8, –	*08:02-08:03, 08:52
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
		27																				47		B8, -	*08:04
		27																				47		B8	*08:07, 08:14
		27		30																		47		B8	*08:09
																						47		B8, -	*08:13, 08:20, 08:40
		27										38										47		-	*08:17, 08:54
		27																				47		B8	*08:21
																						47		B8	*08:25
		27																				47		B8, -	*08:26, 08:50, 08:62, 42:07 ⁶
		27																				47		B8, -	*08:28, 08:37
		27																44				47		B8	*08:32
		27																				47		-	*08:35
		27																				47		-	*08:36
		27										38										47		-	*08:38
		27		29																		47		-	*08:49
																						47		-	*08:55
		27		29																		47		-	*08:60
		27																				47		-	*08:65
												38										46		B13, -	*13:01:01-13:01:05, 13:17, 13:20, 13:22:01- 13:23, 13:25, 13:28-13:29
												38										46		B13, -	*13:02:01-13:02:09, 13:08Q, 13:15, 13:18- 13:19, 13:27, 13:30, 13:32 13:34, 13:37-13:38, 13:40
												38			41							46		B49, B15	*13:03
												38										46		B49, B15, B13	*13:04, 13:10
												38		40								46		B13	*13:06
												38										46		Null	*13:07N
												38										47		-	*13:09
												38					43					46		B13	*13:11
												38										46		B13	*13:12-13:13
												38										46		B13	*13:14
										35		38										46		B13	*13:16
												38										46		-	*13:21, 13:35
												38										46		-	*13:26
										w		38								45	46			-	*13:31
								33														46		-	*13:36
												38										47		-	*13:39
																						47		B64, Null, -	*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.

[illegible]

25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
													38							45	46			B15	*15:43
													38							45		47		B62	*15:45
												37	38							45		47		-	*15:46
													38									47		B62, -	*15:48, 15:150, 15:188
												37	38							45		47		-	*15:53
													38							45		47		B72	*15:54
								33														47		B15	*15:55
					w								38							45		47		B62	*15:57
													38									47		-	*15:62
																						47		B71	*15:64
													38							45		47		B62	*15:66
													38							45		47		-	*15:68
													38									47		B72	*15:69
													38			41		43		45		47		B62	*15:73
													38									47		B72	*15:74
			28					w												45		47		-	*15:76
													38							45		47		B62	*15:77
																						47		B70	*15:80
					30								38									47		B62	*15:83
													38							45		47		B62, -	*15:85, 15:154, 15:165
													38		40							47		-	*15:86
								33													46			B77	*15:89
													38									47		-	*15:91, 15:131
			28										38							45		47		-	*15:101
												37	38									47		-	*15:106
25													38									47		B62	*15:116
25																						47		B71	*15:124
										w			38							45		47		-	*15:137
													38									47		-	*15:138
													38							45		47		-	*15:142
												37								45		47		-	*15:143
								33												45		47		-	*15:144
																						47		-	*15:161
																				45		47		-	*15:180
																						47		-	*15:186
								33												45		47		-	*15:189
													38							45		47		-	*15:202
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
																			44			47		B18, Null, -	*18:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:52

[illegible]

[illegible]

28															46				Negative Control	–	*38:26					
																		47		B3901, B39, Low B39, Null, –	*39:01:01:01-39:01:01:02L, 39:01:03-39:01:08, 39:01:10-39:01:12, 39:03, 39:05:01-39:06:02, 39:09, 39:11, 39:14-39:15, 39:24, 39:26-39:29, 39:31, 39:35, 39:37 39:38Q, 39:40N-39:41, 39:44, 39:46, 39:48, 39:51 39:57, 39:59-39:62					
																		47		B3901, B39, –	*39:01:09, 39:12, 39:45					
										38								47		B3902, B39	*39:02:01, 39:08					
										38								47		B3902, B39, –	*39:02:02, 39:13:01-39:13:02, 39:23, 39:49					
																		47		B39	*39:04					
																		47		B16	*39:07					
																		47		B39	*39:10, 39:16, 39:20					
												40		43				47		B39	*39:17					
																45		47		B39	*39:18					
															44			47		B39	*39:19:01					
															44			47		B39	*39:19:02					
										38								47		B39	*39:22					
																45		47		–	*39:36					
																		47		B39	*39:39					
																		47		B39	*39:42					
																		47		–	*39:43					
																		47		–	*39:58					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43		44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43		44	45	46	47	48		Well No.
25												37	38									47			*40:01:01-40:01:06, 40:01:08-40:01:10, 40:01:12-40:01:17, 40:07, 40:51, 40:66, 40:73, 40:79, 40:124:01, 40:139-40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N	
												37	38									47	B60		*40:01:07, 40:72:01, 40:77	
25												37	38		40							47	–		*40:01:11	
	26											37	38									47	B61, B41, Null, –		*40:02:01-40:02:11, 40:04, 40:39, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N	
	26											37	38									47	B61		*40:03, 40:20	

[illegible]

					29															46			Negative Control	B44, Null, –	*44:02:01-01-44:02:14, 44:11, 44:19N, 44:21- 44:24, 44:27, 44:33-44:34, 44:48, 44:52N, 44:55, 44:59, 44:63, 44:66-44:68, 44:71-44:74, 44:80, 44:84, 44:86-44:87, 44:89, 44:93, 44:101-44:102, 44:104 44:112-44:113
					29															46				B44, Null, –	*44:03:01-44:03:06, 44:03:08-44:05:02, 44:13, 44:26, 44:28:01-44:30, 44:32, 44:35-44:36, 44:38- 44:39, 44:42, 44:45, 44:51, 44:56N, 44:58N, 44:61N, 44:69-44:70, 44:76-44:79, 44:82, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:105, 44:107-44:109, 44:114- 44:115
					29							38								46				B44, –	*44:03:07, 44:07, 44:81, 44:103, 44:111
					29	30					36		39				44		46					B44	*44:06
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
				29									38	39							46			B44	*44:08
				29								37	38	39								47		B45	*44:09
				29									38			41					46			B44	*44:10
				29										39							46			B44	*44:12
				29									38								46			B44	*44:14
					30								38			41					46			B12	*44:15
				29									38	39							46			B47, B44, –	*44:16, 44:91
				29									38								46			B44, –	*44:17, 44:43:01-44:43:02
					30				34				38			41					46			B44	*44:18
				29	30								38	39							46			B44, –	*44:20, 44:100
				29					34		36		38								46			B44	*44:25, 44:50
25				29									38								46			B44	*44:31
													38								46			B44, –	*44:37, 44:64:01-44:64:02
				29									38								46			B44	*44:40
				29									38	39							46			B44, –	*44:41, 44:53, 44:99
				29									38	39							46			B44	*44:44
				29								37	38									47		B12, –	*44:46, 44:75
				29									38								46			B44, –	*44:47, 44:65
				29									38	39							46			B44	*44:49
				29									38								46			B44	*44:54

[illegible]

[illegible]

					30					35	36							44	46			B51	*51:24:02		
					30					35	36							44	46			B51, -	*51:24:03-51:24:04		
					30					35	36							44	46			B51	*51:36		
					30					35	36							44	46			B51, -	*51:37, 51:63		
				29	30						36							44	46			-	*51:42		
					30						36							44	46			-	*51:45		
										35								44	46			-	*51:54, 51:78		
					30					w	36							44	45	46		-	*51:61		
					30						36			41	43	44		44	46			-	*51:62		
					30						36							44	46			-	*51:64		
					30					35	36							44	46			-	*51:68		
					30						36							44	46			-	*51:81		
										35	36							44	46			-	*51:93		
					30					35	36							44	46			-	*51:97		
					30					35	36							44	46			-	*51:101		
					30			33		35	36								46			-	*51:104		
					30					35	36			41				44	46			-	*51:106		
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
										35	36		38							46			B52, -	*52:01:01, 52:01:03, 52:01:06, 52:04-52:05, 52:07-52:08, 52:10-52:13, 52:15, 52:17-52:18, 52:22	
					30					35	36		38							46			B52	*52:01:02, 52:01:04, 52:02-52:03, 52:06:01-52:06:02, 52:09	
										35	36		38							46			B52, -	*52:01:05, 52:14	
										35	36		38							46			-	*52:16	
					30					35	36		38							46			-	*52:19	
										35			38							46			-	*52:20	
					30					35	36		38						45	46			-	*52:21	
											36							44	46				B53, -	*53:01:01-53:01:05, 53:05, 53:08:01-53:08:02, 53:10, 53:15-53:16, 53:18, 53:20-53:21, 53:23-53:24	
					31					36								44	46				B53	*53:02	
																		44	46				B53, B35	*53:03, 53:09, 53:11-53:13	
										36								44	46				B53, -	*53:04, 53:07, 53:19	
					30	31				36								44	46				B53, B51	*53:06	
										36								44	46				-	*53:14	
										36		38							46				-	*53:17	
				29						36								44	46				-	*53:22	

			28			32								39	40												47		B54, Null, –	*54:01-54:02, 54:05N, 54:07-54:08N, 54:10, 54:13, 54:16-54:19, 54:21- 54:23	
			28			32								39	41												47		B22	*54:03	
			28			32								39													47		B54, –	*54:04, 54:15	
			28			32								39												45	47		–	*54:06	
			28			32								39													47		–	*54:09, 54:14	
			28			32								39													47		B54	*54:11	
			28			32				34				39	40											46		–	*54:12		
			28			32				35				39	40												47		–	*54:20	
			28			32				35				40													47		B55, –	*55:01:01-55:01:06, 55:03, 55:05, 55:11, 55:15, 55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:44-55:45	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48							Well No.	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48								Well No.
			28				32								40												47		B55, B54, –	*55:02:01-55:02:06, 55:07, 55:10, 55:12, 55:16, 55:19, 55:26, 55:30, 55:37, 55:39, 55:41- 55:43, 55:47-55:48	
			28				32																				47		B55, B56, Null, –	*55:04, 55:08, 55:13, 55:27, 55:46, 56:15, 56:19N, 56:22 ¹⁰	
			28				32				35																47		B22, B55	*55:09, 55:24	
			28				32											42									47		–	*55:14	
			28				32				35																47		B55, –	*55:17, 55:28	
														38	40												47		–	*55:18	
		27	28		30		32																				47		B22	*55:20	
			28				32				w				40						45					47		B55	*55:21		
			28				32																				47		B22	*55:22	
			28				32																				47		B55, –	*55:23, 55:32, 56:18 ¹¹	
			28				32							38	40												47		–	*55:34	
			28				32			w					40												47		–	*55:35	
			28				32				35			38	40												47		–	*55:40	
			28				32									41											47		B56, Null, –	*56:01:01-56:01:04, 56:08, 56:14, 56:16, 56:20, 56:24, 56:26-56:30	
			28				32									41		43								47		B56	*56:02, 56:04		
			28				32															45				47		B22, B15	*56:03		
			28				32				35				40											47		B56	*56:05:01		
			28				32				35															47		B56	*56:05:02		
			28				32				35				40					w						47		B78	*56:06		
			28				32									41										46			B56	*56:07	

[illegible]

			28																	45	47		B73, –	*73:01-73:02	
										35				40					44		47		B78, –	*78:01, 78:02:02-78:03, 78:07	
										35									44		47		B78, B35	*78:02:01, 78:04	
										35			38								47		–	*78:05	
										35			38								47		–	*78:06	
			28				32	33													47		B81	*81:01	
			28					33													47		B81	*81:02	
			28					33													47		Null, –	*81:03-81:05	
			28				32									41					47		B82, –	*82:01-82:03	
			28	29			32							39							47		–	*83:01	
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.
																									A*23:31, A*24:106, C*16:10
																				45					A*68:56, C*06:20, C*12:50
																									C*01:30
																		w							C*02:06
																									C*02:23, C*04:77
																									C*03:05, C*03:25, C*03:27
																41									C*03:12, C*03:19
																41									C*03:102
																									C*07:46
															40										C*15:02:04
				29																					C*15:25
						31																			C*15:39
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48		Well No.