

[illegible]

[illegible]

[illegible]

[illegible]

*27:02:01-27:02:02, 27:53, 27:57, 27:65N, 27:83	B27, Null, –					5										16												28			32
*27:03-27:05:15, 27:05:17-27:06, 27:09-27:10, 27:13, 27:17, 27:20, 27:31, 27:35-27:37, 27:45-27:46, 27:48-27:51, 27:54-27:56, 27:58, 27:60-27:61, 27:64N, 27:66N, 27:68-27:69, 27:72-27:74, 27:78-27:80, 27:82, 27:84-27:86	B27, Null, –					5										16												28			32
*27:05:16, 27:21, 27:38, 27:47, 27:63, 27:70, 27:76	B27, –															16												28			32
*27:07:01, 27:11, 27:32, 27:34, 27:43	B27, –															16												28			32
*27:08, 27:26, 27:40, 27:42, 27:44	B2708, B27, B7, –					5										16												28			32
*27:12	–					5										16															
*27:14, 27:81	B27, –															16							24					28			32
*27:15, 27:28, 27:71	B27, –															16												28			32
*27:16, 27:39, 27:59N	B27, Null, –					5										16															
*27:18	–					5						11				16															
*27:19	B27															16	17											28			32
*27:23	–					5										16															
*27:24	B27															16												28			32
*27:25	B27															16												28			32
*27:27	B27					5										16												28			32
*27:29	B27					5						11				16															
*27:30	B27															16	17											28			32
*27:33	–															16												28			32
*27:41	–					5						12				16												28			32
*27:52	–					5																						28			32
*27:62	–															16												28			32
*27:67	–					5																						28			32
*27:75	–					5																						28			32
*27:77	–					5										16															
Well No.		1	2	3	4	5	6	7	8	9	10	11																			

[illegible]

[illegible]

[illegible]

*44:16, 44:91, 44:132	B47, B44, –			4		6										17												29					
*44:17, 44:43:01-44:43:02	B44, –			4							12	13				17											29						
*44:18	B44			4			8			11																		30					
*44:20, 44:100	B44, –			4			8																				29	30					
*44:24	B44			4												17											29						
*44:25, 44:50	B44			4												17											29						
*44:31	B44			4																		25				29							
*44:37, 44:64:01-44:64:02	B44, –			4		6										17											29						
*44:40, 44:130	B44, –									11					16	17											29						
*44:41:01-44:41:02, 44:53, 44:99	B44, –			4																							29						
*44:44	B44									11					16	17											29						
*44:45, 44:51, 44:77, 44:82, 44:107, 44:123	B44, –			4												17											29						
*44:46, 44:129	B12, –			4												17											29						
*44:47, 44:65	B44, –			4																							29						
*44:49	B44		2	4												17											29						
*44:54	B44			3	4																						29						
*44:57	–			w												17											29						
*44:60, 44:110	–			3	4											17											29						
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*44:62	–			4				8									17											29					
*44:75	–			4													17											29					
*44:83	–																17											29					
*44:90	–			4													17											28	29		32		
*44:95	–			4													17											29					
*44:97	–			4													17											28	29		32		
*44:106	–			3	4																							29					
*44:127	–			4													17											29					
*44:131	–																17											29					
*44:134	–																17											29					
*44:135	–			3	4																												
*45:01, 45:05, 45:07, 45:11-45:13	B45, –			4				8			11													24				30					
*45:02-45:03	B45			4				8																24				30					
*45:04	B21			4				8																24				30					
*45:06	B45			4				8			11													24			28	30					
*45:08	–			4				8																24				30					
*45:09	–			4				8				12	13											24				30					
*45:10	–			4				8		10														24				30					
*46:01:01-46:01:05, 46:01:07, 46:04-46:05, 46:07N, 46:09-46:10, 46:14-46:16, 46:22-46:24, 46:27-46:30	B46, Null, –							7				12															28		31				
*46:01:06	–							7				12															28		31				
*46:02	B46							7				12															w		31				
*46:03, 46:08, 46:13:01-46:13:03, 46:19, 46:21:01-46:21:02, 46:25-46:26	B46, –							7				12															28		31				
*46:06	–							7				12													25		28		31				
*46:11	B22							7				12															28		31				
*46:12, 46:20	–			3				7				12															28		31				
*46:17	–							7				12	13														28		31				
*46:18	–							7																			28		31				
*47:01:01-47:01:02, 47:06-47:08	B47, –			4																					26					32			
*47:02	B47			4																						26				32			
*47:03	B47			4																						26				32			
*47:04-47:05	B17, –			4	5																					26							

[illegible]

[illegible]

[illegible]

[illegible]

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*59:01:01:01-59:01:01:02, 59:05	B59, –																																
*59:02	B59																																
*59:03	–																																
*59:04	–																																
*67:01:01	B67					5																21		24				28					
*67:01:02	B67																					21		24				28					
*67:02	–																					21		24				28					
*67:03	–					5																21		24				28					
*73:01-73:02	B73, –																								24			28					
*78:01:01-78:01:02, 78:07	B78, –																	18															
*78:02:01, 78:04	B78, B35																	18															
*78:02:02	B78																	18															
*78:03	B78																	18															
*78:05	–																	18															
*78:06	–																																
*81:01	B81																											28				32	
*81:02	B81																							24				28					
*81:03-81:05	Null, –																											28					
*82:01-82:03	B82, –							8			11								19										28				32
*83:01	–																	17										28	29			32	
A*23:31, A*24:106, C*16:10						6																											
A*24:146																																	
A*24:174														13																			
A*26:68, A*68:56, C*06:20, C*12:50																																	
C*01:30					4																												
C*02:06, C*02:47																																	
C*02:23, C*04:77		1																															
C*03:05, C*03:25, C*03:27				3																													
C*03:12, C*03:19																																	
C*03:102																		17															
C*03:129																										25							
C*05:30																																	
C*07:46								8																									
C*08:51																																	
C*15:02:04																																	
C*15:25																														29			
C*15:39																																31	
C*15:51																						21											
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

[illegible]

[illegible]

[illegible]

[illegible]

Negative Control

[illegible]

				38								47												57								63				Negative Control	B3902, B39, –		*39:02:02, 39:13:01-39:13:02, 39:23, 39:49				
								47												57								63				B39		*39:04									
								47												57								63				B16		*39:07									
								47												57				62				63				B39		*39:10:01, 39:16, 39:20									
				40				43				47								57								63				–		*39:10:02									
								45				47								57								63				B39		*39:17									
								44				47																63				B39		*39:19:01									
								44				47																63				B39		*39:19:02									
				38								47								57								63				B39		*39:22									
												47								57								63				–		*39:24:02									
												47								57								63				B39, Null		*39:25N, 39:30, 39:34									
												47				50												63				B16		*39:32									
												47																63				B39, –		*39:33, 39:47, 39:50									
								45				47								57								63				–		*39:36									
												47								57								63				B39		*39:39									
												47								57								63				B39		*39:42									
												47				50								57				63				–		*39:43									
												47				50								57				63				–		*39:48									
												47												57				63				–		*39:58									
				40								47								57				58				62				63				–		*39:63					
												47								57												–				*39:68							
				37 38								47																								B60, Null, –		*40:01:01-40:01:05, 40:01:08-40:01:10, 40:01:12 40:01:17, 40:01:19-40:01:23, 40:07, 40:66, 40:73, 40:79, 40:124:01, 40:139-40:140, 40:146-40:147, 40:150, 40:152-					
				37 38								47								59												B60		*40:01:06									
				37 38								47																				B60, –		*40:01:07, 40:01:18, 40:72:01									
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.										
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.										
				37 38				40								47																								–		*40:01:11	
				37 38												47								55												B61, Null, –		*40:02:01-40:02:12, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N					
				37 38												47								55												B61		*40:03					
				37 38												47																				B61, B41		*40:04, 40:39					
				37 38												47																				B4005, B50		*40:05					
				37 38												47																				B61, –		*40:06:01:01-40:06:04,					
				37												47				51				55												B61, –		*40:08					
				37 38												47								55												B61, –		*40:09, 40:18, 40:24, 40:27:01-40:27:02, 40:29, 40:56-40:57, 40:78, 40:82, 40:85, 40:90-40:91					

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
	34						40						46											57					63		B59, –	*59:01:01:01-59:01:01:02, 59:05	
	34												46					52						57							B59	*59:02	
	34												46											57					63		–	*59:03	
	34							41					46																63		–	*59:04	
														47										57				62	63		B67	*67:01:01	
														47										57				62	63		B67	*67:01:02	
														47															63		–	*67:02	
														47										57				62			–	*67:03	
												45	47															61			B73, –	*73:01-73:02	
		35					40				44			47		49	51								58				63		B78, –	*78:01:01-78:01:02, 78:07	
		35									44			47		49	51												63		B78, B35	*78:02:01, 78:04	
		35					40				44			47		49	51												63		B78	*78:02:02	
		35					40				44			47		49									58				63		B78	*78:03	
		35			38									47		49										59	60		63		–	*78:05	
		35			38									47		49										59	60	61	63		–	*78:06	
33														47	48							55									B81	*81:01	
33														47	48							55									B81	*81:02	
33														47	48							55									Null, –	*81:03-81:05	
								41						47									56						63		B82, –	*82:01-82:03	
							39							47							54	56									–	*83:01	
																			52													A*23:31, A*24:106, C*16:10	
																																A*24:146	
																																A*24:174	
												45																				A*26:68, A*68:56, C*06:20, C*12:50	
																																C*01:30	
																																C*02:06, C*02:47	
																																C*02:23, C*04:77	
																																C*03:05, C*03:25, C*03:27	
																																C*03:12, C*03:19	
																																C*03:102	
																																C*03:129	
																				53												C*05:30	
																								57								C*07:46	
																		50														C*08:51	
							40																54									C*15:02:04	
																																C*15:25	
																																C*15:39	
																																C*15:51	
33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

Negative Control