

INTERPRETATION TABLE						
HLA-B*07 Add-on SSP typing						
	Well					
	1	2	3	4	5	6
Length of spec.	105	355	355	425	350	405
PCR product						
Length of int.	800	1070	1070	1070	1070	1070
pos. control ¹						
5'-primer(s) ²	540	693	693	272	756	693
	5'-Cgg 3'	5'-ggT 3'	5'-ggT 3'	5'-CTA 3'	5'-ACT 3'	5'-ggT 3'
3'-primer(s) ³	605	916	916	412	970	962
	5'-gCT 3'	5'-gAT 3'	5'-gAC 3'	5'-gTT 3'	5'-TCT 3'	5'-CAA 3'
Well No.	1	2	3	4	5	6
HLA-B allele ⁴						
*07:02:01-07:02:05, 07:02:07-07:02:25, 07:02:27-07:03, 07:07-07:18:02, 07:20-07:24, 07:27-07:32, 07:35-07:39, 07:41-07:42, 07:44-07:47, 07:49N-07:52, 07:54, 07:56:01-07:59, 07:62-07:65, 07:67N-07:79, 07:81-07:89, 07:91-07:96:02, 07:98-07:99, 07:101-07:104, 07:106-07:111N, 07:113-07:121, 07:123-07:130, 07:132-07:136, 07:141-07:145, 07:148-07:160, 07:162-07:163, 07:166-07:175, 07:177-07:181N, 08:79, 40:98, 40:128, 48:05, 48:08	1		3			
*07:02:06, 35:66, 44:150, C*04:77	1					
*07:02:26, 07:04, 07:19, 07:25-07:26, 07:33, 07:43, 07:48, 07:55, 07:60, 07:66, 07:122, 07:131, 07:139, 07:146-07:147, 07:164-07:165, 27:36, 40:01:01-40:01:29, 40:07, 40:10:01-40:10:02, 40:14:01-40:14:03, 40:22N, 40:49, 40:51-40:52, 40:54-40:55, 40:61, 40:63, 40:66-40:67, 40:72:01-40:73, 40:77, 40:79, 40:81, 40:87:02-40:88, 40:124:01, 40:138-40:141, 40:146-40:147, 40:150-40:155N, 40:158, 40:172, 40:175, 40:179, 40:186, 40:193, 40:204, 40:207-40:208, 40:210, 40:212-40:213, 40:215-40:216N, 40:221-40:223, 44:31, 46:06, 48:01:01-48:01:03, 48:03:01-48:04, 48:06-48:07, 48:09-48:12, 48:14-48:28, 81:05			3			
*07:05:01-07:05:06	1	2		4		
*07:06, 07:90, 07:97, 07:105, 07:112, 07:137-07:138, 07:140, 07:176	1		3	4		
*07:34, 07:40, 07:53, 07:80, 81:01-81:02, 81:04N, 81:06			3	4		
*07:61	1		3		5	
*07:161N	1		3			6
*35:46		2				
*37:07	1	2				
Well No.	1	2	3	4	5	6
Length of spec.	105	355	355	425	350	405
PCR product						
Well No.	1	2	3	4	5	6

*40:09, 40:11:01-40:13, 40:18, 40:20-40:21, 40:24-40:28, 40:30-40:31, 40:33-40:38, 40:42-40:48, 40:59-40:60, 40:62, 40:64-40:65, 40:69-40:71, 40:74-40:76, 40:78, 40:80, 40:83-40:84, 40:86-40:87:01, 40:90, 40:92-40:93, 40:95-40:96, 40:99-40:121, 40:123, 40:124:02-40:127, 40:129-40:137, 40:145, 40:148-40:149, 40:156-40:157, 40:160:01-40:160:02, 40:162-40:163, 40:165, 40:168-40:171, 40:173, 40:177-40:178, 40:180-40:185, 40:187-40:192, 40:194-40:197, 40:199-40:203, 40:206, 40:217-40:219, 40:224-40:228, 40:231-40:233			?			
*40:15-40:16, 40:23, 40:32, 40:198	1		?			
*40:166			?	4		
*42:01:01-42:02, 42:04-42:05:02, 42:07-42:08, 42:10, 42:12-42:15, 42:17-42:18, 54:04, 54:11, 55:04, 55:08, 55:17, 55:20, 55:28, 55:49, 55:51, 56:05:01-56:06, 56:12, 56:15, 56:21, 81:03				4		
HLA-B allele						
Well No.	1	2	3	4	5	6