

Well No.	1	2	3	4	5	6	7
HLA-B allele ^{1,2}							
*07:02:01-07:02:05, 07:02:07-07:02:25, 07:02:27-07:03, 07:07-07:18:02, 07:20-07:24, 07:27-07:32, 07:35-07:39, 07:41-07:42, 07:44N-07:47, 07:49N-07:52, 07:54, 07:56:01-07:59, 07:62-07:65, 07:67N-07:79, 07:81-07:89, 07:91-07:96:02, 07:98-07:99, 07:101-07:104, 07:106-07:111N, 07:113-07:121, 07:123-07:130, 07:132-07:136:02, 07:141-07:145, 07:148-07:160, 07:162-07:163, 07:166-07:175, 07:177-07:181N, 07:183-07:200, 07:202-07:205, 07:207-07:208, 07:211-07:212, 07:214-07:221, 07:224, 07:227-07:234, 07:236-07:248, 07:250, 07:252-07:255, 07:261, 07:266-07:267, 08:79, 40:98, 40:128, 48:05, 48:08	1		3				
*07:02:06, 07:256-07:257, 07:259-07:260, 07:263, 07:265, 07:268, 07:272N-07:275, 08:156, 35:66, 44:150, C*04:77	1						
*07:02:26, 07:04, 07:19, 07:25-07:26, 07:33:01-07:33:03, 07:43, 07:48, 07:55, 07:60, 07:66, 07:122, 07:131, 07:139, 07:146-07:147, 07:164-07:165, 07:223, 07:226, 07:235, 07:251N, 27:36, 27:125, 40:01:01-40:01:36, 40:01:38-40:01:47, 40:07, 40:10:01:01-40:10:02, 40:14:01-40:14:03, 40:22N, 40:49, 40:51-40:52, 40:54-40:55, 40:61, 40:63, 40:66-40:67, 40:72:01-40:73, 40:77, 40:79, 40:81, 40:87:02-40:88, 40:124:01, 40:138-40:140, 40:146-40:147, 40:150, 40:155:01N-40:155:02N, 40:158, 40:171-40:172, 40:175, 40:179, 40:186:01-40:186:02, 40:193, 40:237, 40:249-40:253, 40:258-40:265N, 40:272, 40:277-40:278, 40:285-40:286N, 40:301, 40:316, 40:321, 40:323, 40:329, 40:332-40:333, 40:338N, 44:31, 46:06, 48:01:01-48:01:03, 48:01:05-48:01:06, 48:03:01-48:04:01, 48:06-48:07, 48:09-48:12, 48:14-48:28, 48:34, 48:37, 48:40, 81:05			3				
*07:05:01:01-07:05:08, 07:264	1	2		4			
*07:06:01-07:06:02, 07:90, 07:97, 07:105, 07:112, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 07:206, 07:210, 07:213, 07:222, 07:249	1		3	4			
*07:34, 07:40, 07:53, 07:80, 07:209, 81:01-81:02, 81:04N, 81:06			3	4			
*07:61	1		3		5		
*07:161N	1		3			6	
*07:225, 37:07	1	2					
*07:258, 07:262, 07:269-07:270	1			4			
*08:123, 08:132, 08:146, 42:01:01-42:02:01:02, 42:04-42:05:02, 42:07-42:08, 42:10, 42:12-42:15, 42:17-42:19, 42:21-42:23, 54:04, 54:11, 55:04, 55:08, 55:17, 55:20, 55:28, 55:49, 55:51, 56:05:01-56:06, 56:12, 56:15, 56:21, 81:03, 81:07-81:08				4			
*35:46, C*03:04:36		2					
*40:09, 40:11:01-40:13, 40:18, 40:20-40:21, 40:24-40:28, 40:30-40:31, 40:33-40:38, 40:42-40:48, 40:59-40:60, 40:62, 40:64-40:65, 40:69-40:70:01, 40:71, 40:74-40:76, 40:78, 40:80, 40:83-40:84, 40:86-40:87:01, 40:90, 40:92-40:93, 40:95-40:96, 40:99-40:121, 40:123, 40:124:02-40:127, 40:129-40:130:01, 40:131-40:137, 40:141, 40:145, 40:148-40:149, 40:151-40:154, 40:156-40:157, 40:160:01-40:160:02, 40:162-40:163, 40:165, 40:168-40:170, 40:173, 40:177-40:178, 40:180-40:185, 40:187-40:192, 40:194-40:197, 40:199-40:204, 40:206-40:208, 40:210, 40:212-40:213, 40:215-40:219, 40:221-40:228, 40:231-40:236, 40:238-40:240, 40:242, 40:244-40:248, 40:256N-40:257, 40:273-40:276, 40:279-40:283, 40:287-40:300			?				
*40:15-40:16, 40:23, 40:32, 40:198	1		?				
*40:166			?	4			
Well No.	1	2	3	4	5	6	7

Negative Control