

Well No.	1	2	3	4	5	6	7
HLA-B allele ^{1,2}							
*07:02:01:01-07:02:05, 07:02:07-07:02:25, 07:02:27-07:03, 07:07:01-07:18:02, 07:20-07:24, 07:27-07:31, 07:35-07:39, 07:41-07:42, 07:44N-07:47, 07:49N-07:52, 07:54, 07:56:01-07:59, 07:62-07:65, 07:67N-07:68:03, 07:70-07:77, 07:79, 07:81-07:89, 07:91-07:96:02, 07:98-07:99, 07:101-07:104, 07:106-07:111N, 07:113-07:121, 07:124-07:130, 07:132-07:136:02, 07:141-07:145, 07:148-07:160, 07:162-07:163, 07:166-07:175, 07:177-07:181N, 07:183-07:200, 07:202-07:205, 07:208, 07:211-07:212, 07:215-07:221, 07:224, 07:227-07:234, 07:236-07:248, 07:250, 07:252-07:257, 07:259-07:261, 07:263, 07:265-07:268, 07:271-07:277, 07:279-07:282, 07:284-07:286, 07:288-07:292, 07:294-07:303:02, 07:305-07:316N, 07:318N-07:323, 07:325N-07:326, 07:328-07:331, 07:334-07:339, 07:341-07:342, 07:344-07:348, 07:350-07:351N, 07:354, 07:360Q, 07:362, 07:364-07:367, 07:369-07:377, 07:379-07:385, 07:390-07:394, 07:396-07:398, 07:400-07:406, 40:32, 40:98, 48:08	1		3				
*07:02:06, 08:156, 35:66, 44:150, C*04:77	1						
*07:02:26, 07:04:01-07:04:02, 07:19, 07:25-07:26, 07:33:01-07:34, 07:43, 07:48, 07:55, 07:60, 07:66, 07:80, 07:122, 07:131, 07:139:01-07:139:02, 07:146-07:147, 07:164-07:165, 07:209, 07:223, 07:226, 07:235, 07:251N, 07:343N, 07:349, 07:361, 07:363, 07:386N, 07:388, 07:407, 27:36, 27:125, 40:01:01-40:01:36, 40:01:38-40:01:63, 40:01:65-40:01:70, 40:07, 40:10:01-01-40:10:02, 40:12:01-40:12:02, 40:14:01-40:14:03, 40:22N, 40:30-40:31, 40:42-40:43, 40:48-40:49, 40:51-40:52, 40:54-40:55, 40:61, 40:63, 40:66-40:67, 40:72:01-40:74, 40:77, 40:79-40:81, 40:87:02-40:88, 40:113-40:114:01:02, 40:117, 40:123-40:124:01, 40:125:02-40:126, 40:130:02, 40:138-40:141, 40:146-40:147, 40:150, 40:155:01N-40:155:02N, 40:160:01, 40:170-40:172, 40:175, 40:178-40:179, 40:182-40:183, 40:186:01-40:186:02, 40:193, 40:204, 40:237, 40:245, 40:249-40:253, 40:258-40:265N, 40:272-40:273, 40:277-40:278, 40:285-40:286N, 40:298:02, 40:301, 40:310, 40:316, 40:321, 40:323:01-40:324, 40:329, 40:332-40:333, 40:338N, 40:344, 40:346, 40:353:01-40:353:02, 40:355, 40:357, 40:365, 40:367-40:368, 40:370, 40:372N, 40:377, 40:379-40:383, 40:385-40:386, 40:391-40:395, 40:397, 40:401-40:404, 40:406-40:407, 40:416-40:419, 40:421Q-40:422, 40:424, 40:426N, 40:428N, 40:430-40:431, 40:433, 40:436, 40:439, 40:443, 40:446-40:448, 40:450, 40:452, 40:456, 40:461-40:462, 44:31, 46:06, 48:01:01-01-48:01:03, 48:01:05-48:01:10, 48:03:01-01-48:04:01:02, 48:06-48:07, 48:09-48:12, 48:14-48:28, 48:34, 48:37, 48:39-48:50, 81:01:01-01-81:02, 81:04N-81:06, 81:09			3				
*07:05:01:01-07:05:10, 07:69, 07:264, 07:333, 07:352-07:353, 07:356-07:357, 07:368, 07:378, 07:387, 07:399, 48:05	1	2		4			
*07:06:01:01-07:06:05, 07:32, 07:78, 07:90, 07:97, 07:105, 07:112, 07:123, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 07:206-07:207, 07:210, 07:213-07:214, 07:222, 07:249, 07:258, 07:262, 07:269-07:270, 07:278, 07:283, 07:287, 07:293, 07:304, 07:317, 07:324, 07:327, 07:332, 07:340, 07:358, 07:389, 07:395, 08:79, 40:16:01-01-40:16:01:02, 40:23, 40:128	1		3	4			
*07:40, 07:53			3	4			
*07:61	1		3		5		
*07:161N	1		3			6	
*07:225, 07:355	1	2					
*08:20:01-08:20:02, 08:40, 13:18, 13:73, 13:128, 35:18, 39:08, 39:11, 39:33, 40:161, 41:62, 51:05, 51:29, 51:54, 51:82, 51:217, 51:279, 52:24, 54:10, 54:20, 55:09, 55:37, 55:52, 56:51, 56:63, 57:02:01-01-57:02:02, 57:12, 57:28N, 57:42, 57:63, C*01:90, C*01:116, C*05:238, C*14:92				4			
*35:01:65, 35:46, 51:01:84, 81:10, C*03:04:36		2					
*35:172				w			
*37:07	1	2	3				

Negative Control

HLA-B07 Add-on 2020 7L6

*40:06:16, 40:18, 40:21, 40:24-40:25, 40:27:02, 40:28, 40:33-40:36, 40:38, 40:44-40:47, 40:59-40:60, 40:62, 40:65, 40:69-40:70:01, 40:71, 40:75-40:76, 40:78, 40:83-40:84, 40:87:01, 40:92-40:93, 40:95-40:96, 40:100-40:109, 40:112, 40:114:02-40:116, 40:118N, 40:120-40:121, 40:124:02-40:125:01, 40:127, 40:130:01, 40:131-40:137, 40:145, 40:148-40:149, 40:151-40:154, 40:156-40:158, 40:160:02, 40:162-40:163, 40:165-40:166, 40:168-40:169, 40:173, 40:177, 40:180-40:181, 40:184-40:185, 40:187-40:192, 40:194-40:197, 40:199-40:201, 40:203, 40:206-40:208, 40:210, 40:212-40:213, 40:215-40:218, 40:221-40:224, 40:226-40:228, 40:231-40:236, 40:238-40:240, 40:242, 40:244, 40:246-40:248, 40:256N-40:257, 40:274-40:276, 40:279-40:283, 40:287-40:289, 40:291N-40:295, 40:297-40:298:01, 40:299-40:300, 40:306, 40:308, 40:311-40:313, 40:315, 40:318-40:319, 40:325-40:328, 40:330, 40:335-40:337N, 40:339-40:341, 40:348-40:352, 40:354, 40:358, 40:360, 40:363-40:364, 40:366, 40:374, 40:376, 40:388, 40:396, 40:398-40:400							
*40:15, 40:198	1		?	4			
Well No.	1	2	3	4	5	6	7