

HLA-B13 2020 7K5

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
HLA-B allele ^{1,2,3}																															
*13:01:01:01-13:01:10, 13:01:12-13:01:16, 13:51, 13:60, 13:73, 13:77-13:80, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109-13:110, 13:122, 13:129, 13:135, 13:138	1	2	3																												
*13:01:11	1	2	3						10																						
*13:02:01:01-13:02:25, 13:02:27-13:02:32, 13:54, 13:65-13:66, 13:68, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:99-13:102, 13:104-13:105, 13:111-13:119, 13:121, 13:123Q-13:126, 13:130, 13:132-13:134, 13:136-13:137N, 13:139N	1	2		4																											
*13:02:26, 13:14 ⁴	1	2		4											15																
*13:03, 13:120	1				5																										
*13:04	1					6				11																					
*13:06, 13:53	1						7																								
*13:07N	1	2	3																											30	
*13:08	1	2		4					9																						
*13:09, 13:67		2		4						10																					
*13:10	1	2									11																				
*13:11	1	2										12																			
*13:12:01	1	2	3										13																		
*13:12:02		2	3										13																		
*13:13:01-13:13:02	1	2	3											14																	
*13:15, 13:33 ⁴	1	2		4												16															
*13:16	1	2		4	5																			24							
*13:17	1	2	3													16									24						
*13:18	1	2		4				8																							
*13:19	1	2		4									13																		
*13:20	1	2	3		5																				24						
*13:21	1	2	3							10				14												24					
*13:22:01	1		3		5										15																
*13:22:02	1		3		5																										
*13:23	1	2	3																											29	
*13:25	1	2	3																				22								
*13:26:01-13:26:02	1	2	3								11																				
*13:27, 13:49N ⁴	1	2		4													17														
*13:28	1	2	3															18													
*13:29	1	2	3																		21										
*13:30	1	2		4																	21										
*13:31	1	2		4	5			8																24							
*13:32	1	2		4														18													
*13:34, 13:38 ⁴	1	2		4															19												
*13:35, 13:59	1	2		4										14																	
*13:36	1	2	3																		20										
*13:37	1	2		4																				23							
*13:39		2	3							10																					
*13:40	1	2		4																											
*13:41	1	2		4	5			8														21	22								
*13:42	1	2		4					9															23							
*13:43	1	2	3															17													
*13:44	1	2		4														18						23							
*13:45	1			4	5																										
*13:46	1					6																									
*13:47	1	2		4																						26					
*13:48	1				5																			24							
*13:50	1	2	3																							26					
*13:52	1	2	3																19									27			
*13:55	1	2		4																										29	
*13:56:01N-13:56:02N	1	2		4		6																									
*13:57	1	2	3										13		16																
*13:58, 13:97-13:98		2		4																											
*13:61	1	2	3																									27			
*13:62	1																							24							
*13:63N	1	2	3																										28		
*13:64	1	2																							25						
*13:69	1	2		4																								27			
*13:70	1	2		4																											
*13:71	1	2								10				14												25					
*13:72, 13:127	1	2		4		6					11																				
*13:76N	1	2	3																19												
*13:83	1	2	3												15																
*13:86	1																						22								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
*13:96	1	2		4																19								27			

Negative Control

*13:103N	1	2	4																	21				26							
*13:108	1	2											14																		
*13:128	1		4	5			8							15									24								
*13:131	1			5		7	8																24								
*07:02:01:01-07:04:02, 07:06:02, 07:07:01-07:08:02, 07:10, 07:13, 07:15-07:16, 07:19-07:24, 07:26-07:31, 07:33:01-07:33:03, 07:35-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:58:03, 07:70-07:77, 07:79, 07:81-07:83, 07:85:01-07:89, 07:91-07:96:02, 07:98-07:104, 07:106-07:111N, 07:113-07:122, 07:124-07:136:02, 07:139:01-07:139:02, 07:141-07:148, 07:150-07:154, 07:156-07:161N, 07:165-07:175, 07:178-07:181N, 07:183-07:198, 07:200, 07:203-07:205, 07:208, 07:211-07:212, 07:214-07:219, 07:221, 07:224, 07:228:01-07:234, 07:236, 07:238-07:247, 07:250-07:254, 07:256-07:257, 07:259-07:260, 07:262-07:263, 07:265-07:268, 07:271-07:277, 07:279-07:282, 07:284-07:286, 07:288-07:292, 07:294-07:300, 07:302-07:303:02, 07:305-07:316N, 07:318N-07:323, 07:325N-07:326, 07:328-07:331, 07:334-07:339, 07:341-07:344, 07:347-07:348, 07:351N, 07:354-07:355, 07:360Q-07:362, 07:364-07:367, 07:369-07:377, 07:379-07:385, 07:388, 07:390-07:394, 08:156, 14:01:01:01-14:12, 14:14-14:36, 14:38-14:40, 14:42, 14:44-14:47, 14:49-14:52, 14:54-14:55, 14:57-14:58, 14:60-14:86, 15:02:12, 15:09:01-15:10:05, 15:17:01:01-15:17:04, 15:17:06-15:17:07, 15:30:01:01-15:30:01:03, 15:37, 15:45, 15:58, 15:63, 15:77, 15:90, 15:99, 15:133, 15:168, 15:177, 15:196, 15:208, 15:216, 15:229-15:230, 15:233, 15:243, 15:248, 15:253, 15:269, 15:273, 15:287, 15:313, 15:323-15:324, 15:329, 15:334, 15:338-15:339, 15:342, 15:356, 15:361, 15:400N, 15:403, 15:416, 15:420, 15:423-15:424, 15:427, 15:435, 15:439, 15:445-15:446, 15:455, 15:460, 15:462, 15:488, 15:500, 15:503, 15:506, 15:516, 15:523, 15:530, 15:532, 15:541, 15:546Q, 15:553, 15:555, 27:04:01-27:04:06, 27:06:01:01-27:06:01:02, 27:10, 27:15, 27:21, 27:25, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105-27:106, 27:108-27:109, 27:112-27:115, 27:120, 27:138, 27:149, 27:173, 27:183, 27:192, 27:208, 27:210-27:211, 27:214, 35:44, 35:96, 35:167, 35:213, 35:384, 35:413, 37:07, 37:13, 44:76, 44:79:01-44:79:02, 44:146, 44:452, 44:459, 46:61, 46:70, 46:74, 46:81, 47:10, 48:08, 49:01:01:01-49:01:13, 49:01:15-49:01:16, 49:03, 49:06, 49:08, 49:10, 49:12-49:14, 49:16-49:33, 49:35-49:40, 49:42-49:44, 49:46-49:49, 49:51-49:54, 49:56-49:65, 51:01:01:01-51:01:08, 51:01:10-51:01:20, 51:01:22-51:01:24, 51:01:26-51:04:02, 51:06:01-51:07:02, 51:11N-51:12, 51:14, 51:17-51:18, 51:22-51:24:01, 51:24:05, 51:26-51:28, 51:30, 51:32-51:33, 51:35, 51:39, 51:41N, 51:43, 51:46, 51:48-51:52, 51:55-51:60, 51:64-51:72, 51:74-51:80, 51:83-51:91, 51:94-51:96, 51:98N-51:100, 51:102-51:105, 51:107, 51:109-51:114, 51:116-51:119, 51:121, 51:123-51:138, 51:140, 51:142, 51:144-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:180, 51:182-51:193, 51:195-51:196, 51:198-51:215, 51:218-51:219, 51:221-51:222, 51:224, 51:226-51:227, 51:229-51:250, 51:252-51:258, 51:260-51:262, 51:264N-51:273N, 51:277-51:278, 51:281-51:286, 51:288-51:294, 51:296-51:299, 51:301-51:307, 52:01:01:01-52:01:32, 52:01:34-52:02:02, 52:04-52:07, 52:09:01-52:09:02, 52:11-52:13, 52:15-52:18, 52:20, 52:22-52:23, 52:26-52:27, 52:30-52:36, 52:38, 52:40-52:53, 52:55-52:66, 52:68-52:70, 52:72-52:73, 52:76-52:83, 52:85-52:94N, 56:05:01-56:06, 56:21, 57:09, 57:57, 57:63, 58:08:01-58:08:02, 78:01:01:01-78:03, 78:05-78:06, 78:09																															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31

Negative Control

[illegible]

[illegible]

[illegible]

Negative Control

	☐	☐	☐	☐
Negative Control				

HLA-B13 2020 7K5

*56:20:02					5				10			15																			
*56:25, 56:37					5													24													
*56:34					5								16																		
*56:36					5				10																						
*56:60		2	3																												
C*01:90					5		7		10																						
C*02:159, C*02:161															17																
C*14:92							7	8																							
C*15:128					5		7		10				15																		
C*17:08																	19														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31