

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
HLA-B allele ^{1,2,3}																																
*13:01:01-13:01:10, 13:01:12-13:01:18, 13:51, 13:60, 13:73, 13:77-13:80, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109-13:110, 13:122, 13:129, 13:135, 13:138, 13:142, 13:144, 13:148	1	2	3																													
*13:01:11	1	2	3							10																						
*13:02:01-13:02:25, 13:02:27-13:02:32, 13:54, 13:65-13:66, 13:68, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:99-13:102, 13:104-13:105, 13:111-13:119, 13:121, 13:123Q-13:126, 13:130, 13:132-13:134, 13:136-13:137N, 13:139N, 13:143, 13:147, 13:149-13:152, 13:154-13:155	1	2		4																												
*13:02:26, 13:14 ⁴	1	2		4											15																	
*13:03, 13:120, 13:146	1				5																											
*13:04	1					6					11																					
*13:06, 13:53	1						7																									
*13:07N	1	2	3																												30	
*13:08	1	2		4						9																						
*13:09, 13:67	1	2		4							10																					
*13:10	1	2										11																				
*13:11	1	2											12																			
*13:12:01	1	2	3											13																		
*13:12:02		2	3											13																		
*13:13:01-13:13:02	1	2	3												14																	
*13:15, 13:33 ⁴	1	2		4												16																
*13:16	1	2		4	5																			24								
*13:17	1	2	3													16																
*13:18	1	2		4				8																								
*13:19	1	2		4										13																		
*13:20	1	2	3		5																			24								
*13:21	1	2	3							10				14																		
*13:22:01	1		3		5										15																	
*13:22:02	1		3		5																											
*13:23	1	2	3																												29	
*13:25	1	2	3																			22										
*13:26:01-13:26:02	1	2	3								11																					
*13:27, 13:49N ⁴	1	2		4													17															
*13:28	1	2	3															18														
*13:29	1	2	3																			21										
*13:30	1	2		4																		21										
*13:31	1	2		4	5			8																24								
*13:32	1	2		4															18													
*13:34, 13:38 ⁴	1	2		4																19												
*13:35, 13:59	1	2		4										14																		
*13:36	1	2	3																		20											
*13:37	1	2		4																				23								
*13:39		2	3							10																						
*13:40	1	2		4																		21	22									
*13:41	1	2		4	5			8																								
*13:42	1	2		4						9														23								
*13:43	1	2	3																													
*13:44	1	2		4														17														
*13:45, 13:153	1			4	5																											
*13:46	1					6																										
*13:47	1	2		4																												
*13:48	1				5																											
*13:50	1	2	3																													
*13:52	1	2	3																	19												
*13:55	1	2		4																												29
*13:56:01N-13:56:02N, 13:140	1	2		4		6																										
*13:57	1	2	3											13		16																
*13:58, 13:97-13:98		2		4																												
*13:61	1	2	3																											27		
*13:62	1																															
*13:63N	1	2	3																					24						28		
*13:64	1	2																														
*13:69	1	2		4																												
*13:70	1	2		4																												
*13:71	1	2								10				14																		
*1																																

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]