

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
HLA-B allele ^{1,2,3}																						
*14:01:01-14:01:06, 14:26, 14:40, 14:47, 14:49, 14:54	1	2						8			11											
*14:02:01:01-14:02:15, 14:24-14:25, 14:30-14:31, 14:35-14:36, 14:38-14:39, 14:43, 14:45, 14:48, 14:50-14:52	1		3								11											
*14:03	1		3	4							11											
*14:04	1		w		5						11											
*14:05			3			6					11											
*14:06:01-14:06:02			3				7				11											
*14:07N	1	2			5			8			11											
*14:08:01-14:08:02, 39:43 ⁴		2					7	8			11											
*14:09	1		3						9		11											
*14:10		2						8			11				15							
*14:11	1		3							10	11											
*14:12	1	2						8			11	12										
*14:13			3								11		13									
*14:14	1	2						8			11			14								
*14:15	1		3								11				15							
*14:16, 14:23 ⁵	1		3								11					16						
*14:17	1		3			6					11											
*14:18	1		3								11			14								
*14:19	1	2						8				12										
*14:20			3	4							11											
*14:21, 14:42	1										11											
*14:22	1		3								11						17					
*14:27	1		3									12										
*14:28	1									10	11											
*14:29	1		3			6				10	11											
*14:32	1	2						8	9		11					16						
*14:33	1		3								11									20		
*14:34	1		3								11							18				
*14:37			3								11						17					
*14:41N	1		3								11								19			
*14:44	1		3																			
*14:46		2						8													21	
*14:53		2						8			11		13									
*14:55		2					7	8			11						17					
*07:02:01:01-07:02:14, 07:02:17-07:02:31, 07:02:33-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44N-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 07:229-07:235, 07:237-07:238, 07:240-07:243, 07:245, 07:247-07:272N, 07:274-07:290, 15:71, 15:175, 15:381, 27:08, 27:12, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 35:03:21, 35:50, 35:84, 35:197, 35:231, 35:269, 40:02:01:01-40:02:14, 40:02:16-40:06:14, 40:06:16, 40:08-40:09, 40:11:01-40:11:02, 40:18, 40:20, 40:24, 40:26-40:27:02, 40:29, 40:35:01-40:35:02, 40:37, 40:40, 40:44, 40:50, 40:56, 40:64, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:297, 40:302-40:307, 40:309, 40:311, 40:313, 40:317, 40:320, 40:322, 40:330-40:331, 40:334, 40:337N, 40:341, 40:343, 40:345N, 40:347-40:351, 44:131, 48:02:02, 50:43, 56:35, 73:01-73:02, 81:01-81:07, C*03:137, C*03:182, C*07:351, C*12:82								8														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

*39:05:01-39:05:02, 39:07-39:08, 39:11, 39:55-39:56, 39:113, 39:118	2							8			11		14										
*39:32	2						7	8					13										
*39:37	2					6		8			11		14										
*39:40:01N-39:40:02N	2				5			8			11												
*39:48, 67:06	2						7	8			11		13										
*39:99								w			11												
*40:28								8					13		15		17						
*40:51, 40:163							7				11		13										
*40:113, 51:06:01-51:06:03, 51:62, 51:194							7						13										
*40:243								8														21	
*42:06, 42:20						6		8			11												
*44:190, 55:77											11				15								
*44:203, 57:14:01-57:14:02															15								
*46:48																							
*48:07							6	8									18						
*49:09, A*33:40													14										
*50:51, 57:11, 58:02:01-58:02:02, 58:06-58:07, 58:25, 58:38, 58:43, 58:46, 58:60, C*01:35, C*01:107, C*03:17:01-03:17:02, C*03:71, C*03:249, C*03:271, C*03:302, C*06:96, 06:197, C*12:122, C*14:20												12											
*51:22, 78:03			3												13								
*51:29, 51:82, 51:217, 52:24				4											13								
*51:56:01-51:56:03															13		15		17				
*51:116, 51:176, 51:179, 51:218, 52:16, 52:27															13	14							
C*01:10, C*02:05:01-02:05:03, 02:17, C*06:08, C*12:119, C*14:25, C*16:29, 16:50, C*17:21																						21	
C*01:30, C*14:76	1			4															17				
C*01:101, 01:113, C*15:55																		16					
C*05:16, C*05:85, C*05:107, C*06:129, C*07:364, C*08:12									9									16					
C*07:516, C*07:521								7												17			
C*08:08:01-08:08:02						6																	
C*08:51, C*08:114				4				7												17			
C*12:87	1																			17			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	