

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
HLA-B allele ^{1,2,3}																						
*14:01:01:01-14:01:09, 14:26, 14:40, 14:47, 14:49, 14:54, 14:65, 14:71, 14:77-14:78N, 14:80-14:82	1	2						8			11											
*14:02:01:01-14:02:16, 14:02:18-14:02:23, 14:24-14:25, 14:30-14:31, 14:35-14:36, 14:38-14:39, 14:43, 14:45, 14:48, 14:50-14:52, 14:58-14:61, 14:63-14:64, 14:66-14:70Q, 14:72N, 14:74-14:76N, 14:79N	1		3								11											
*14:02:17, 14:33	1		3								11									20		
*14:03	1		3	4							11											
*14:04	1		w		5						11											
*14:05			3			6					11											
*14:06:01-14:06:02, 14:56			3				7				11											
*14:07N	1	2			5			8			11											
*14:08:01-14:08:02, 39:43 ⁴		2					7	8			11											
*14:09	1		3						9		11											
*14:10		2						8			11			15								
*14:11	1		3							10	11											
*14:12	1	2						8			11	12										
*14:13			3								11		13									
*14:14	1	2						8			11			14								
*14:15	1		3								11				15							
*14:16, 14:23 ⁵	1		3								11					16						
*14:17	1		3			6					11											
*14:18	1		3								11			14								
*14:19	1	2						8			12											
*14:20			3	4							11											
*14:21, 14:42, 14:57	1										11											
*14:22	1		3								11						17					
*14:27	1		3								12											
*14:28	1									10	11											
*14:29	1		3			6				10	11											
*14:32	1	2						8	9		11				16							
*14:34	1		3								11						18					
*14:37			3								11						17					
*14:41N	1		3								11							19				
*14:44	1		3																			
*14:46		2						8													21	
*14:53		2						8			11		13									
*14:55		2					7	8			11						17					
*14:62, 15:44, 39:129, 59:08 ⁴			3								11											
*14:73	1										11			14								
*07:02:01:01-07:02:14, 07:02:17-07:02:31, 07:02:33-07:02:77, 07:02:79-07:05:04, 07:05:06-07:15, 07:17-07:18:02, 07:20-07:26, 07:28, 07:30, 07:32-07:33:03, 07:35, 07:39-07:42, 07:44N-07:49N, 07:51-07:64, 07:66-07:69, 07:73-07:74, 07:76-07:78, 07:80, 07:82-07:85:01, 07:87-07:90, 07:92-07:110, 07:112-07:114, 07:116-07:148, 07:150-07:178, 07:181N-07:183, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:212-07:217, 07:221-07:227, 07:229-07:235, 07:237-07:238, 07:240-07:243, 07:245, 07:247-07:272N, 07:274-07:296, 07:298-07:305, 07:307-07:312, 07:314-07:326, 07:328-07:351N, 07:353-07:382, 07:384-07:385, 15:71, 15:175, 15:381, 27:08, 27:12:01:01-27:12:01:02, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 27:204, 35:03:21, 35:08:12, 35:50, 35:84, 35:197, 35:231, 35:269, 35:411, 35:423, 40:02:01:01-40:02:14, 40:02:16-40:06:14, 40:06:16-40:06:20, 40:06:22, 40:08-40:09, 40:11:01-40:11:02, 40:18, 40:20:01:01-40:20:01:02, 40:24, 40:26-40:27:02, 40:29, 40:35:01-40:35:02, 40:37, 40:40, 40:44, 40:50, 40:56, 40:64:01:01-40:64:02, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:115, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:244, 40:248, 40:254-40:256N, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:290-40:291N, 40:293-40:297, 40:302-40:307, 40:309, 40:311, 40:313, 40:317, 40:320, 40:322, 40:330-40:331, 40:334, 40:337N, 40:341, 40:343, 40:345N, 40:347-40:351, 40:354, 40:356, 40:359, 40:361N-40:364, 40:366, 40:369, 40:371, 40:373-40:376, 40:378, 40:384, 40:387-40:390, 40:399N, 40:405, 40:408-40:415, 40:425, 40:427, 40:429, 40:434-40:435, 40:437-40:438N, 40:440-40:442, 44:131, 48:02:02, 48:48, 50:43, 56:35, 73:01-73:02, 81:01:01-81:07, C*03:03:51, C*03:137, C*03:182, C*03:459, C*07:351, C*08:02:22, C*12:82, C*16:116								8														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*07:02:15, 07:19, 07:31, 07:34, 07:43, 08:94, 08:226, 37:37, 39:01:09, 39:01:26, 39:02:02:01-39:02:03, 39:06:03, 39:10:01-39:10:02, 39:16-39:17, 39:19:01, 39:23, 39:39:02, 39:45, 39:78, 39:83, 39:89, 39:92, 39:96, 39:101, 39:105, 39:110, 39:126, 39:131, 39:133N, 39:143, 39:145, 40:39, 40:184, 42:01:01-42:02:01:02, 42:04-42:05:02, 42:07-42:10, 42:12-42:19, 42:21, 42:23-42:27, 54:02, 55:10, 55:82, 67:01:02-67:02:01:02, 67:05								8			11											
*07:02:32, 07:85:02, 39:33, 39:47, 39:50, 39:74, 39:80, 39:102		2						8														

Negative Control

Negative Control

[illegible]

*51:01:68											10			13										
*51:22, 78:03			3											13										
*51:29, 51:82, 51:217, 51:279, 52:24				4										13										
*51:56:01-51:56:03														13		15		17						
*51:116, 51:176, 51:179, 51:218, 52:16, 52:27, 52:77														13	14									
C*01:10, C*02:05:01-02:05:03, C*02:17, C*06:08, C*12:119, C*14:25, C*16:29, C*16:50, C*17:21																							21	
C*01:30, C*14:76	1			4														17						
C*01:101, C*01:113, C*15:55																	16							
C*02:142, C*07:516, C*07:521:01-07:521:02							7											17						
C*05:16, C*05:85, C*05:107, C*06:129, C*07:364, C*08:12									9								16							
C*05:216, C*08:51, C*08:114				4			7											17						
C*06:207, C*12:87	1																	17						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22		