

INTERPRETATION TABLE																																																		
HLA-B*15 high resolution SSP typing																																																		
Amplification patterns of the B*15010101 to B*9578 alleles																																																		
	Well																																																	
	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44						
Length of spec.	395	290	120	180	385	525	330	7	105	200	300	140	405	395	465	105	120	435	195	400	360	205	190	230	205	315	230	115	240	255	320	395	320	380	200	160	165	360	170	225	305	220	465	345	525	515	475	355	320	420
PCR product(s) ¹										520				435	515		190				505	360	425			300	385				355	395		255					380	425	460									
Length of int.	800	1070	1070	800	1070	1070	1070	1070	1070	800	1070	1070	800	1070	1070	1070	1070	1070	1070	800	1070	1070	800	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	
pos. Control ²																																																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44						
HLA-B allele ³	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8		
*15010101, 150103, 150106-150112	+	+	+																			+																			+			+	+	+	+	+		
*15010102N	+	+	+	+																		+																				+			+	+	+	+	+	
*150102	+	+	+														+					+																				+			+	+	+	+	+	
*150104	+		+																			+																				+			+	+	+	+	+	
*150201, 150202	+	+			+	+																																					+	+		+	+	+	+	+
*150301, 150302	+						+																+																				+	+		+	+	+	+	+
*150303	+	+					+																+																				+	+		+	+	+	+	+
*1504	+	+	+					+															+																			+		+	+	+	+	+	+	
*1505	+	+							+														+																				+	+		+	+	+	+	+
*1506	+	+								+													+																				+	+		+	+	+	+	+
*1507	+	+	+								+												+																				+	+		+	+	+	+	+
*1508		+	+		+							+											+																			+			+	+	+	+	+	
*1509				+		+							+	+	+	+							+																			+	+		+	+	+	+	+	
*1510				+		+		+					+	+									+																			+	+		+	+	+	+	+	
*151101, 151102		+	+		+			+															+																			+	+		+	+	+	+	+	
*151103		+	+		+																		+																				+	+		+	+	+	+	+
*151104			+		+																		+																				+	+		+	+	+	+	+
*1512	+	+	+														+						+																				+	+		+	+	+	+	+
*1513	+	+			+	+												+					+																			+	+		+	+	+	+	+	
*1514	+	+	+																+				+																				+	+		+	+	+	+	+
*1515	+	+	+		+																		+																				+	+		+	+	+	+	+
*1516							+										+		+	+	+	+																				+	+	+	+	+	+	+	+	
*15170101-151702		+																+		+			+																			+	+		+	+	+	+	+	
*151801				+		+							+										+																			+	+		+	+	+	+	+	
*151802				+		+							+										+																			+	+		+	+	+	+	+	
*151803				+		+							+										+																			+	+		+	+	+	+	+	
*1519	+	+	+													+							+	+																		+	+		+	+	+	+	+	
*1520	+	+																				+																					+	+		+	+	+	+	+
*1521		+			+	+							+										+																			+	+		+	+	+	+	+	
*1523				+		+		+					+					+					+																			+	+		+	+	+	+	+	
*1524	+	+	+															+					+																			+	+		+	+	+	+	+	
*152501, 152502	+	+			+																		+																				+	+		+	+	+	+	+
*1526N	+	+	+																				+			+																+	+		+	+	+	+	+	
*152701-152703	+	+	+							+													+																			+	+		+	+	+	+	+	
*1528	+	+	+																				+					+														+	+		+	+	+	+	+	
*1529				+		+							+										+																			+	+		+	+	+	+	+	
*1530	+	+	+												+								+																			+	+		+	+	+	+	+	
*1531	+	+		+				+															+																											

[illegible]

Well No.									1				1				1				1				2				2				2				2				3				3				3				3				4				4				4				4			
HLA-B allele ³	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4	4																								
*9562		+															+	+																						+	+	+	+	+	+	+	+																									
*9563		+	+	+																																					+		+	+	+	+	+	+																								
*9564		+	+	+																																						+	+	+	+	+	+	+																								
*9565		+	+	+			+																																			+	+	+	+	+	+	+																								
*9566		+	+	+																																						+	+	+	+	+	+	+																								
*9567		+	+	+																																						+	+	+	+	+	+	+																								
*9568			+														+	+																							+		+	+	+	+	+	+																								
*9569		+	+	+																																						+	+	+	+	+	+	+																								
*9570		+	+			+	+																																			+	+				+	+	+																							
*9571		+	+	+																																						+	+	+	+	+	+	+																								
*9572		+	+	+																																						+	+	+	+	+	+	+																								
*9573		+					+																																			+	+	+	+	+	+	+																								
*9574		+	+	+																																						+	+	+	+	+	+	+																								
*9575		+	+	+																																						+	+	+	+	+	+	+																								
*9576						+		+					+																													+	+	+	+	+	+	+																								
*9577			+															+	+																							+		+	+	+	+	+	+																							
*9578		+	+	+																																						+		+	+	+	+	+	+																							
*070201-070210, 070212-070217, 0703, 070501, 070502, 0706-0713, 0715, 0716, 071801, 071802, 0721-072202, 0727, 0730-0733, 0735-0742, 0744, 0746, 0747, 0749N-0751, 0753, 0754, 0756-0759, 0761-0764, 0767N, 0769-0771, 0773-0779, 0781-0785, 0787-0795, 0797-0799, 1318, 1821, 3707, 4015, 4016, 4023, 4032, 4098, 4805, 4808, 5182, Cw*1508																																																																								
*070211, 070503																+																													+																											
*0720, 0724, 0760, 350801-350804, 3518, 3561, 3580, 3599, 5105, 5129, 5154, 5311, 5316, 5822																																																																								
*0723																																																																								
*0729																																																																								
*0745, 0752, 3908, 3911																																																																								
*0748, 1310, 1311, 1810, 1811, 2707, 2711, 2733, 2734, 2743, 3531, 3930, 400101-400103, 400105, 400107-4003, 4007-4011, 4013-401403, 4018-4022N, 4024, 4025, 4027, 4029, 4031, 4033, 4036-4040, 4042, 4045-4050, 4054-4058, 4061, 4062, 4065-4067, 4069, 407201-4074, 4076, 4078, 4080-4082, 4084, 4085, 4088-4091, 4094, 4097, 4431, 4465, 480101, 480102, 480301, 4804, 4806, 4807, 4809-4811, 4813, 4816, 4818, 4820, 5181, 8101-8104N, Cw*0203																																																																								
*0765, 0772, 0786, 5410, 5509, 5537																																																																								
Well No.										1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4																						
HL A-B allele ³	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	3	3	3	3	3	3	3	3	3	4	4	4	4	4	4	4																							

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

																																																												Well No.
																																																												HLA-B allele ³
																																																												*9562
																																																												*9563
																																																												*9564
																																																												*9565
																																																												*9566
																																																												*9567
																																																												*9568
																																																												*9569
																																																												*9570
																																																												*9571
																																																												*9572
																																																												*9573
																																																												*9574
																																																												*9575
																																																												*9576
																																																												*9577
																																																												*9578
																																																												*070201-070210, 070212-070217, 0703, 070501, 070502, 0706-0713, 0715, 0716, 071801, 071802, 0721-072202, 0727, 0730-0733, 0735-0742, 0744, 0746, 0747, 0749N-0751, 0753, 0754, 0756-0759, 0761-0764, 0767N, 0769-0771, 0773-0779, 0781-0785, 0787-0795, 0797-0799, 1318, 1821, 3707, 4015, 4016, 4023, 4032, 4098, 4805, 4808, 5182, Cw*1508
																																																												*070211, 070503
																																																												*0720, 0724, 0760, 350801-350804, 3518, 3561, 3580, 3599, 5105, 5129, 5154, 5311, 5316, 5822
																																																												*0723
																																																												*0729
																																																												*0745, 0752, 3908, 3911
																																																												*0748, 1310, 1311, 1810, 1811, 2707, 2711, 2733, 2734, 2743, 3531, 3930, 400101-400103, 400105, 400107-4003, 4007-4011, 4013-401403, 4018-4022N, 4024, 4025, 4027, 4029, 4031, 4033, 4036-4040, 4042, 4045-4050, 4054-4058, 4061, 4062, 4065-4067, 4069, 407201-4074, 4076, 4078, 4080-4082, 4084, 4085, 4088-4091, 4094, 4097, 4431, 4465, 480101, 480102, 480301, 4804, 4806, 4807, 4809-4811, 4813, 4816, 4818, 4820, 5181, 8101-8104N, Cw*0203
																																																												*0765, 0772, 0786, 5410, 5509, 5537

																																																Well No.
HLA-B allele ³																																																
*0768																																																
*080101-080109, 0804, 0805, 0807-081203, 0814-0819N, 0822, 0824, 0827, 0829-0834, 0836, 0838, 0839, 0841-0847, 4506, 5412, 5503, 5512, 5516, 5525																																																
*0806, 0820																																																
*0813, 4043																																																
*0821, 5120, 5621																																																
*0825																																																
*0826, 1411, 3904, 5401, 5404, 5405N, 5407, 5408N, 5411, 5413, 5415-5419, 550101-550205, 5505, 5507, 5511, 5514, 5515, 5517, 5519, 5520, 5522, 5524, 5526-5530, 5533-5536, 5538, 5612, 5619N, 8301																																																
*0828, 0835, 0837																																																
*0840																																																
*130101, 130102, 1306, 1307N, 1312, 1313, 1317, 1320-1323, 1325, 1326, 1328, 1329, 4408, 4457, 4460, 570103, 5704, 5705, 5725, 5726																																																
*1303, 350101-350403, 3506, 3507, 350901-3513, 3516, 3517, 3519-3521, 3523-3528, 3530, 3534, 3536, 3538-3542, 3547-3550, 3552-3557, 3559, 3563, 3564, 356801, 3570, 3574, 3575, 3577, 3578, 3581-3585, 3590-3592, 3594, 3595, 3598, 4005, 4028, 4410, 4414, 4445, 4451, 4462, 4477, 4802, 510101-5104, 5107-510902, 5111N, 5112, 5114, 5115, 5119, 5122-512402, 5126-5128, 5130, 5132, 5133, 5135, 5138-5141N, 5143, 5144N, 5146, 5148-5153, 5155-5158, 5160, 5165-5167, 5169-5180, 5183, 5184, 520101-5207, 5209-5213, 5215, 5216, 530101-5306, 5308-5310, 5312, 5313, 5315, 5317-5319, 5607, 5711, 5716, 580101-5802, 5804, 5806, 5808-5815, 5817N, 5819, 5821, 5823, 5824, 5904, 780201, 780202, 7804-7806, 8201, 8202, Cw*1515																																																
*1304, 3537, 4904, 511301, 511302, 5137, 5163, 5214																																																
Well No.																																																
HLA-B allele ³																																																
Well No.																																																
HLA-B allele ³																																																
*1405, 1413																																																
*140601, 1815, 3948, 4051																																																

[illegible]

[illegible]