

*18:02	1	3															
*18:03		2		4													
*18:04	1	2			5										14		
*18:05	1	2				6											
*18:06	1	2															
*18:07:01-18:07:02	1	2					7										
*18:08	1	2							9								
*18:09, 18:54		2									11						
*18:10	1							8			11						
*18:11, 18:56, 18:58	1										11						
*18:12	1	2										12					
*18:13	1	2												13			
*18:14	1														14		
*18:15	1	2														15	
*18:17N	1	2															16
*18:18	1	2															
*18:19	1	2												13		15	16
*18:20	1	2															
*18:21	1							8			11						
*18:22	1																
*18:23N	1	2								10							
*18:24	1	2			5												
*18:25	1	2													14		
*18:26	1	2															
*18:27	1								9								
*18:28	1	2															
*18:29		2															
*18:30	1	2												13		15	
*18:31	1	2														15	
*18:32	1	2															
*18:33	1	2															
*18:34, 18:52 ⁶	1	2															
*18:35	1							8			11						
*18:36, 18:67 ⁷	1	2									11						
*18:37, 18:46 ⁸	1	2															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Length of spec.	170	260	260	100	140	130	90	190	125	85	125	85	85	170	100	95	
PCR product(s)							150		255		610	140	265	260		360	
											635						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
*18:38, 18:45 ⁹	1	2															
*18:39	1																
*18:40, 18:51 ¹⁰	1	2															
*18:41	1	2					7					12					
*18:42, 18:43 ¹¹	1	2															
*18:44:01-18:44:02	1												13				
*18:47	1	2															

*18:48	1	2														
*18:49	1	2														
*18:50	1	2						8								
*18:53	1	2														
*18:57	1	2									11				15	
*18:61	1	2														
*18:68	1	2													14	
*18:69	1															
*18:72, 15:185, 39:32, 39:43, 56:31 ¹²		2														
*18:73	1	2										12				
*18:74N	1	2														
*18:75	1	2								9						
*07:02:01-07:04, 07:07-07:13, 07:15-07:16, 07:18:01-07:18:02, 07:21-07:23, 07:25-07:27, 07:29-07:30, 07:33, 07:35-07:39, 07:41-07:42, 07:44-07:52, 07:54-07:59, 07:61-07:67N, 07:70-07:77, 07:79, 07:81-07:89, 07:91-07:95, 07:98-07:99, 07:101-07:104, 07:106-07:108, 07:110-07:111N, 07:113-07:114, 07:116- 07:122, 07:124-07:130, 07:132-07:136, 07:139, 07:141-07:162, 13:35, 15:40, 15:47:01- 15:47:02, 15:49, 15:52, 15:114, 15:117, 15:124, 15:238, 27:34, 27:41, 35:15, 35:33, 35:86, 37:07, 40:03, 40:18, 40:20, 40:24, 40:31-40:33, 40:44, 40:52, 40:59, 40:98, 40:120, 40:159, 46:06, 46:30, 48:08, 48:23, 57:13, 57:25, 58:37, C*02:02:19, C*02:06, C*02:23, C*02:36, C*15:42								8								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Length of spec.	170	260	260	100	140	130	90	190	125	85	125	85	85	170	100	95
PCR product(s)							150		255		610	140	265	260		360
											635					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

*07:14, 07:17, 07:28, 07:96:01-07:96:02, 07:115, 08:14, 08:28, 08:49, 08:74, 08:87, 13:46, 13:51, 15:02:01-15:02:06, 15:06, 15:13:01-15:13:02, 15:17:01:01-15:17:02, 15:21, 15:25:01-15:25:03, 15:31, 15:36, 15:44, 15:55, 15:77, 15:88-15:89, 15:106, 15:112, 15:121, 15:128, 15:137, 15:139, 15:144, 15:156, 15:162, 15:168, 15:170, 15:177, 15:196, 15:208, 15:213-15:214, 15:216, 15:233, 15:240, 27:01-27:06, 27:08-27:10, 27:12-27:19, 27:21, 27:23, 27:25-27:26, 27:29- 27:32, 27:35-27:40, 27:42, 27:44-27:61, 27:63- 27:92, 35:164, 37:01:01-37:03N, 37:06, 37:08, 37:10-37:11, 37:13-37:21, 37:23-37:34, 38:22, 39:17, 40:09, 40:42, 40:60, 40:89, 40:124:01- 40:124:02, 40:172, 40:180, 44:02:01:01- 44:03:11, 44:03:13-44:09, 44:11-44:13, 44:15- 44:16, 44:18-44:30, 44:32-44:42, 44:44, 44:46- 44:50, 44:52N-44:61N, 44:63-44:76, 44:78- 44:81, 44:83-44:106, 44:109-44:121, 44:124- 44:134, 44:136-44:145, 44:147-44:149N, 44:151-44:155, 45:01-45:08, 45:10-45:13, 47:01:01:01-47:08, 48:17, 49:01:01-49:02, 49:05-49:08, 49:10-49:22, 50:01:01-50:02, 50:04-50:17, 51:42, 53:07, 54:15, 55:41, 57:04- 57:05, 57:32, 57:41, 83:01																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Length of spec.	170	260	260	100	140	130	90	190	125	85	125	85	85	170	100	95
PCR product(s)							150		255		610	140	265	260		360
											635					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

[illegible]

[illegible]

*51:08-51:09:02, 51:19-51:20, 51:44N, 51:53, 51:73, 51:97, 51:115, 51:120, 51:122, 52:10:01-52:10:02, 52:19, 52:29, 56:34																
*51:16, 52:08															15	
*51:17-51:18															15	
*51:21, 51:101, 78:07															15	
*51:61:02, 52:21													13		15	16
*51:82													13		15	
*52:28															15	
*73:01																
*78:06															15	
A*24:02:37, A*26:01:16																
A*66:04													w			
C*01:30													13		15	
C*08:51		2											13		15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

				21													*18:02
				21													*18:03
				21				26									*18:04
				21													*18:05
	19			21													*18:06
				21													*18:07:01-18:07:02
				21													*18:08
				21													*18:09, 18:54
																	18:10
																	*18:11, 18:56, 18:58
				21													*18:12
				21													*18:13
				21													*18:14
18				21													*18:15
				21													*18:17N
17				21													*18:18
				21													*18:19
		20															*18:20
w					23												*18:21
				21	22												*18:22
				21													*18:23N
				21													*18:24
				21				26									*18:25
				21		24											*18:26
				21													*18:27
				21										32			*18:28
18				21													*18:29
	w			21													*18:30
17				21													*18:31
	18			21													*18:32
				21	22												*18:33
				21								30					*18:34, 18:52 ⁶
							25										*18:35
				21													*18:36, 18:67 ⁷
				21						27							*18:37, 18:46 ⁸
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		Well No.
100	85	105	290	215	125	150	190	75	90	80	100	65	145	135	175		Length of spec.
255	555	160			150	515		185	180	135	180	120	420				PCR product(s)
	580	405			270												
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		Well No.
				21							28						*18:38, 18:45 ⁹
		19		21													*18:39
				21		23											*18:40, 18:51 ¹⁰
				21													*18:41
				21				26									*18:42, 18:43 ¹¹
				21													*18:44:01-18:44:02
				21								29					*18:47

				21				25								*18:48
				21									31			*18:49
				21								29				*18:50
		19		21		23										*18:53
	18			21												*18:57
				21		23	24									*18:61
				21												*18:68
				21	w											*18:69
				21												*18:72, 15:185, 39:32, 39:43, 56:31 ¹²
				21					27							*18:73
				21					27	28						*18:74N
				21	22											*18:75
																*07:02:01-07:04, 07:07-07:13, 07:15-07:16, 07:18:01-07:18:02, 07:21-07:23, 07:25-07:27, 07:29-07:30, 07:33, 07:35-07:39, 07:41-07:42, 07:44-07:52, 07:54-07:59, 07:61-07:67N, 07:70-07:77, 07:79, 07:81-07:89, 07:91-07:95, 07:98-07:99, 07:101-07:104, 07:106-07:108, 07:110-07:111N, 07:113-07:114, 07:116-07:122, 07:124-07:130, 07:132-07:136, 07:139, 07:141-07:162, 13:35, 15:40, 15:47:01 15:47:02, 15:49, 15:52, 15:114, 15:117, 15:124, 15:238, 27:34, 27:41, 35:15, 35:33, 35:86, 37:07, 40:03, 40:18, 40:20, 40:24, 40:31-40:33, 40:44, 40:52, 40:59, 40:98, 40:120, 40:159, 46:06, 46:30, 48:08, 48:23, 57:13, 57:25, 58:37, C*02:02:19, C*02:06, C*02:23, C*02:36, C*15:42
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
100	85	105	290	215	125	150	190	75	90	80	100	65	145	135	175	Length of spec.
255	555	160			150	515		185	180	135	180	120	420			PCR product(s)
	580	405			270											
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.

																		*07:14, 07:17, 07:28, 07:96:01-07:96:02, 07:115, 08:14, 08:28, 08:49, 08:74, 08:87, 13:46, 13:51, 15:02:01-15:02:06, 15:06, 15:13:01-15:13:02, 15:17:01:01-15:17:02, 15:21, 15:25:01-15:25:03, 15:31, 15:36, 15:44, 15:55, 15:77, 15:88-15:89, 15:106, 15:112, 15:121, 15:128, 15:137, 15:139, 15:144, 15:156, 15:162, 15:168, 15:170, 15:177, 15:196, 15:208, 15:213-15:214, 15:216, 15:233, 15:240, 27:01-27:06, 27:08-27:10, 27:12-27:19, 27:21, 27:23, 27:25-27:26, 27:29, 27:32, 27:35-27:40, 27:42, 27:44-27:61, 27:63, 27:92, 35:164, 37:01:01-37:03N, 37:06, 37:08, 37:10-37:11, 37:13-37:21, 37:23-37:34, 38:22, 39:17, 40:09, 40:42, 40:60, 40:89, 40:124:01-40:124:02, 40:172, 40:180, 44:02:01:01-44:03:11, 44:03:13-44:09, 44:11-44:13, 44:15-44:16, 44:18-44:30, 44:32-44:42, 44:44, 44:46, 44:50, 44:52N-44:61N, 44:63-44:76, 44:78-44:81, 44:83-44:106, 44:109-44:121, 44:124-44:134, 44:136-44:145, 44:147-44:149N, 44:151-44:155, 45:01-45:08, 45:10-45:13, 47:01:01:01-47:08, 48:17, 49:01:01-49:02, 49:05-49:08, 49:10-49:22, 50:01:01-50:02, 50:04-50:17, 51:42, 53:07, 54:15, 55:41, 57:04, 57:05, 57:32, 57:41, 83:01
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.		
100	85	105	290	215	125	150	190	75	90	80	100	65	145	135	175	Length of spec.		
255	555	160			150	515		185	180	135	180	120	420			PCR product(s)		
	580	405			270													
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.		

																*07:34, 08:01:01-08:05, 08:08N-08:10, 08:12:01-08:13, 08:15-08:20, 08:22-08:24, 08:26-08:27, 08:29-08:36, 08:38-08:39, 08:41- 08:48, 08:50-08:69, 08:71-08:73, 08:75-08:76, 08:78, 08:80-08:82N, 08:84-08:86N, 08:88, 08:90-08:94, 13:06, 13:53, 15:42, 15:83, 15:86, 15:188, 15:224, 15:252, 35:60, 35:87, 37:12, 38:01:01, 38:01:03-38:01:05, 38:01:07- 38:07, 38:09-38:21, 38:23-38:28, 38:31-38:37, 38:39-38:40, 39:01:01:01-39:01:01:03, 39:01:03-39:06:02, 39:08-39:14, 39:16, 39:18- 39:20, 39:22-39:24:02, 39:26-39:28, 39:30- 39:31, 39:34-39:35, 39:37, 39:39:01, 39:41- 39:42, 39:44-39:46, 39:51-39:54, 39:56-39:66, 39:68-39:73, 39:75, 40:39, 40:184, 40:199, 41:01-41:03:02, 41:05-41:09, 41:11, 41:13- 41:16, 41:20, 42:01:01-42:02, 42:04-42:08, 42:10-42:15, 42:17, 46:18, 51:36, 54:01:01- 54:01:03, 54:04-54:05N, 54:07-54:08N, 54:10- 54:13, 54:17-54:25, 54:27, 55:01:01-55:05, 55:07, 55:10-55:13, 55:15, 55:17-55:20, 55:25- 55:26, 55:28-55:40, 55:42-55:45, 55:47-55:50, 55:52-55:56, 56:10, 56:12, 56:23, 58:20, 59:01:01:01-59:03, 59:05, 67:01:01-67:03, <i>C</i> *15:24
								25								*07:68:01-07:68:03, 15:138, 15:241, 35:66
										27						*07:109, 13:26
							24									*08:70, 39:36, 55:21
							24				28					*08:83, 38:38
			20							27	28					*13:10
											28					*13:11, 40:91, 40:197
				21								29				*13:15, 51:31
																*13:31, 13:41, 15:27:03, 35:14:01, 35:62, 35:163, 46:01:03, A*26:68, C*01:09, C*02:02:14, C*07:22, C*07:138, C*12:03:16
			20													*14:01:01-14:04, 14:07N, 14:09, 14:11-14:12, 14:14-14:22, 14:24-14:32
			20													*14:05
			20													*14:06:01-14:06:02, 14:08
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
100	85	105	290	215	125	150	190	75	90	80	100	65	145	135	175	Length of spec.
255	555	160			150	515		185	180	135	180	120	420			PCR product(s)
	580	405			270											
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
			20		22											*14:10
				21			24									*14:13
				21												*14:23, 51:29, 51:61:01, 52:24
				21												*15:37, 51:06:01-51:06:02, 51:45, 51:62, 51:135

				21												*15:38:01
				21												*15:38:02
18																*15:71, 15:175, 15:225, 35:50, 46:14, 56:35, 57:39
18		20														*15:204, 37:22, 49:03
		20						25								*15:230, 44:150, 48:26
		20												32		*15:250
		20														*27:28, 27:62
								26								*35:02:02, 35:20:02
				21	22											*35:21, 35:96, 35:188, 40:28, 51:04, 51:46, 51:56:01-51:56:02, 53:06, 53:28, C*15:39
				21	22											*35:24:01-35:24:02, 35:81, 35:109, 35:157, 35:190, 53:02, 57:14, 58:09
														31		*35:27, 35:56, 35:185, 35:203
		20		w												*37:04:01-37:04:02
		20												31		*37:05
						24		26								*39:39:02
				21		24										*39:48, 40:163
				21		24										*40:51
								25								*40:77, 40:87:01-40:87:02, 40:121, 40:158, 48:12, 48:14
				21									29			*40:113
				21												*40:174, 51:64
18						24										*41:12, 54:02, 55:16
		20						26								*44:03:12
		20														*44:108N
		20														*44:146
								27								*48:28
		20														*49:09, 51:85, 52:26
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
100	85	105	290	215	125	150	190	75	90	80	100	65	145	135	175	Length of spec.
255	555	160			150	515		185	180	135	180	120	420			PCR product(s)
	580	405			270											
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
				21												*51:01:01-51:01:34, 51:03, 51:07:01-51:07:02, 51:11N-51:14, 51:22, 51:24:01-51:24:04, 51:26-51:28, 51:30, 51:32-51:33, 51:35, 51:37 51:39, 51:41N, 51:43, 51:48-51:52, 51:55, 51:57-51:58, 51:60, 51:63, 51:65-51:72, 51:74 51:80, 51:83-51:84, 51:86-51:92, 51:94-51:96, 51:98N-51:100, 51:102-51:107, 51:109- 51:114, 51:116-51:119, 51:121, 51:123- 51:134, 51:136-51:137, 52:01:01:01-52:02, 52:04-52:07, 52:09, 52:12-52:18, 52:20, 52:22 52:23, 52:25, 52:27, 52:30, 56:05:01-56:06, 56:21, 58:08:01-58:08:02, 78:01:01-78:03, 78:05

				21													*51:08-51:09:02, 51:19-51:20, 51:44N, 51:53, 51:73, 51:97, 51:115, 51:120, 51:122, 52:10:01-52:10:02, 52:19, 52:29, 56:34
				21							29						*51:16, 52:08
																	*51:17-51:18
				21			24										*51:21, 51:101, 78:07
				21													*51:61:02, 52:21
				21							29						*51:82
17				21													*52:28
											29						*73:01
				21									31				*78:06
									26								A*24:02:37, A*26:01:16
																	A*66:04
																	C*01:30
																	C*08:51
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		Well No.