

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45					
HLA-B allele ^{1,2,3}																																																		
*27:01	1					6		8															23																											
*27:02:01-27:02:02, 27:75, 27:77, 27:95, 27:126		2				6		8															23																											
*27:03, 27:111			3	4		6		8															23																											
*27:04:01, 27:04:04-27:04:05, 27:100, 27:103, 27:105, 27:109, 27:112-27:114					5	6		8	9													21	23																											
*27:04:02-27:04:03					5	6		8	9													?	23																											
*27:05:02-27:05:03, 27:05:05-27:05:19, 27:05:21-27:05:31, 27:84, 27:87-27:88, 27:90:01-27:90:03, 27:93, 27:96:01-27:96:02, 27:98-27:99, 27:117, 27:121-27:122, 27:124, 27:128-27:129, 27:132-27:133, 27:135, 27:137, 27:140-27:141, 27:143-27:145				4		6		8															23																											
*27:05:04, 27:116				4				8															23																											
*27:05:20				4		6		8																																										
*27:06					5	6			9	10												21	23																											
*27:07:01-27:07:04, 27:130				4		6					11																																							
*27:08					5		7	8															23																											
*27:09				4		6														20			23																											
*27:10				4		6		8	9														23																											
*27:11					5	6					11																																							
*27:12					5		7	8					13										23																											
*27:13				4		6		8				12											23																											
*27:14, 27:81				4		6											17																																	
*27:15					5	6		8	9													22																												
*27:16				4		6		8					13										23																											
*27:17				4		6		8						14									23																											
*27:18	1				5		7	8	9						15								23	24																										
*27:19				4		6										16																																		
*27:20					5	6			9		11												23																											
*27:21					5	6																																												
*27:23, 27:131	1				5	6		8															23																											
*27:24					5	6					11							18				21																												
*27:25					5	6		8	w										19	20		21																												
*27:26					5		7	8															22	23																										
*27:27				4		6					11												23																											
*27:28				4		6		8															22																											
*27:29	1			4		6		8								15							23	24																										
*27:30		2				6											16																																	
*27:31					5	6		8															22	23																										
*27:32				4		6																																												
*27:33					5		7				11																																							
*27:34				4		6								14																																				
*27:35				4		6				10									19				23																											
*27:36					5	6		8												20			23																											
*27:37, 27:55, 27:101 ⁴				4		6		8														22	23																											

Negative Control

Negative Control

[illegible]

*27:89								7	8													23																																			
*27:91						5	6			9	10											23																																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45												
*27:92	1					4		6	8	9			13									23																																			
*27:94N						4		6	8													23														36																					
*27:97						4		6	8													23															37																				
*27:102		2						6														23																																			
*27:104						4			8													23					27																														
*27:106-27:107								5	6		9	10										23																																			
*27:108								5	6		8	9								20		23																																			
*27:110, 27:142									6		8											23																																			
*27:115								5	6		8	w							20	21		23																																			
*27:118								4			6	8										23										31		34																							
*27:125								5	6				11																																												
*27:127								4			6																																														
*27:136								4			6												23																																		
*27:138									5		6											21																																			
*27:139								3	4				6		8					19																																					
*07:05:01-07:06, 07:32, 07:40, 07:53, 07:69, 07:78, 07:80, 07:90, 07:97, 07:105, 07:112, 07:123, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 07:206-07:207, 07:209-07:210, 07:213-07:214, 07:249, 08:40, 08:79, 13:01:01-13:01:02, 13:01:04-13:02:16, 13:02:18, 13:07N-13:09, 13:11, 13:13:01-13:23, 13:25, 13:27-13:33, 13:36-13:45, 13:47, 13:49N-13:50, 13:52, 13:54-13:61, 13:63N-13:71, 13:73-13:78, 13:80-13:85, 35:31, 35:88, 39:47, 39:50, 40:01:21, 40:16, 40:23, 40:40, 40:261, 41:19, 51:10, 51:16, 51:31, 51:34, 51:81-51:82, 51:93, 52:08, 54:26, 55:09, 55:22, 55:24, 58:27, 59:09, 78:08																																																									
*07:27, 07:50, 08:04, 08:17, 08:54, 08:110, 15:03:01:01-15:03:05, 15:47:01-15:47:02, 15:54, 15:61-15:62, 15:64:01-15:64:02, 15:68-15:69, 15:91, 15:98, 15:103, 15:123, 15:127, 15:131-15:132, 15:151, 15:156, 15:158, 15:173, 15:210, 15:220, 15:235, 15:242, 15:253, 15:259, 15:266, 15:274, 15:281-15:282, 15:369, 38:03, 39:02:01-39:02:02, 39:08, 39:13:01-39:13:02, 39:23, 39:39:01-39:39:02, 39:49, 39:88, 39:101, 39:105, 42:11, 44:40, 44:44, 44:130, 44:156, 44:210, 48:02:01-48:02:03, 48:08, 48:17, 48:23, 48:25, 52:16, 55:18, 56:44																																																									
*07:28, 07:118, 08:14, 08:87, 44:54, 44:106, 44:135																																																									
*07:174, 07:202																				18																																					
*07:197									8	9										20		23																																			
*07:222												11							18																																						
*08:100, 15:292, 35:113, 39:62, 44:98, 46:27, 52:31:01-52:31:02													12																																												
*08:120, 38:43, 41:38, 51:181																																																									

[illegible]

[illegible]

[illegible]