

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

INTERPRETATION TABLE																																																																														
HLA-B*35 SSP subtyping																																																																														
Amplification patterns of the B*35:01 to B*35:205 alleles																																																																														
Well ⁷																																																																														
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79																																								
130	105			165	165	160	390	385	180	65	220	135	160	300	235	185	160	150	135	175	260	60	245	220	115	200	100	325	135	160	220	205	160	135	220	385	310	100																																								
		250	190																																																																											
1070	800	1070	1070	800	1070	800	1070	1070	1070	1070	1070	1070	1070	1070	1070	800	1070	1070	800	1070	1070	1070	1070	1070	1070	1070	1070	1070	1070	800	1070	800	1070	1070	1070	1070	1070	1070																																								
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79																																								
Well No																																																																														
HLA-B allele ^{2,3}																																																																														
*35:01:01-01-35:01:03, 35:01:05-35:01:16, 35:01:18-35:01:22, 35:01:24-35:01:27, 35:01:29-35:01:31, 35:42:02, 35:148, 35:168, 35:170-35:171, 35:175, 35:177-35:178, 35:180, 35:189, 35:194, 35:200-35:202																																																																														
*35:01:04																																																																														
*35:01:17																																																																														
*35:01:23, 35:42:01																																																																														
*35:01:28																																																																														
*35:02:01-35:02:04, 35:183																																																																														
*35:03:01, 35:03:03-35:03:06, 35:03:08-35:03:10, 35:98, 35:152-35:153, 35:155-35:160, 35:181, 35:204																																																																														
*35:03:02																																																																														
*35:03:07																																																																														
*35:03:11																																																																														
*35:04:01-35:04:02																																																																														
*35:04:03																																																																														
*35:05:01-35:05:02																																																																														
*35:05:03																																																																														
*35:06																																																																														
*35:07																																																																														
*35:08:01-35:08:02, 35:08:04-35:08:05, 35:08:07, 35:08:09																																																																														
*35:09:01																																																																														
*35:09:02-35:09:03																																																																														
*35:10																																																																														
*35:11:01																																																																														
*35:11:02																																																																														
*35:12:01-35:12:03																																																																														
*35:13																																																																														
*35:14:01																																																																														
*35:14:02																																																																														
*35:15																																																																														
*35:16																																																																														
*35:17																																																																														
*35:18																																																																														
*35:19																																																																														
*35:20:01-35:20:02																																																																														
*35:21																																																																														
*35:22																																																																														
*35:23																																																																														
*35:24:01-35:24:02																																																																														
*35:25																																																																														
*35:26																																																																														
*35:27, 35:203																																																																														
*35:28																																																																														
*35:29:01-35:29:02																																																																														
*35:30																																																																														
*35:31																																																																														
*35:32:01-35:32:02																																																																														
*35:33																																																																														
*35:34																																																																														
*35:35																																																																														
*35:36																																																																														
*35:37																																																																														
*35:38																																																																														
Well No																																																																														
Length of spec																																																																														
PCR product(s)																																																																														

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]