

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*37:01:01-37:01:09, 37:31-37:32, 37:38, 37:41, 37:49	1				5							12												
*37:02		2			5							12												
*37:03N	1		3		5							12												
*37:04:01-37:04:02	1			4	5							12												
*37:05	1					6						12												
*37:06:01-37:06:02	1				5		7					12												
*37:07					5			8				12												
*37:08	1				5				9															
*37:09					5						11	12				16								
*37:10	1											12	13											
*37:11, 37:37	1					6						12		14	15			18						
*37:12					5							12				16								
*37:13	1		3	4	5							12												
*37:14	1					w				10		12						18						
*37:15	1				5							12	13											
*37:16Q	1				5							12								20				
*37:17	1				5							12										22		
*37:18	1				5						11	12												
*37:19					5							12											23	
*37:20	1				5							12				16	17							
*37:21	1				5							12						19						
*37:22	1								9			12												
*37:23	1				5							12											23	
*37:24, 37:27 ⁴	1				5							12									21			
*37:25	1				5							12					17							
*37:26, 37:46	1				5					10		12												
*37:28	1	2			5							12												
*37:29	1				5							12						18						
*37:30N	1				5				9			12												
*37:33N	1				5							12	13								21			
*37:34	1											12												
*37:35	1				5							12				16								
*37:36	1				5		7					12								20				
*37:39				4	5							12								20				
*37:40, 37:44 ⁴					5							12												
*37:42N	1				5		7		9			12												
*37:43	1				5						11	12											23	
*37:45	1				5							12			15				19					
*37:47	1				5							12		14										
*37:48	1			4	5							12								20				
*07:02:01-07:03, 07:05:01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 08:79, 15:138, 15:230, 15:241, 15:297, 44:150, 48:05, 48:08, 48:15, C*02:23, C*04:77								8																
*07:04, 07:146								8			11				16									
*07:19, 07:25, 08:01:01-08:01:22, 08:01:24-08:05, 08:07-08:08N, 08:10-08:11, 08:15, 08:17-08:19N, 08:21-08:24, 08:26-08:27, 08:29-08:31, 08:33-08:39, 08:41-08:48, 08:50-08:54, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:71-08:73, 08:75-08:78, 08:80-08:83, 08:85-08:86N, 08:88, 08:90-08:106, 08:108-08:110, 08:112-08:123, 42:01:01-42:01:03, 42:05:01-42:08, 42:10-42:12, 42:14-42:16, 44:166											11				16									
*07:27		2			5			8	9															
*07:65, 07:134						6		8							15			18						
*07:133						6		8																
*07:143								8						14										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*07:197	2				8																			
*08:01:23, 08:14, 08:28, 08:74, 08:87, 08:111, 42:20, 44:106, 44:158											11													
*08:09, 08:12:01-08:12:03, 08:16, 08:60, 08:84, 08:89, 08:107, 15:51, 15:83, 15:179:01-15:179:02, 15:199, 15:218Q, 38:30, 42:04, 42:13, 44:14, 44:17, 44:51, 44:62, 44:123, 51:08:01-51:08:02, 51:20, 51:36, 51:44N, 51:97, 51:141, 51:153, 52:19, 53:22, 55:20, 55:56, 56:13, 57:09, 57:24, 82:01-82:03														16										
*08:32, 35:87						6					11			15	16		18							
*08:49, C*07:68, C*07:302	1																							
*08:62											11			14	16		19							
*13:23, 13:55, 40:47, 40:96, 40:157, 40:201, 44:15, 44:55, 44:103, 44:188, 49:02												12	13											
*14:10, 38:20, 39:42, 44:117, 51:32, 58:52, C*15:05:08, C*17:01:10																					23			
*15:32:01-15:32:02, 15:299, 53:07, C*03:90, C*03:103, C*03:178, C*07:03, C*07:186, C*07:242, C*07:265				4													20							
*15:46, 15:53, 15:106, 15:143, 18:49, 35:03:03, 35:19, 35:27, 35:47, 35:56, 35:154, 35:203, 39:69, 40:10:01-40:10:02, 40:40, 40:49, 40:58, 40:129, 40:164, 40:191, 40:210, 40:219, 40:229, 44:09, 44:46, 44:75, 44:90, 44:129, 47:02, 50:13, 78:06						6																		
*15:71, 15:175, 15:204, 15:210, 15:225, 27:06, 27:24, 27:33, 40:46, 40:93, 40:208, 40:265N, 48:22, 48:30, 50:06											12		14											
*15:202, 15:239, 15:308, 18:01:13, 18:04, 18:25, 18:43, 35:01:01:01-35:01:38, 35:01:40-35:02:02, 35:02:04-35:03:02, 35:03:04-35:08:01, 35:08:03-35:08:04, 35:08:06-35:18, 35:20:01-35:24:02, 35:28-35:37, 35:39-35:45, 35:51, 35:53N-35:55, 35:57-35:58, 35:60-35:62, 35:64:01-35:72, 35:74-35:83, 35:85-35:86, 35:88-35:114, 35:116-35:153, 35:155-35:161, 35:163-35:168, 35:170-35:197, 35:199-35:202, 35:204-35:206, 35:208-35:209, 35:211-35:216N, 35:218-35:221, 35:223-35:230, 35:232-35:241, 35:243-35:247, 35:249-35:253, 39:19:01-39:19:02, 46:40, 56:06, 78:01:01-78:05, 78:07						6							15		18									
*15:209N, 53:03									9															
*15:212, 35:63, 40:01:01-40:01:04, 40:01:06-40:06:10, 40:09, 40:11:01-40:11:02, 40:14:01-40:14:03, 40:18, 40:20, 40:22N, 40:24, 40:26-40:31, 40:33-40:36, 40:38-40:39, 40:42-40:45, 40:48, 40:50-40:57, 40:59-40:67, 40:70:01-40:75, 40:77-40:92, 40:94-40:95, 40:97, 40:99-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:128, 40:130:01-40:134, 40:138-40:151, 40:154-40:156, 40:159-40:161, 40:163, 40:165, 40:167-40:182, 40:184, 40:186-40:187, 40:189-40:190, 40:192-40:197, 40:199-40:200, 40:202-40:207, 40:211-40:212, 40:214-40:218, 40:221-40:228, 40:230, 40:233-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:278, 40:280, 40:282-40:285, 44:131, 45:01:01-45:02, 45:04-45:08, 45:10-45:13, 50:01:01-50:02, 50:04-50:05, 50:07-50:12, 50:14, 50:16-50:20, 50:31-50:38						6					12		14											
*18:01:01:01-18:01:12, 18:01:14-18:03, 18:05-18:06, 18:08, 18:10-18:11, 18:13-18:15, 18:17N, 18:19-18:24, 18:26-18:28, 18:30-18:36, 18:38-18:42, 18:44:01-18:47, 18:50-18:53, 18:55, 18:57-18:65, 18:68-18:78, 18:80-18:103						6								14	15		18							
*18:07:01-18:07:02, 18:29, 35:50, 35:84, 35:162, 35:231						6								14	15		18	19						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*18:09													13											
*18:12, 40:185, 40:209						6						12		14	15		18							

*18:18			4		6								14	15		18									
*18:37, 18:66, 39:31, 46:14, 54:02, 55:16, 56:35, 73:01-73:02													14												
*18:48, 40:25, 40:69, 40:166, 40:244, 50:15					6								14												
*18:54					5			9																	
*18:56					6								14	15	16		18								
*18:79					6												19								
*27:01		2									12	13													
*27:02:01-27:03, 27:05:02-27:05:28, 27:07:01-27:07:04, 27:09-27:10, 27:13-27:17, 27:19, 27:27-27:29, 27:31-27:32, 27:34-27:36, 27:38-27:39, 27:41-27:43, 27:45-27:56, 27:58-27:65N, 27:67, 27:70-27:90:02, 27:93, 27:95-27:99, 27:101, 27:104, 27:108, 27:110-27:111, 27:116-27:121, 27:123-27:124		2									12														
*27:04:01-27:04:04, 27:08, 27:12, 27:18, 27:25-27:26, 27:40, 27:44, 27:66N, 27:68, 27:100, 27:103, 27:105, 27:112-27:115		2									12		14												
*27:11, 27:20-27:21, 27:30, 27:91, 27:102, 27:106-27:107, 27:125, 40:19, 40:76, 40:109, 40:117, 44:18, 49:01:01-49:01:06, 49:03-49:09, 49:11-49:23, 49:27-49:30											12														
*27:23		2															19								
*27:37, 47:04, 48:26		2																							
*27:57, 27:122		2									12									21					
*27:69		2									12		14										23		
*27:92		2						9																	
*27:94N		2									12	13								21					
*27:109		2											14												
*35:01:39, 35:248														15			18								
*35:02:03					6												18								
*35:38, 35:115, 35:169					6									15	16		18								
*35:48			4		6									15			18		20						
*35:52					6									15											
*35:59, 35:242					6									15			18						23		
*35:217, 40:08, 40:68, 40:106, 40:232					6								14				19								
*35:222																	18								
*38:17		2			5			9																	
*40:01:05, 40:37, 40:135, 40:256N, 40:281					6						12														
*40:07, 40:162, 40:220					6						12		14				19								
*40:13											12						19								
*40:15-40:16, 40:23, 40:32, 40:98, 40:158, 40:183, 40:198					6		8				12		14												
*40:136, 40:231, 40:279, 41:01, 41:03:01-41:03:02, 41:06-41:09, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25, 41:28-41:30, 41:32, 45:09, 45:14					6						12		14		16										
*40:152					6						12		14							21					
*40:153					6						12		14								22				
*40:188						7					12														
*40:213					6						12		14										23		
*41:02:01-41:02:05, 41:04, 41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31					6					11	12		14		16										
*41:05					w						12				16										
*41:10					6					11	12		14												
*41:26											12		14		16										
*42:02, 42:09, 42:17-42:18										11			14		16										
*45:03					6						12		14				17								
*47:01:01-47:01:02, 47:06-47:09						7																			
*47:03					w																				
*47:05		2				7																			
*51:103, 51:116, 51:151																	19								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Negative Control