

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	
HLA-B allele ^{1,2,3}																															
*37:01:01-37:01:11, 37:01:13-37:01:22, 37:31-37:32, 37:38, 37:41, 37:49-37:51, 37:53-37:54, 37:57, 37:59-37:61, 37:63-37:65, 37:69-37:76, 37:78-37:82N, 37:86N-37:91	1				5							12																			
*37:01:12, 37:34	1											12																			
*37:02		2			5							12													25						
*37:03N	1		3		5							12																			
*37:04:01-37:04:02	1			4	5							12																			
*37:05	1					6						12																			
*37:06:01-37:06:02	1				5		7					12																			
*37:07					5			8				12																			
*37:08	1				5				9																						
*37:09					5						11	12				16															
*37:10	1											12	13																		
*37:11, 37:37	1					6						12		14	15			18													
*37:12, 37:56					5							12				16															
*37:13	1		3	4	5							12																			
*37:14	1					w				10		12						18													
*37:15	1				5							12																28			
*37:16Q	1				5							12								20											
*37:17	1				5							12										22									
*37:18	1				5						11	12																			
*37:19:01					5							12											23								
*37:19:02, 37:40, 37:44, 37:66, 37:83					5							12																			
*37:20	1				5							12				16	17														
*37:21	1				5							12							19												
*37:22	1							9				12																			
*37:23	1				5							12											23								
*37:24, 37:27 ⁴	1				5							12									21										
*37:25	1				5							12					17														
*37:26, 37:46 ⁴	1				5				10			12																			
*37:28	1	2			5							12																			
*37:29	1				5							12						18													
*37:30N	1				5			9				12																			
*37:33N	1				5							12									21								29		
*37:35, 37:77, 37:84	1				5							12				16															
*37:36	1				5		7					12									20										
*37:39				4	5							12									20										
*37:42N	1				5		7	9				12																			
*37:43	1				5						11	12											23								
*37:45	1				5							12				15			19												
*37:47	1				5							12		14																	
*37:48	1			4	5							12									20										
*37:52	1				5							12																			
*37:55, 37:62, 37:85	1				5							12																			
*37:58	1				5							12															26				
*37:67	1					6								14	15			18	19												
*37:68	1				5							12																27			
*07:02:01-07:07:03, 07:05:01-07:18:02, 07:20-07:24, 07:26, 07:28-07:32, 07:34-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:59, 07:61-07:64, 07:66-07:99, 07:101-07:121, 07:123-07:132, 07:135N-07:138, 07:140-07:142, 07:144-07:145, 07:147-07:154, 07:156-07:163, 07:166-07:196, 07:198-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:237-07:260, 07:262-07:272N, 07:274-07:300, 07:302-07:328, 07:330N-07:331, 07:333-07:342, 07:344-07:345, 07:347-07:348, 07:350-07:358, 07:360Q-07:362, 07:364-07:385, 07:387, 07:389-07:395, 08:79, 08:156, 15:138, 15:230, 15:241, 15:297, 44:150, 48:05, 48:08, 48:15, C*02:23, C*03:260, C*04:77								8																							
*07:04:01-07:04:02, 07:146, 07:349								8			11					16															
*07:19, 07:25, 08:01:01-08:01:22, 08:01:24-08:05, 08:07-08:08N, 08:10-08:11, 08:15, 08:17-08:19N, 08:21-08:24, 08:26-08:27, 08:29-08:31, 08:33-08:39, 08:41-08:48, 08:50-08:54:02, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:71-08:73, 08:75-08:78, 08:80-08:83, 08:85-08:86N, 08:88, 08:90-08:106, 08:108-08:110, 08:112-08:125, 08:127-08:128, 08:130, 08:132, 08:134-08:138, 08:140-08:142, 08:144-08:154, 08:157-08:170, 08:172-08:179, 08:182-08:183, 08:185-08:199, 08:201-08:204, 08:206-08:225, 08:228-08:231, 08:233-08:238, 08:240-08:241, 08:243-08:244, 08:246-08:249, 42:01:01-42:01:04, 42:05:01-42:08, 42:10-42:12, 42:14-42:16, 42:21-42:25, 44:166, 44:213											11					16															

Negative Control

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
*07:27, 07:236		2			5			8	9						15		18													
*07:65, 07:134						6		8							15		18													
*07:133						6		8																						
*07:143								8						14																
*07:197		2						8																						
*07:273					5			8																						
*07:329								8														23								
*08:01:23, 08:14, 08:28, 08:74, 08:87, 08:111, 08:200, 08:227, 08:245, 42:20, 44:106, 44:158											11																			
*08:09, 08:12:01-08:12:03, 08:16, 08:60, 08:84, 08:89, 08:107, 08:129, 08:181, 08:184, 08:232, 15:51, 15:83:01:01-15:83:01:02, 15:179:01-15:179:02, 15:199, 15:218Q, 15:518, 38:30, 39:156, 42:04, 42:13, 44:14, 44:17, 44:51, 44:62, 44:123, 44:230, 51:08:01:01-51:08:04, 51:20, 51:36, 51:44N, 51:97, 51:141, 51:153, 51:263, 51:275, 52:19, 53:22, 55:20, 55:56, 56:13, 57:09, 57:24, 57:119, 82:01:01:01-82:03, C*07:626															16															
*08:32, 08:133, 08:180, 08:239, 35:87						6					11				15	16		18												
*08:49, C*07:68, C*07:302	1																													
*08:62, 08:242, 42:26											11			14	16			19												
*08:126		2			5				9		11					16														
*08:131, 08:139		2							9		11					16														
*08:155, 41:47						6					11					16														
*08:226, 42:02:01:01-42:02:02, 42:09, 42:17-42:18, 42:27											11			14	16															
*13:23, 13:55, 40:47, 40:96, 40:157, 40:201, 44:15:01:01-44:15:01:02, 44:55, 44:103, 44:188, 44:212, 49:02												12	13																	
*14:10, 15:227, 15:460, 38:20:01, 39:42, 39:106, 44:117, 44:251, 44:338, 51:32, 58:52, C*03:26:02, C*15:05:08, C*15:140, C*17:01:10																							23							
*15:32:01-15:32:02, 15:299, 44:209, 53:07, C*03:90, C*03:103, C*03:178, C*03:300, C*03:474, C*07:03, C*07:186, C*07:242, C*07:265, C*07:748				4																20										
*15:46, 15:53, 15:106, 15:143, 15:525, 18:01:21, 18:49, 18:107-18:108, 18:132, 18:142, 35:01:52, 35:02:14, 35:03:03, 35:19, 35:27, 35:47, 35:56, 35:154, 35:203, 35:274, 35:299, 35:355, 35:450, 39:69, 40:06:15, 40:10:01:01-40:10:02, 40:40, 40:49, 40:58, 40:129, 40:164, 40:191, 40:210, 40:219, 40:229, 40:318, 40:325, 40:328, 40:342, 40:399N, 40:449, 44:09, 44:46, 44:75, 44:90, 44:129, 44:254, 44:277, 44:438N, 47:02, 48:41, 50:13, 50:43, 56:67, 78:06, 78:09-78:10						6																								
*15:71, 15:175, 15:204, 15:210, 15:225, 15:344, 15:381, 18:178, 27:06:01:01-27:06:01:02, 27:20, 27:24, 27:33, 27:91, 27:138, 27:153, 27:208, 39:101, 40:46, 40:93, 40:208, 40:265N, 40:400, 40:430, 45:16, 48:22, 48:30, 50:06												12		14																
*15:129, 15:395, 48:21																									25					
*15:202, 15:239, 15:308, 15:376, 18:01:13, 18:04:01-18:04:02, 18:25, 18:43, 35:01:01:01-35:01:38, 35:01:40-35:01:51, 35:01:53-35:02:02, 35:02:04-35:02:08, 35:02:10-35:02:13, 35:02:15-35:03:02, 35:03:04-35:08:01:08, 35:08:03-35:08:04, 35:08:06-35:18, 35:20:01-35:24:02, 35:28:01:01-35:37, 35:39-35:43:01, 35:43:03-35:45, 35:51, 35:53N-35:55, 35:57-35:58, 35:59:02-35:62, 35:64:01-35:72, 35:74-35:83, 35:85-35:86, 35:88-35:114, 35:116-35:153, 35:155-35:161, 35:163-35:168, 35:170-35:197, 35:199-35:202, 35:204-35:206, 35:208:01-35:209, 35:211-35:216N, 35:218-35:221, 35:223-35:230, 35:232-35:241, 35:243-35:247, 35:249-35:253, 35:255-35:259, 35:261-35:270, 35:272-35:273, 35:275-35:279, 35:281-35:292, 35:294-35:298, 35:300-35:309, 35:312-35:316, 35:318-35:322, 35:324-35:328, 35:330-35:354, 35:356-35:360, 35:362-35:423, 35:425-35:432, 35:434-35:440, 35:442-35:449, 35:451-35:466, 39:19:01-39:19:02, 40:365, 46:40:01-46:40:02, 56:06, 56:45, 58:64, 78:01:01:01-78:05, 78:07-78:08						6									15		18													
*15:209N, 39:139N, 53:03, 53:55, A*68:112:02									9																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30

Negative Control

[illegible]

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30
*27:57, 27:122		2										12									21									
*27:69		2										12		14									23							
*27:92, 55:90		2							9																					
*27:94N		2										12									21								29	
*27:109		2												14																
*27:192												12		14									23							
*27:203		2										12																28		
*35:01:39, 35:248, 35:424															15			18												
*35:38, 35:115, 35:169, 35:260						6									15	16		18												
*35:48, 35:433				4		6									15			18		20										
*35:52, 35:310						6									15															
*35:59:01, 35:242						6									15			18					23							
*35:217, 40:08, 40:68, 40:106, 40:232						6								14					19											
*35:222, 35:329																		18												
*35:293															15															
*38:17		2			5				9																					
*38:61																										26				
*40:01:05, 40:37, 40:135, 40:256N, 40:281, 40:306, 40:335, 40:344, 40:423						6						12																		
*40:07, 40:162, 40:220						6						12		14					19											
*40:13												12							19											
*40:15-40:16:01:02, 40:23, 40:32, 40:98, 40:158, 40:183, 40:198, 40:355						6		8				12		14																
*40:136, 40:231, 40:279, 41:01:01-41:01:07, 41:03:01-41:03:02, 41:06-41:09, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25, 41:28-41:30, 41:32, 41:34-41:35, 41:37, 41:44, 41:48, 41:50, 41:53-41:54, 41:56-41:57, 41:59, 41:61, 41:63, 45:09, 45:14						6						12		14		16														
*40:152						6						12		14							21									
*40:153						6						12		14								22								
*40:188							7					12																		
*40:213						6						12		14									23							
*40:225:02						6						12		14												25				
*40:429		2				6						12		14												25				
*41:02:01-41:02:09, 41:04, 41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:43, 41:45N, 41:49, 41:51, 41:58						6					11	12		14		16														
*41:05						w						12				16														
*41:10, 41:52						6					11	12		14																
*41:26												12		14		16														
*41:33						6										16														
*41:46											11	12				16			19											
*41:55						6					11	12		14		16								24						
*41:60												12				16														
*44:257, 47:01:01-47:01:05, 47:06-47:10							7																							
*45:03						6						12		14			17													
*47:03						w																								
*47:05		2					7																			25				
*54:02, C*03:278, C*03:504, C*07:714, C*07:724, C*16:86														14										24						
C*06:77																								24						
C*07:494Q																				20										
C*07:516				4																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30

Negative Control