

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*38:01:01, 38:01:03-38:01:07, 38:38-38:40, 38:42	1	2	3	4									13		15									
*38:01:02	1	2	3	4											15									
*38:02:01-38:02:02, 38:02:04-38:02:05, 38:35, 38:43-38:44	1	2	3		5								13		15									
*38:02:03		2	3		5								13		15									
*38:03	1				5	6	7						13											
*38:04		2	3		5		7						13		15									
*38:05		2	3	4				8					13											
*38:06		2							9				13		15									
*38:07	1	2							9	10			13		15									
*38:08	1	2	3		5						11				15									
*38:09	1	2	3	4								12	13		15									
*38:10	1	2	3	4										14	15				19				23	
*38:11, 38:19 ^d	1	2	3	4									13		15	16								
*38:12	1	2	3	4						10			13		15									
*38:13	1		3	4							11		13											
*38:14, 38:41	1		3	4								12	13											
*38:15	1	2	3		5			8					13		15									
*38:16	1			4						10			13		15									
*38:17, 39:37 ^d	1	2	3										13		15	16								
*38:18	1	2	3		5						11		13		15									
*38:20	1	2	3	4				8					13		15									
*38:21	1	2	3	4			7		9				13		15									
*38:22	1	2	3	4				8							15									
*38:23	1	2	3		5								13		15		17							
*38:24	1	2	3	4									13		15			18						
*38:25		2	3				7						13		15									
*38:26	1			4								12	13											
*38:27	1	2	3	4							11		13		15									
*38:28	1			4									13		15				20					
*38:29	1	2	3		5										15							22		
*38:30	1	2	3	4											15				19		21			
*38:31	1	2	3	4									13		15								23	
*38:32	1	2	3	4									13	14	15								23	
*38:33		2	3	4				8		10			13											
*38:34N	1	2	3	4		6	7						13		15									
*38:36	1	2	3	4											15				19					
*38:37	1	2	3	4											15			18						
*38:45	1	2	3		5										15									
*07:02:01-07:02:31, 07:02:33-07:04, 07:07-07:12, 07:14-07:15, 07:17-07:24, 07:26, 07:28-07:31, 07:33:01-07:33:02, 07:35-07:36, 07:39, 07:41-07:47, 07:49N, 07:52, 07:54-07:55, 07:57-07:68:03, 07:70-07:71, 07:73-07:74, 07:76-07:77, 07:79, 07:81-07:84, 07:86-07:89, 07:92-07:96:02, 07:98-07:104, 07:106-07:109, 07:111N, 07:113-07:122, 07:124-07:136:01, 07:139, 07:141-07:148, 07:150-07:154, 07:156-07:163, 07:165-07:175, 07:177-07:179, 07:181N, 07:183-07:200, 07:203-07:204, 14:21, 15:01:01:01-15:01:04, 15:01:06-15:02:06, 15:04:01, 15:06-15:08:01, 15:11:01-15:17:02, 15:19, 15:21, 15:24:01-15:28, 15:32:01-15:36, 15:38:01-15:40, 15:43-15:46, 15:50, 15:53, 15:56-15:57, 15:60, 15:63, 15:65-15:67, 15:70, 15:76-15:79N, 15:81-15:82, 15:85, 15:87, 15:89, 15:92, 15:94N-15:97, 15:101-15:102, 15:104, 15:106, 15:109-15:110, 15:112-15:113, 15:116-15:118, 15:120-15:122, 15:125-15:126, 15:128-15:129, 15:135, 15:137-15:144, 15:146-15:149N, 15:152, 15:154, 15:157, 15:159-15:160, 15:163-15:172, 15:174, 15:177-15:178, 15:180-15:184, 15:187, 15:190N-15:196, 15:201-15:209N, 15:211-15:217, 15:219, 15:223, 15:225, 15:227-15:228, 15:230-15:234, 15:236, 15:239-15:241, 15:244-15:249, 15:254-15:262N, 15:264, 15:267-15:273, 15:276-15:280, 15:284-15:289, 15:291, 27:04:01-27:04:03, 27:06, 27:10, 27:15, 27:21, 27:25, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91, 27:100, 27:103, 27:105-27:107, 35:11:01-35:11:03, 35:14:01-35:14:02, 35:21, 35:43:01, 35:58, 35:67, 35:79, 35:86, 35:96, 35:99, 35:102, 35:117-35:118, 35:135, 35:167, 35:185, 35:188, 35:213, 35:232, 40:32, 44:76, 44:79:01, 44:146, 44:150, 46:01:01-46:05, 46:07N-46:08, 46:10, 46:12, 46:14-46:17, 46:20, 46:22-46:24, 46:26-46:32, 46:34-46:42, 49:01:01-49:02, 49:04-49:10, 49:12-49:24, 50:01:01-50:02, 50:04-50:20, 50:31-50:34, 54:06, 55:14, 55:58, 56:03, 56:32, 57:13, 57:22																								
*07:02:32	1											12		14										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:05:01-07:06, 07:32, 07:34, 07:40, 07:53, 07:78, 07:80, 07:90, 07:97, 07:105, 07:112, 07:123, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 08:20, 08:79, 13:16, 13:20, 13:48, 13:62, 15:42, 27:20, 27:24, 40:15-40:16, 40:23, 40:51, 40:158, 40:183, 40:198, 42:19, 51:01:40, 51:78:01-51:78:02, 51:104, 51:112, 51:147, 52:02:01-52:02:02, 52:20, 52:32, 54:20, 55:01:01-55:01:11, 55:03, 55:05, 55:09, 55:11, 55:15, 55:17, 55:24-55:25, 55:28-55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:44-55:45, 55:52-55:53, 55:59-55:60, 56:05:01-56:06, 56:21, 56:25, 56:37, 57:57, 57:63, 78:01:01-78:03, 78:05-78:07													14						19				23	
*07:13, 07:110														14		16								
*07:16, 07:37-07:38, 07:75, 07:91		2												14	15									
*07:27		2				6								14	15	16								
*07:50, 18:15, 18:19, 18:21, 18:30, 18:57		2				6								14	15									
*07:56:01-07:56:02, 07:72														14						20				
*07:69			3											14					19				23	

Negative Control

Negative Control

[illegible]

[illegible]

*39:39:02									6								13												Nega
*39:42	1	3					8					12	13																
*39:49	2								6	7									15					22					
*39:52, 39:54, 39:58, 67:01:01, 67:03	1												12	13															
*39:63																	13					20							
*39:73	1	3											12	13					17										
*39:76	3												12	13					16										
*40:05, 40:26, 40:28, 40:95, 40:148, 40:161, 40:174																	14	15					19	23					
*40:21, 40:58, 40:137																	13	15					19	?					
*40:184																					?	19							
*40:188																					?	19							
*40:248													9					15					19	?					
*44:06, 53:22													9					15					19	?	21				
*44:130	2				5	6	7											15											
*44:156	2				5	6	7																						
*48:05, 48:15	2				6				7							14	15					19	23						
*48:06	2																		15					19					
*48:31	2				7														15					19					
*49:03													9					14					20						
*49:26	4												10					14											
*51:01:01-51:01:18, 51:01:20-51:01:30, 51:01:32-51:01:38, 51:01:41-51:02:02, 51:02:04-51:04, 51:06:01-51:07:02, 51:11N-51:14, 51:16, 51:21-51:24:05, 51:26-51:30, 51:32-51:35, 51:38-51:39, 51:41N, 51:43, 51:46, 51:48-51:52, 51:55-51:56:02, 51:58-51:60, 51:62-51:71, 51:74-51:77, 51:79-51:80, 51:82-51:84, 51:86, 51:88-51:89, 51:95-51:96, 51:98N-51:100, 51:102, 51:105-51:107, 51:109-51:111, 51:113-51:114, 51:117-51:119, 51:121, 51:123-51:134, 51:136-51:138, 51:140, 51:142-51:145, 51:149N-51:152, 51:154, 52:01:01:01-52:01:09, 52:01:12-52:01:19, 52:04-52:09, 52:11, 52:13-52:15, 52:17-52:18, 52:22-52:25, 52:30-52:31:01, 58:08:01-58:08:02													9					14					19	23					
*51:01:19													9					14					w	w					
*51:01:31, 51:02:03, 51:17-51:18, 51:37, 51:45, 51:85, 51:90, 51:92:01, 51:94, 51:135, 51:146, 52:01:11, 52:12, 52:26, 53:06, 53:08:01-53:08:02, 53:28, 58:06, 58:19, 58:46													9					14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24					
*51:01:39													9					14					23						
*51:05, 51:09:01-51:10, 51:15, 51:31, 51:40, 51:73, 51:81, 51:93, 51:108, 51:115, 51:122, 51:139, 51:148, 52:03, 52:10:01-52:10:03, 58:27-58:28													9									19							
*51:08:01, 51:20, 51:44N, 51:97, 51:141, 51:153, 52:19													9									19	21						
*51:19, 51:42, 53:01:01-53:02, 53:04-53:05, 53:07, 53:10, 53:14-53:18, 53:20, 53:23-53:27, 53:29, 58:01:01-58:02, 58:04-58:05, 58:07, 58:09-58:11, 58:13-58:16:02, 58:18, 58:21-58:26, 58:29, 58:31N-58:35, 58:37-58:43, 58:45:01, 58:47-58:48													9																
*51:36													9					14					19	21					
*51:53, 51:120, 52:28-52:29													9									19	23						
*51:57, 51:103													9					14	15					19	23				
*51:61:01-51:61:02, 52:21													9					14					19	w					
*51:72, 51:87, 51:91													9					14					19						
*51:101													9					13	14					23					
*51:116	2												9					14	15					19	23				
*52:01:10	4												9					14					19	23					
*52:16	2				6				7							14					19	23							
*52:27	2				6				7			9					14					19	23						
*53:19													9									w							
*53:21													9					15											
*55:54, C*15:24																					19	23							
*55:55N																	14					23							
*55:56, 57:09																	14					19	21						
*58:12	2												9																
*58:20													9					13											
C*02:02:07, C*05:01:06									8																				
C*02:68																					19	w							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24					

Negative Control