

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-B allele ^{1,2,3}																																		
*38:01:01:01-38:01:02, 38:01:04-38:01:07, 38:01:09-38:01:12, 38:01:14, 38:01:16-38:01:18, 38:01:20-38:01:21, 38:38-38:40, 38:42, 38:51, 38:53, 38:56-38:60, 38:66, 38:69, 38:71, 38:73, 38:77-38:78, 38:80N, 38:83N, 38:86-38:87, 38:90-38:91, 38:93-38:95, 38:161, 38:169, 38:171	1	2	3	4									13	15													27							
*38:01:03, 38:01:13, 38:67	1	2	3	4									13	15																				
*38:01:08		2	3										13	15													27							
*38:01:15	1	2	3										13	15													27							
*38:01:19		2	3	4									13	15													27							
*38:02:01:01-38:02:02:02, 38:02:04-38:02:11, 38:43-38:44, 38:47-38:49, 38:62, 38:72, 38:75-38:76, 38:79, 38:82, 38:85, 38:92, 38:97, 38:160, 38:162-38:165N, 38:167, 38:172-38:173Q	1	2	3		5								13	15																				
*38:02:03		2	3		5								13	15																				
*38:03	1				5	6	7						13																					
*38:04		2	3		5		7						13	15																				
*38:05		2	3	4				8					13														27							
*38:06		2							9				13	15													26	27						
*38:07	1	2							9	10			13	15													26	27						
*38:08	1	2	3		5						11			15																				
*38:09	1	2	3	4								12	13	15																				
*38:10	1	2	3	4										14	15			19				23					27							
*38:11, 38:19⁴	1	2	3	4									13	15	16												27							
*38:12	1	2	3	4						10			13	15													27							
*38:13	1		3	4							11		13														27							
*38:14	1		3	4									13														27						31	
*38:15	1	2	3		5			8					13	15																				
*38:16	1			4						10			13	15													27							
*38:17	1	2	3										13	15	16								24											
*38:18	1	2	3		5						11		13	15																				
*38:20:01-38:20:02	1	2	3	4				8					13	15													27							
*38:21	1	2	3	4				7	9				13	15													27							
*38:22	1	2	3	4				8					13	15													27							
*38:23	1	2	3		5								13	15			17																	
*38:24	1	2	3	4									13	15			18										27							
*38:25		2	3					7					13	15													27							
*38:26	1			4									13														27						31	
*38:27	1	2	3	4							11		13	15													27							
*38:28	1			4									13	15						20							27							
*38:29	1	2	3		5									15								22												
*38:30	1	2	3	4										15				19		21							27							
*38:31	1	2	3	4									13	15												25	27							
*38:32	1	2	3	4									13	14	15								23				27							
*38:33		2	3	4				8	10				13														27							
*38:34N	1	2	3	4			6	7					13	15													27							
*38:35	1	2	3		5	6							13	15																				
*38:36	1	2	3	4										15					19								27							
*38:37	1	2	3	4										15				18									27							
*38:41	1		3	4									13																					
*38:45	1	2	3		5									15																				

Negative Control

*14:02:01:01-14:02:02, 14:02:04-14:02:05, 14:02:07, 14:02:09-14:02:19, 14:02:21-14:03, 14:05-14:06:02, 14:09, 14:15-14:17, 14:20, 14:22-14:23, 14:25, 14:27, 14:29-14:31, 14:33-14:35, 14:37-14:39, 14:44-14:45, 14:50-14:52, 14:58, 14:61-14:62, 14:64, 14:67-14:70Q, 14:74-14:76N, 14:79N, 14:84-14:85N, 14:88-14:89, 14:91-14:92, 14:94, 14:98, 14:100N-14:101N, 14:106-14:107, 14:109		3			8				14											31
*14:02:08, 14:02:20, 14:60, 14:63, 14:72N		3			8				14											
*14:04		3			w				14											31
*14:11		3			8			10	14											31
*14:13, 14:96		3			8				14			19		23						31
*14:14	1	2	3						14	15										
*14:18		2	3						8											
*14:28			3						10											31
*14:36, 14:42:02, 14:57, 14:99									8											31
*14:41N, 14:43, 14:56, 14:59, 14:90			3						8											31
*14:48									8											31
*14:53, 39:107	1		3						10											31
*14:54		1																		31
*14:73			2			6	8													31
*14:93			3	4			8													31
*15:03:01:01-15:03:04, 15:03:06-15:03:11, 15:03:13, 15:47:01-15:47:02, 15:54, 15:62, 15:98, 15:103, 15:127, 15:131-15:132, 15:156, 15:158, 15:173, 15:210, 15:220:01:01-15:220:03, 15:242:01-15:242:02, 15:251:01-15:251:03, 15:253, 15:266, 15:274, 15:281-15:282, 15:369, 15:379, 15:389, 15:429, 15:444, 15:448, 15:475, 15:502, 15:517, 15:527, 15:545, 15:560, 15:588, 15:610, 15:614:01:01-15:614:01:02, 15:616Q-15:617, 50:77		2				6	7													
*15:03:05, 15:64:01-15:64:02		2				6														
*15:03:12, 15:49, 15:74		2					7													
*15:09:01-15:09:02, 15:229, 15:252, 15:329, 15:435		2	3																	
*15:10:01:01-15:10:08, 15:18:01:01-15:18:09, 15:23, 15:37, 15:72, 15:80, 15:90, 15:93, 15:99, 15:108, 15:119, 15:133-15:134, 15:153, 15:161, 15:197:01-15:198, 15:221, 15:226N, 15:238, 15:263, 15:290, 15:292-15:293, 15:306-15:307, 15:311, 15:313-15:314, 15:323, 15:335, 15:337-15:338, 15:351, 15:354, 15:380N, 15:382, 15:388, 15:414, 15:426-15:427, 15:433-15:434, 15:436, 15:439, 15:445, 15:447, 15:451, 15:455, 15:487N, 15:498, 15:503, 15:506, 15:526, 15:530, 15:548, 15:570, 15:573, 15:607, 15:611, 15:620		2	3																	
*15:13:02, 49:01:01:01-49:01:13, 49:01:15-49:01:22, 49:04:01-49:10, 49:12-49:14, 49:16-49:21, 49:23-49:24, 49:27-49:30, 49:32-49:44, 49:46-49:47:02, 49:49-49:54, 49:56-49:66, 49:68-49:69, 49:71-49:74, 49:76																				27
*15:29:01:01-15:29:01:02, 15:509, 15:619		2																		
*15:51		2	3																	
*15:52, 15:114, 15:124, 15:176, 15:200, 15:275:01-15:275:02, 15:312		2	3																	
*15:61, 15:397						6	7													14
*15:68, 15:556, 15:571, 15:582, 37:07, 37:97, 48:08, 48:25		2				6	7													14 15
*15:69		2				6	7													14
*15:71, 15:175, 15:381, 15:446, 27:18, 27:92, 40:98, 50:43																				14 15
*15:91, 15:123, 15:151, 15:235, 15:539, 15:628		2				6	7													
*15:115		2	3		5															14
*15:186		2	3																	22
*15:189:01-15:189:02							8		10											14
*15:199, 44:452																				14
*15:243		2	3				7													14
*15:265																				19
*15:294N, 15:563, 15:612			3																	14
*15:376		2	3				7													14

Negative Control

