

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	
HLA-B allele ^{1,2,3}																																								
*39:01:01:01, 39:01:01:03-39:01:01:04, 39:01:03, 39:01:06-39:01:08, 39:01:10-39:01:22, 39:65-39:67, 39:69, 39:71, 39:80-39:81, 39:84-39:85, 39:94, 39:98, 39:100, 39:103	1	2			5														19		21		23	24																
*39:01:01:02L, 39:60 ⁵	1	2			5														19		21		23	24	25															
*39:01:04	1	2			5														19		21		23	24																
*39:01:05, 39:59 ⁵	1	2			5				9										19		21		23	24																
*39:01:09	1	2																	19		21		23	24																
*39:02:01	1				5	6													18		21		23	24																
*39:02:02, 39:101, 39:105	1				5	6													18		21		23	24																
*39:03	1	2			5		7												19				23	24																
*39:04	1	2			5		8												19		21		23																	
*39:05:01-39:05:02	1	2	3		5														19		21		24																	
*39:06:01-39:06:02, 39:06:04-39:06:05, 39:90	1	2			5				9										19				23	24					29											
*39:06:03	2								9										19										29											
*39:07	1	2	3		5				10	12									19		21		24																	
*39:08	1		3		5	6				11									18		21		24																	
*39:09:01-39:09:03	1	2			5				10						16				19		21		23	24																
*39:10:01, 39:96, 67:01:02-67:01:03, 67:05 ⁴	1													14					19		21																			
*39:10:02	1													14						21																				
*39:11	1	2	3		5					11									19		21		24																	
*39:12	1	2			5									15					19		21		23																	
*39:13:01-39:13:02	1		3		6														18		21																			
*39:14	1	2			5		7								16				19				23	24																
*39:15	1	2			5					12									19		21		23	24																
*39:16	1													14	16				19																					
*39:17	1						7							14					19		21																			
*39:18	1	2			5				9	11									19		21		23	24						30										
*39:19:01	1	2								10									19		21																			
*39:19:02	1	2			5					10									19		21		23	24																
*39:20	1		3											14					19		21																			
*39:22	1	2			5														18		21		23	24																
*39:23	1					6													17	18	21																			
*39:24:01-39:24:02	1	2			5		7						13						19				23	24																
*39:25N, 39:44 ⁵	1	2			5														17		21		23	24																
*39:26, 39:51 ⁵	1	2		4	5														19		21		23	24																
*39:27	1	2			5														19	20	21		23	24																
*39:28	1	2			5								13						19				23	24																
*39:29	1	2			5		7					12			16				19				23	24																
*39:30, 39:107	1	2			5														19				23	24							31									
*39:31	1	2			5									15					19		21		23	24																
*39:32	2				5														19				23	24					27	28										
*39:33	2				5														19				23	24					28			31								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	
*39:34	2				5														19				23	24					29		31									
*39:35, 39:75 ⁵	1	2			5														19		21		23	24					30											
*39:36	2				5					11									19				23	24							31									
*39:37	1	2	3		5		7												19				24																	
*39:38Q	1	2			5									15					19		21		23	24			27													
*39:39:01	1				5	6													19		21		23	24																
*39:39:02	1				5	6													19		21																			
*39:40:01N-39:40:02N	1	2			5					11									19		21		23	24				28												
*39:41, 39:77 ⁵	1	2			5								13						19		21		23	24																
*39:42	2				5								13						19		w		23	24																
*39:43	2	3			5					10									19				23	24																
*39:45	1	2			5														18	19	21		23																	
*39:46	1	2			5		8												19		21		23	24																
*39:47, 14:46 ⁴	2				5		8												19				23	24																
*39:48	1	2			5														19		21		23	24				27												
*39:49	1		3			6					12								18		21																			
*39:50	2				5		8												19				23	24					29											
*39:52	1				5			</																																

[illegible]

[illegible]

[illegible]

*54:10, 54:20, 55:09, 55:21, 55:37, 55:52										11		14																	29	31									
*54:14-54:16, 55:23, 55:27, 55:41, 55:46, 55:55N, 55:58, 55:68												14																	29										
*56:11												14																	28										
*56:31			3							10		14																											
*56:34												14																27	28			31							
*57:12																																31				35			
*67:01:01, 67:04	1				5							14			19	21	23	24																					
*67:03	1				5	7						14			19	21	23	24																					
*73:01-73:02															19															29									
*81:08			3									14																				31							
A*02:84, C*02:57, C*03:97, C*05:104, C*07:181, C*07:328																	22																						
A*02:403, A*30:63, A*68:108						7																									30								
A*23:01:18																		w																					
A*25:01:08																			23																				
C*01:30																																							
C*01:56N																																							
C*04:188	1							9																												33			
C*05:70	1					7																																	
C*06:34:02, C*07:447										11																													
C*06:40	1					7		9																															
C*07:137:01-07:137:02	1					7								16																									
C*07:175																																							
C*08:83, C*15:31																																							
C*12:87			3						10																														
C*14:63	1							9	10																														
C*15:51						7		9				12	13																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39

Negative Control

Negative Control