

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
HLA-B allele ^{1,2,3}																																																
*39:01:01:01, 39:01:01:03-39:01:01:07, 39:01:03:01-39:01:03:02, 39:01:06-39:01:08, 39:01:10-39:01:24, 39:65-39:67, 39:69, 39:71, 39:80-39:81, 39:84-39:85, 39:94, 39:100, 39:103, 39:111, 39:114-39:115, 39:117, 39:119, 39:121-39:123, 39:125	1	2			5														19		21		23	24																			43					
*39:01:01:02L	1	2			5														19		21		23	24	25																		43					
*39:01:04	1	2			5														19		21		23	24																				43				
*39:01:05, 39:59 ⁵	1	2			5				9										19		21		23	24																					43			
*39:01:09	1	2																	19		21		23	24																				43				
*39:02:01	1				5	6												18			21		23	24																					43			
*39:02:02:01-39:02:03, 39:101, 39:105, 39:110	1					6												18			21		23	24																				43				
*39:03:01:01-39:03:01:02	1	2			5		7												19				23	24																				43				
*39:04	1	2			5			8											19		21		23																					43				
*39:05:01-39:05:02, 39:113, 39:118	1	2	3		5														19		21		24																					43				
*39:06:01-39:06:02:04, 39:06:04-39:06:05, 39:90, 39:109, 39:127		2			5				9										19				23	24					29															43				
*39:06:03		2							9										19										29																43			
*39:07	1	2	3		5					10		12							19		21		24																						43			
*39:08	1		3		5	6					11							18			21		24																						43			
*39:09:01:01-39:09:03	1	2			5					10						16			19		21		23	24																			42	43				
*39:10:01, 39:96	1													14					19		21																						43					
*39:10:02	1													14						21																							43					
*39:11	1	2	3		5						11								19		21		24																					43				
*39:12	1	2			5										15				19		21		23																					43				
*39:13:01-39:13:02	1		3			6												18			21																								43			
*39:14	1	2			5		7									16			19				23	24																					43			
*39:15	1	2			5							12							19		21		23	24																					43			
*39:16	1													14		16			19																										43			
*39:17	1						7							14					19		21																								43			
*39:18	1	2			5				9		11								19		21		23	24					30																43			
*39:19:01	1	2								10									19		21																								43			
*39:19:02	1	2			5					10									19		21		23	24																					43			
*39:20	1		3											14					19		21																								43			
*39:22	1	2			5													18			21		23	24																				43				
*39:23	1					6												17	18		21																								43			
*39:24:01-39:24:02	1	2			5		7						13						19				23	24																					43			
*39:25N, 39:44 ⁵	1	2			5													17	19		21		23	24																					43			
*39:26, 39:51 ⁵	1	2		4	5														19		21		23	24																				43				
*39:27	1	2			5														19	20		21		23	24																				43			
*39:28	1	2			5								13						19				23	24																						43		
*39:29	1	2			5		7					12				16			19				23	24																							43	
*39:30		2			5														19				23	24						31															43			

Negative Control

*39:31	1	2		5										15			19	21	23	24															43									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
*39:32		2			5													19				23	24			27	28														43			
*39:33		2			5						11								19				23	24			28			31												43		
*39:34		2			5														19				23	24				29		31												43		
*39:35	1	2			5														19	21			23	24					30															
*39:36		2			5						11								19				23	24						31														
*39:37	1	2	3		5		7												19				24																			43		
*39:38Q	1	2			5										15				19	21			23	24			27															43		
*39:39:01	1				5	6													19	21			23	24																		43		
*39:39:02	1				6														19	21																						43		
*39:40:01N-39:40:02N	1	2			5						11								19	21			23	24			28															43		
*39:41, 39:77 ⁵	1	2			5								13						19	21			23	24																		43		
*39:42		2			5								13						19	w			23	24																		43		
*39:43		2	3		5					10									19				23	24																			43	
*39:45	1	2			5													18	19			21	23																			43		
*39:46	1	2			5			8											19	21			23	24																		43		
*39:47		2			5			8											19				23	24																		43		
*39:48	1	2			5														19	21			23	24			27															43		
*39:49	1		3		6							12						18			21																						43	
*39:50		2			5			8											19				23	24				29														43		
*39:52	1				5														19	21			23	24					30															
*39:53	1	2			5						11								19	21			23	24																		43		
*39:54	1				5										15				19	21			23	24																		43		
*39:55	1	2	3		5						12								19	21			24																			43		
*39:56	1	2	3		5														19	21			24			28																43		
*39:57		2			5			9		11									19				23	24				29														43		
*39:58	1			4	5														19	21			23	24																		43		
*39:60	1	2			5														19	21			23	24																				
*39:61	1	2			5						12						17		19	21			23	24																		43		
*39:62, 39:64 ⁵		2			5			9											19				23	24				29	30													43		
*39:63	1												14						19	21						27																43		
*39:68	1	2			5		7												19	21			23	24																		43		
*39:70	1	2			5										16				19	21			23	24																		43		
*39:72	1	2			5														19	21			23																			43		
*39:73		2			5														19			22	23	24																	43			
*39:74		2			5														19				23	24			28														43			
*39:75	1	2			5														19	21			23	24					30													43		
*39:76	1	2			5		7					13							19				23																			43		
*39:78	1	2			5															21			23																			43		
*39:79	1	2			5														19				23	24						32												43		
*39:82, 39:107, 14:53 ⁴		2			5														19				23	24						31														
*39:83								9											19									29														43		

Negative Control

*39:86	1	2			5											19	21	23	24											34							43							
*39:87N, 39:93 ⁵	1	2			5											19	21	23	24										33							43								
*39:88	1				5	6				10						16	19	21	23	24																42	43							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
*39:89	1													14				19		21										32											43			
*39:91	1	2			5					10		12						19		21		23	24																		43			
*39:92	1	2			5													19		21		23											35							43				
*39:95N	1	2			5													19		21		23	24												37					43				
*39:97N	1	2			5													19		21		23	24													38				43				
*39:98	1	2			5													19		21		23	24														39				43			
*39:99	1											12		14				19		21																					43			
*39:102		2			5													19				23	24											36							43			
*39:104, 38:41	1	2																19		21		23	24																		43			
*39:106		2			5													19				23	24																		43			
*39:108	1	2			5													19		21		23	24																41		43			
*39:112		2			5													19				23	24							31										42	43			
*39:116N	1	2			5													19		21		23	24															40			43			
*39:120	1	2			5		7						13					19				23	24																41		43			
*39:124	1	2			5													19		21		w	24																		43			
*39:126	1													14		16		19		21																					43			
*39:128		2			5									15				19				23	24							31											43			
*07:02:01-07:02:31, 07:02:33-07:04, 07:07, 07:09, 07:11-07:12, 07:14-07:15, 07:17-07:26, 07:28-07:31, 07:33:01-07:33:03, 07:35, 07:39, 07:41-07:49N, 07:51-07:52, 07:55-07:64, 07:66-07:68:03, 07:70-07:74, 07:76-07:77, 07:79, 07:82, 07:86-07:89, 07:92-07:96:02, 07:98-07:104, 07:106-07:111N, 07:113-07:122, 07:124-07:131, 07:133-07:136:02, 07:139, 07:141-07:148, 07:150-07:175, 07:177-07:179, 07:184-07:191, 07:193-07:200, 07:202-07:205, 07:208, 07:211-07:212, 07:215-07:218, 07:220-07:221, 07:223-07:227, 07:229-07:235, 07:237-07:245, 07:247-07:248, 07:250-07:257, 07:260-07:261, 07:263, 07:265-07:269, 07:272N, 07:274-07:277, 07:279-07:282, 07:284-07:286, 07:288-07:289, 07:291-07:292, 07:294-07:296, 42:09, 42:16, 42:20													14				19																											
*07:02:32					5									14				19				23	24																					
*07:05:01-07:06:02, 07:34, 07:40, 07:53, 07:80, 07:90, 07:97, 07:105, 07:112, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 07:206, 07:209-07:210, 07:213, 07:222, 07:249, 07:258, 07:262, 07:264, 07:270, 07:278, 07:283, 07:287, 07:293, 08:123, 08:132, 08:146, 42:01:01-42:02:01:02, 42:05:01-42:08, 42:10, 42:12-42:15, 42:17-42:19, 42:21, 42:23-42:24, 81:02													14				19													31														
*07:08:01-07:08:02, 07:10, 07:36, 07:81, 07:83, 07:149, 07:192, 07:236, 07:273, 08:01:23, 08:07, 08:14, 08:28, 08:37, 08:49, 08:74, 08:87, 08:89, 08:107, 08:111, 08:156, 50:15																	19																											

Negative Control

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
*07:16, 07:37, 07:75:01:01-07:75:01:02, 07:91			3											14					19																													
*07:27, 07:50			3			6													19																													
*07:32, 08:01:01:01-08:01:22, 08:01:25-08:02, 08:05, 08:08N, 08:11-08:12:03, 08:15-08:16, 08:18-08:22, 08:24, 08:26:01-08:27, 08:29-08:36, 08:39, 08:41-08:48, 08:50-08:53:02, 08:56:01-08:64, 08:66-08:73, 08:75-08:83, 08:85-08:86N, 08:88, 08:90-08:100, 08:102-08:106, 08:109, 08:112-08:118, 08:120-08:122, 08:124-08:131, 08:134-08:138, 08:140-08:145, 08:149-08:155, 08:157-08:170, 08:172-08:184, 42:22																			19											31																		
*07:38, 07:228:01-07:228:02			3																19																													
*07:54, 07:65, 07:181N, 07:259, 07:271, 07:290, 15:11:01-15:11:07, 15:76, 15:101, 15:143, 15:148, 15:152, 15:209N, 15:215, 15:255, 15:284, 15:305, 15:350, 15:367, 35:76, 44:90, 45:06, 54:06, 54:09, 55:14, 56:03, 56:09, 56:18, 56:32, 56:50, 83:01														14																																		
*07:69, 08:65		2																	19												31																	
*07:78											11			14					19											29	31																	
*07:84														14					19											29																		
*07:85		2																	19																													
*07:123, 54:38, 55:01:07, 55:04, 55:08, 55:49, 55:51, 56:01:05, 56:05:01-56:06, 56:10, 56:12, 56:15, 81:01, 81:03-81:04N, 81:06-81:07														14																		31																
*07:180		2	3																19																													
*07:183																			w																													
*07:207, 08:03, 08:23, 08:133, 08:139			3																19												31																	
*07:214								8			11			14					19												31																	
*07:219, 15:29, 37:67, 57:58, 58:12, C*06:147			3																																													
*07:246											11			14					19																													
*08:01:24				4	5			8							15				19				23								31																	
*08:04:01-08:04:02						6													19												31																	
*08:09, 08:84																			19											29	31																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44				

Negative Control

[illegible]

[illegible]

[illegible]

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
*13:02:17, 13:08, 13:34-13:35, 13:72, 13:89, 40:06:10, 40:44, 40:131, 40:159, 40:307, 40:341																												29															43	
*13:03, 35:06, 35:59:01-35:59:02, 40:71, 44:10, 51:15, 56:33, 56:44, 59:04																											28		31														43	
*13:15																											27	29	31													43		
*13:16, 13:19, 13:54, 13:75, 13:85, 15:42, 15:83, 27:130, 40:95, 40:103, 40:148, 40:161, 40:244, 41:01:01-41:01:05, 41:05-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:25-41:26, 41:28-41:30, 41:32-41:35, 41:37, 41:44, 41:50, 46:18, 51:16, 51:21, 51:34, 51:36, 51:82, 51:143, 51:153, 52:08, 52:37, 52:67, 54:12, 54:24, 55:34, 55:47, 78:07																												29	31															
*13:18											11																	29	31													43		
*13:31, 13:41											11																	29	31															
*13:46, 27:14, 27:81, 41:21, 45:08, 45:10, 49:07, 50:31, 50:45, 51:197, C*05:152, C*07:463, C*12:183																												29																
*13:48, 45:14, 46:11, 56:07-56:08, 56:14, 56:46, 58:73																											28		31															
*13:73											11																			31													43	
*13:86, 35:02:01:01-35:02:05, 35:02:07-35:02:08, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:157, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:273, 35:285, 35:309, 35:311, 35:323, 35:335, 35:339, 40:01:28, 40:305, 53:19, 53:36																														w													43	
*13:96												13																29	31													43		
*14:01:01:01-14:01:05, 14:07N, 14:12, 14:19, 14:26, 14:32, 14:40, 14:47		2			5													19					23	24		26																		
*14:01:06-14:02:02, 14:02:04, 14:02:07-14:02:08, 14:02:10-14:03, 14:09, 14:11, 14:15-14:17, 14:20, 14:22-14:23, 14:25, 14:27, 14:29-14:31, 14:33-14:35, 14:38-14:39, 14:41N, 14:44-14:45, 14:50-14:52		2			5													19					23			26																		
*14:02:03, 14:02:06		2																19							26																			
*14:02:05		2																19					23		26																			
*14:02:09, 14:05-14:06:02		2			5													19					23																					
*14:04		2			5																		23		26																			
*14:08:01-14:08:02, 14:10, 14:55		2			5													19					23	24																				
*14:13		2			5													19					23						31															
*14:14		2	3		5													19						24	26																			
*14:18		2	3		5													19							26																			
*14:21													14					19							26																			
*14:24		2			5													19					23		26								35											
*14:28		2			5			8						15				19					23		26																			
*14:36, 14:48					5													19					23		26																			
*14:37		2			5													19					23					29																
*14:42																		19							26																			
*14:43		2			5													19					23		26																		43	

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44			
*14:46		2			5			8										19					23	24																							
*14:49					5																		23	24		26																					
*14:54					5													19					23	24		26																					
*14:56		2	3		5				10									19					23																					43			
*14:57					5													w					23		26																						
*15:03:01:01-15:03:04, 15:03:06, 15:47:01-15:47:02, 15:54, 15:62, 15:68, 15:98, 15:103, 15:127, 15:131-15:132, 15:156, 15:158, 15:173, 15:210, 15:220, 15:242:01-15:242:02, 15:251, 15:266, 15:274, 15:281-15:282, 15:369, 15:379, 15:389, 15:429, 37:01:01-37:01:11, 37:03N-37:07, 37:09-37:11, 37:13-37:20, 37:23-37:25, 37:27-37:36, 37:38, 37:41-37:50, 37:52-37:56, 37:58-37:59, 37:61-37:66, 37:68, 44:221, 48:08, 48:25, 49:25			3			6												18																													
*15:03:05, 15:64:01-15:64:02, 18:21, 18:56, 18:101, 37:08			3			6																																									
*15:09:01-15:09:02, 15:252, 15:329		2	3																											31																	
*15:10:01-15:10:04, 15:18:01:01-15:18:06, 15:23, 15:37, 15:51, 15:72, 15:80, 15:90, 15:93, 15:99, 15:108, 15:115, 15:119, 15:134, 15:153, 15:161, 15:176, 15:197:01-15:198, 15:200, 15:221, 15:238, 15:263, 15:290, 15:292-15:293, 15:306-15:307, 15:311-15:314, 15:323, 15:337-15:338, 15:351, 15:354, 15:380N, 15:382, 15:388, 15:426-15:427		2	3																																												
*15:32:01-15:32:02, 15:299, C*07:168																																													42		
*15:38:01-15:38:02, 15:364, 15:368, 35:21, 35:188, 51:37, 51:45, 51:216, 53:06, 53:28, 57:14:01-57:14:02, 58:09, 58:76, C*07:516, C*07:521			3						10																																						
*15:49, 15:74, 37:21, 37:26			3															18																													
*15:52, 15:114, 15:124, 15:275:01-15:275:02		2	3																																										43		
*15:58, 15:73, 46:61										11																	28		31																		
*15:61, 15:397, 37:37					6													18																													
*15:69	1		3		6				9	10	12							18			21								30																		
*15:91, 15:123, 15:151, 15:235, 37:02, 37:39, 44:130, 44:156, 44:224, 44:258, 48:02:01, 48:02:03, 48:17, 48:23, 48:26			3		6													18																											43		
*15:133		2	3																								28																				
*15:185, 18:04:01-18:04:02, 18:07:01-18:07:02, 18:29, 18:79-18:80, 18:98, 18:102, 18:107, 18:125, 18:136, 35:24:01-35:24:02, 35:190, 35:287, 53:02			3						10																																					43	
*15:186	1	2	3						9	10	12										21																								43		
*15:189					5			8															23																								
*15:226N, 15:414		2	3																															33													
*15:229			2	3																								28		31																	
*15:243			2	3														18													31																
*15:253			3		6													18										28																			
*15:265									9	10		13																		30																	
*15:294N		2																																													

Negative Control

[illegible]

*37:40			3		6							13				18																		43										
*37:51			3		6											18																	42											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44
*38:01:01-38:01:02, 38:01:04-38:01:07, 38:01:09-38:02:02, 38:02:04-38:02:07, 38:09, 38:11-38:12, 38:17-38:18, 38:21, 38:24, 38:27, 38:31, 38:34N-38:35, 38:38-38:40, 38:42-38:44, 38:47-38:49, 38:51, 38:53, 38:55Q, 38:57, 38:59-38:62, 38:64, 38:66-38:68Q	1	2	3															19		21			24																				43	
*38:01:03	1	2	3					9										19		21			24																				43	
*38:01:08, 38:02:03, 38:05	1	2	3															19		21																								43
*38:03	1				6													18			21			24																				43
*38:04, 38:25	1	2	3															18			21																						43	
*38:06	1		3															19		21																								43
*38:07	1		3															19		21			24																				43	
*38:08		2	3															19		w			24																					43
*38:10		2	3															19					24						31															43
*38:13	1	2																19		21			24																				43	
*38:14	1	2																19		21		w	24																				43	
*38:15		2	3															19		21			24																				43	
*38:16, 38:28, 38:46	1																	19		21			24																				43	
*38:19	1	2	3			7												19					24																				43	
*38:20		2	3										13					19					24																				43	
*38:22	1	2	3															19					24																				43	
*38:23		2	3															19			22		24																			43		
*38:26	1																	19		21		w	24																			43		
*38:29	1	2	3									12						19		21			24																				43	
*38:30		2	3															19					24					28		31													43	
*38:32	1	2	3															19		21			24						30														43	
*38:33	1	2	3					8										19		21				24																			43	
*38:36, 38:50		2	3															19					24							31													43	
*38:37, 38:45, 38:52		2	3															19					24																				43	
*38:54	1		3															19		21			24						30														43	
*38:56	1	2	3																	21			24																				43	
*38:58	1	2	3															19		21			24									33											43	
*38:63		2	3					8										19					24								31												43	
*38:65		2	3															19					24					28															43	
*40:12, 40:149, 48:01:01-48:01:06, 48:03:01-48:04:02, 48:09-48:14, 48:18, 48:20-48:22, 48:24, 48:27-48:30, 48:32-48:38, 48:40-48:41			3		6													18													31													43
*40:25, 40:68, 40:106, 40:232																		?													31												43	
*40:36, 40:77, 40:314								8																							31												43	
*40:61													14																		31												43	
*40:86								8																					29		31												43	
*40:166													14					?													31													

Negative Control

[illegible]

[illegible]