

HLA-B39 2018 2H5

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	
HLA-B allele ^{1,2,3}																																								
*39:01:01:01, 39:01:01:03-39:01:01:09, 39:01:03:01-39:01:03:02, 39:01:06-39:01:08, 39:01:10-39:01:24, 39:65-39:67, 39:69, 39:71, 39:80-39:81, 39:84-39:85, 39:94, 39:100, 39:103, 39:111, 39:114-39:115, 39:117, 39:119, 39:121-39:123, 39:125, 39:130, 39:137-39:138	1	2			5														19		21		23	24																
*39:01:01:02L	1	2			5														19		21		23	24	25															
*39:01:04	1	2			5														19		21			24																
*39:01:05, 39:59 ⁵	1	2			5				9										19		21		23	24																
*39:01:09	1	2																	19		21																			
*39:02:01	1				5	6													18			21		23	24															
*39:02:02:01-39:02:03, 39:101, 39:105, 39:110	1					6													18			21																		
*39:03:01:01-39:03:01:02	1	2			5		7												19				23	24																
*39:04	1	2			5			8											19		21		23																	
*39:05:01:01-39:05:03, 39:113, 39:118	1	2	3		5														19		21			24																
*39:06:01-39:06:02:04, 39:06:04-39:06:06, 39:90, 39:109, 39:127, 39:132, 39:140-39:141		2			5				9										19				23	24						29										
*39:06:03		2							9										19											29										
*39:07	1	2	3		5					10		12							19		21			24																
*39:08	1		3		5	6					11								18			21		24																
*39:09:01:01-39:09:03	1	2			5					10						16			19		21		23	24																
*39:10:01, 39:96, 39:131	1												14						19		21																			
*39:10:02	1												14								21																			

HLA-B39 2018 2H5

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39			
*39:32		2			5														19				23	24			27	28														
*39:33		2			5						11								19				23	24				28			31											
*39:34		2			5														19				23	24					29		31											
*39:35	1	2			5														19		21		23	24						30												
*39:36		2			5						11								19				23	24							31											
*39:37	1	2	3		5		7												19				24																			
*39:38Q	1	2			5										15				19		21		23	24			27															
*39:39:01	1				5	6													19		21		23	24																		
*39:39:02	1				6														19		21																					
*39:40:01N-39:40:02N	1	2			5						11								19		21		23	24				28														
*39:41, 39:77 ⁵	1	2			5								13						19		21		23	24																		
*39:42		2			5								13						19		w		23	24																		
*39:43		2	3		5					10									19				23	24																		
*39:45	1	2			5													18	19		21		23																			
*39:46	1	2			5			8											19		21		23	24																		
*39:47		2			5			8											19				23	24																		
*39:48	1	2			5														19		21		23	24			27															
*39:49	1		3		6						12							18			21																					
*39:50		2			5			8											19				23	24					29													
*39:52	1				5														19		21		23	24						30												
*39:53	1	2			5						11								19		21		23	24																		
*39:54	1				5										15				19		21		23	24																		
*39:55	1	2	3		5						12								19		21		24																			
*39:56	1	2	3		5														19		21		24				28															
*39:57		2			5				9		11								19				23	24					29													
*39:58	1			4	5														19		21		23	24																		
*39:60	1	2			5														19		21		23	24																		
*39:61	1	2			5						12						17		19		21		23	24																		
*39:62, 39:64 ⁵		2			5				9										19				23	24					29	30												
*39:63	1													14					19		21						27															
*39:68	1	2			5		7												19		21		23	24																		
*39:70	1	2			5										16				19		21		23	24																		
*39:72	1	2			5														19		21		23																			
*39:73		2			5														19			22	23	24																		
*39:74		2			5														19				23	24				28														
*39:75	1	2			5														19		21		23	24						30												
*39:76	1	2			5		7						13						19				23																			
*39:78	1	2			5																21		23																			
*39:79	1	2			5														19				23	24							32											
*39:82, 39:107, 39:136, 14:53 ⁴		2			5														19				23	24							31											
*39:83									9										19										29													
*39:86	1	2			5														19		21		23	24										34								

HLA-B39 2018 2H5

*39:87N, 39:93 ⁵	1	2			5												19	21	23	24																			
*39:88	1				5	6				10						16	19	21	23	24																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39
*39:89	1																19	21																					
*39:91	1	2			5					10		12						19	21		23	24																	
*39:92	1	2			5													19	21		23													35					
*39:95N	1	2			5													19	21		23	24														37			
*39:97N	1	2			5													19	21		23	24															38		
*39:98	1	2			5													19	21		23	24															39		
*39:99	1											12		14				19	21																				
*39:102		2			5													19			23	24														36			
*39:104, 38:41 ⁴	1	2																19	21		23	24																	
*39:106		2			5													19			23	24																	
*39:108	1	2			5													19	21		23	24																	
*39:112		2			5													19			23	24									31								
*39:116N	1	2			5													19	21		23	24																	
*39:120	1	2			5		7						13					19			23	24																	
*39:124	1	2			5													19	21		w	24																	
*39:126	1													14		16		19	21																				
*39:128		2			5									15				19			23	24									31								
*39:129		2			5				9									19			23							29											
*39:133N	1	2			5													19	21		23								30			33							
*39:135	1				5													19	21		23	24																	
*39:139N	1	2			5													19	21		23	24																	
*07:02:01:01-07:02:31, 07:02:33-07:04:02, 07:07, 07:09:01-07:09:02, 07:11-07:12, 07:14-07:15, 07:17-07:26, 07:28-07:31, 07:33:01-07:33:03, 07:35, 07:39, 07:41-07:49N, 07:51-07:52, 07:55-07:64, 07:66-07:68:03, 07:70-07:74, 07:76-07:77, 07:79, 07:82, 07:86-07:89, 07:92-07:96:02, 07:98-07:104, 07:106-07:111N, 07:113-07:122, 07:124-07:131, 07:133-07:136:02, 07:139:01-07:139:02, 07:141-07:148, 07:150-07:175, 07:177-07:179, 07:184-07:191, 07:193-07:200, 07:202-07:205, 07:208, 07:211-07:212, 07:215-07:218, 07:220-07:221, 07:223-07:227, 07:229-07:235, 07:237-07:245, 07:247-07:248, 07:250-07:257, 07:260-07:261, 07:263, 07:265-07:269, 07:272N, 07:274-07:277, 07:279-07:282, 07:284-07:286, 07:288-07:289, 07:291-07:292, 07:294-07:303, 07:305, 07:307-07:316N, 07:318N-07:323, 07:325N-07:326, 07:328-07:331, 07:334-07:335, 42:09, 42:16, 42:20													14				19																						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39
*07:02:32					5									14				19			23	24																	
*07:05:01:01-07:06:03, 07:34, 07:40, 07:53, 07:80, 07:90, 07:97, 07:105, 07:112, 07:137-07:138, 07:140, 07:176, 07:182N, 07:201N, 07:206, 07:209-07:210, 07:213, 07:222, 07:249, 07:258, 07:262, 07:264, 07:270, 07:278, 07:283, 07:287, 07:293, 07:304, 07:317, 07:324, 07:332-07:333, 08:123, 08:132, 08:146, 42:01:01-42:02:01:02, 42:05:01-42:08, 42:10, 42:12-42:15, 42:17-42:19, 42:21, 42:23-42:25, 81:02													14					19												31									

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

HLA-B39 2018 2H5

[illegible]

			43			Negative Control
40	41	42	43	44	45	
40	41	42	43	44	45	

[illegible]

				43		Negative Control
				43		
				43		
				43		
				43		
				43		
				43		
				43		
				43		
				43		
40	41	42	43	44	45	
40	41	42	43	44	45	
</						

