

INTERPRETATION TABLE																																					
HLA-B*40 SSP subtyping																																					
Amplification patterns of the B*40:01 to B*40:99 alleles																																					
	Well ³																																				
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	
Length of spec.	175	215	130	95	190	95	105	235	160	215	85	95	90	90	150	75	120	105	320	135	145	235	95	165	125	120	180	335	235	190	225	255	245	235	140	135	
PCR product																160				285																	
Length of int. pos. control ¹	800	800	1070	800	800	1070	1070	1070	1070	1070	1070	1070	1070	800	800	800	800	800	1070	1070	800	800	800	1070	1070	1070	800	1070	800	1070	1070	1070	1070	800	1070		
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3		
HLA-B allele ²	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	
*40:01:01-40:01:05, 40:01:07-40:01:10	+	+															+											+		+		+					
*40:01:06	+	+															+	+										+		+		+					
*40:02:01-40:02:03, 40:02:05-40:02:10			+	+	+																		+							+		+					
*40:02:04			+	+	+																		+							+		+					
*40:03		+	+			+	+						+										+							+			+				
*40:04		+	+																+									+		+		+	+				
*40:05		+	+	+				+																								+					
*40:06:01:01-40:06:02		+	+						+																						+		+				
*40:07	+									+								+											+		+		+				
*40:08				+	+					+		+											+							+		+		+			
*40:09			+	+	+								+									+										+					
*40:10:01	+												+				+												+		+					+	
*40:11:01		+	+																									+		+		+					
*40:12	+						+											+										+		+		+					
*40:13			+	+						+						+							+							+		+					
*40:14:01		+														+	+			+								+		+		+					
*40:14:02-40:14:03		+															+			+								+		+		+					
*40:15		+		+												+							+				+			+		+					
*40:16		+		+																		+				+				+		+					
*40:18		+	+	+	+								+									+										+					
*40:19		+	+	+										+								+								+							
*40:20		+	+		+																											+					
*40:21	+																	+				+						+		+		+					
*40:22N	+	+																+									+	+		+		+					
*40:23	+	+																		+						+		+		+		+					
*40:24		+	+	+	+								+							+			+									+					
*40:25	+	+									+							+										+		+		+					
*40:26		+	+															+										+				+					
*40:27:01		+	+	+		+													+				+							+		+					
*40:28		+	+																+									+				+					
*40:29		+	+	+																		+	+							+		+					
*40:30		+																+												+		+		+			
*40:31		+		+	+								+					+		+		+										+					
*40:32		+		+	+								+							+		+			+							+					
*40:33	+	+			+													+										+				+					
*40:34		+																+							+							+		+			
*40:35		+	+	+																		+								+		+			+		
*40:36	+	+																+										+		+		+					
*40:37		+	+	+																		+								+		+					
*40:38	+	+																+										+		+	+	+					
*40:39		+	+	+																									+			+					
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3		
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3	
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	
*40:40			W	+																		+							+							+	

Well No.										1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6
*08:09, 08:60, 39:01:07, 39:16, 42:04, 51:01:01-51:01:08, 51:01:10-51:02:02, 51:02:04-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:12, 51:14, 51:19-51:24:01, 51:26, 51:28-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61, 51:63, 51:65-51:74, 51:76-51:80, 51:83, 51:86-51:89, 51:91, 51:95, 52:01:01:01-52:07, 52:09-52:11, 52:13, 52:15, 52:17-52:20, 55:01:06, 55:17, 55:38, 56:05:01-56:06, 56:21, 56:27, 57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:28N, 58:08, 78:01-78:07, C*15:24																																				
*08:11				+																																
*08:12:01, 08:12:03, 08:16, 35:12:01-35:12:02, 35:44, 35:59, 38:01:01, 38:01:03-38:02:02, 38:04-38:11, 38:13-38:18, 38:20-38:21, 39:01:01:01-39:01:01:02L, 39:01:03-39:01:06, 39:01:08-39:01:11, 39:04-39:05:02, 39:09-39:12, 39:18-39:20, 39:22, 39:26-39:28, 39:30, 39:32, 39:35-39:36, 39:38Q, 39:40N-39:42, 39:45-39:46, 39:48, 39:51-39:54, 39:56, 42:13, 51:62, 55:08, 56:04, 56:12, 58:20, 67:01:01-67:02																											+									
*08:12:02, 08:49, 13:10, 14:01:01-14:04, 14:06:01-14:09, 14:11-14:12, 14:14-14:18, 15:13, 15:17:01:01-15:17:02, 15:24, 15:36, 15:43, 15:87, 15:89, 15:157, 15:162, 15:168, 15:177, 15:180, 18:04, 18:25, 18:43, 35:32, 35:39, 35:43:01, 35:61, 35:67, 35:79, 35:99, 38:01:02, 39:07, 39:15, 39:17, 39:25N, 39:43-39:44, 39:55, 44:06, 44:08, 44:57, 44:83, 46:04, 51:42, 51:45, 53:07, 54:06, 54:09, 56:03, 56:18, 57:11, 58:10N, 83:01																											+									
*08:21, 08:25				+				+																												
*08:28, 08:37						+						+																	+							
*08:35, 42:06						+																							+							
*13:01:01-13:01:02, 13:07N, 13:13, 13:17, 13:20, 13:25, 13:28-13:29, 13:36																			+									+		+		+				
*13:01:03																			+									+				+				
*13:02:01-13:02:07, 13:08Q, 13:14-13:16, 13:19, 13:27, 13:30, 13:32-13:33, 13:37																			+											+						
Well No.	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	3	3	
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6

Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6
*13:03, 35:60, 51:15, 54:01:01, 54:03-54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:17-54:21, 55:01:01-55:01:05, 55:02:01-55:03, 55:05, 55:07, 55:10-55:12, 55:15, 55:19-55:21, 55:25-55:26, 55:28-55:34, 55:36-55:37, 55:39-55:40, 56:01:01-56:01:04, 56:07-56:08, 56:13-56:17, 56:19N-56:20, 56:23-56:26, 56:28N-56:29, 59:01:01-59:02, 59:04-59:05																			+																	
*13:04, 15:16, 15:67, 15:95, 35:37, 54:14-54:16, 55:23, 55:27, 58:14																			+																	
*13:06, 13:12, 35:01:01-35:01:09, 35:01:11-35:01:19, 35:03:01-35:03:04, 35:07-35:08:04, 35:10-35:11:01, 35:13-35:14:02, 35:20:01-35:20:02, 35:23, 35:25-35:29, 35:34-35:36, 35:38, 35:40N-35:42:02, 35:45, 35:48-35:49, 35:52-35:57, 35:62, 35:64-35:65Q, 35:68:01-35:71, 35:74-35:78, 35:80, 35:82, 35:85, 35:90-35:94, 35:98, 53:01:01-53:01:05, 53:03-53:05, 53:08:01-53:13, 53:15-53:18, 53:20-53:23, 56:09, 56:11, 57:05, 58:01:01-58:02, 58:04, 58:06-58:07, 58:11-58:13, 58:15-58:17N, 58:19, 58:21-58:26																			+								+									
*13:09																			+		+									+						
*13:11																			+									+		+						
*13:18, 13:31, 55:09, 55:22, 55:24																			+										+							
*13:21																												+		+			+			
*13:22:01-13:22:02, 35:88																			+									+		+			+			
*13:23		+																	+									+		+			+			
*13:26, 35:15, 35:33					+														+									+					+			
*13:34																			+																	
*13:35					+																															
*14:05						+																							+					+		
*14:10																												+								
*14:13, 39:14, 42:10				+																									+					+		
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6
Well No.										1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6

[illegible]

[illegible]

[illegible]

3	3	3	4	4	4	4	4	4	4	4	4	4	5	5	5	5	5	5	5	5	5	5	6	6	6	6	6	6	6	6	6	7	7	7	Well No.
7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2
																																			*08:09, 08:60, 39:01:07, 39:16, 42:04, 51:01:01-51:01:08, 51:01:10-51:02:02, 51:02:04-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:12, 51:14, 51:19-51:24:01, 51:26, 51:28-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61, 51:63, 51:65-51:74, 51:76-51:80, 51:83, 51:86-51:89, 51:91, 51:95, 52:01:01:01-52:07, 52:09-52:11, 52:13, 52:15, 52:17-52:20, 55:01:06, 55:17, 55:38, 56:05:01-56:06, 56:21, 56:27, 57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:28N, 58:08, 78:01-78:07, C*15:24
																																			*08:11
																																			*08:12:01, 08:12:03, 08:16, 35:12:01-35:12:02, 35:44, 35:59, 38:01:01, 38:01:03-38:02:02, 38:04-38:11, 38:13-38:18, 38:20-38:21, 39:01:01:01-39:01:01:02L, 39:01:03-39:01:06, 39:01:08-39:01:11, 39:04-39:05:02, 39:09-39:12, 39:18-39:20, 39:22, 39:26-39:28, 39:30, 39:32, 39:35-39:36, 39:38Q, 39:40N-39:42, 39:45-39:46, 39:48, 39:51-39:54, 39:56, 42:13, 51:62, 55:08, 56:04, 56:12, 58:20, 67:01:01-67:02
																																			*08:12:02, 08:49, 13:10, 14:01:01-14:04, 14:06:01-14:09, 14:11-14:12, 14:14-14:18, 15:13, 15:17:01:01-15:17:02, 15:24, 15:36, 15:43, 15:87, 15:89, 15:157, 15:162, 15:168, 15:177, 15:180, 18:04, 18:25, 18:43, 35:32, 35:39, 35:43:01, 35:61, 35:67, 35:79, 35:99, 38:01:02, 39:07, 39:15, 39:17, 39:25N, 39:43-39:44, 39:55, 44:06, 44:08, 44:57, 44:83, 46:04, 51:42, 51:45, 53:07, 54:06, 54:09, 56:03, 56:18, 57:11, 58:10N, 83:01
																																			*08:21, 08:25
																																			*08:28, 08:37
																																			*08:35, 42:06
																																			*13:01:01-13:01:02, 13:07N, 13:13, 13:17, 13:20, 13:25, 13:28-13:29, 13:36
																																			*13:01:03
																																			*13:02:01-13:02:07, 13:08Q, 13:14-13:16, 13:19, 13:27, 13:30, 13:32-13:33, 13:37
3	3	3	4	4	4	4	4	4	4	4	4	4	5	5	5	5	5	5	5	5	5	5	6	6	6	6	6	6	6	6	6	7	7	7	Well No.
7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]