

INTERPRETATION TABLE

HLA-B*40 SSP subtyping

Amplification patterns of the B*40:01 to B*40:215 alleles

	Well ⁶																																																		
	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48			
	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9		
Length of spec.	175	215	130	95	190	95	105	235	160	215	85	95	90	90	150	160	75	120	105	320	135	145	235	95	165	125	125	120	180	335	235	190	225	255	245	190	140	135	125	120	330	120	205	95	185						
PCR product																																																			
Length of int. pos. control ¹	800	800	1070	800	800	1070	800	1070	1070	1070	1070	1070	800	800	800	160	800	800	800	1070	1070	800	800	800	1070	1070	1070	800	1070	800	1070	1070	1070	1070	1070	235	1070	800	1070	1070	1070	800	1070	1070	1070	1070	1070	1070			
Well No.										1	1	1	1	1	1	1	1	1	1	1	2	2	2	2	2	2	2	2	2	2	3	3	3	3	3	3	3	3	3	3	3	3	3	3	3	3	3	3	3		
HLA-B allele ^{2,3}	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9		
*40:01:01-40:01:05, 40:01:07-40:01:10, 40:01:12-40:01:25, 40:100-40:101, 40:108, 40:112, 40:123, 40:126, 40:134, 40:138, 40:140, 40:146-40:147, 40:152, 40:154, 40:156, 40:170-40:171, 40:175, 40:182, 40:185-40:186, 40:193-40:196, 40:204, 40:207-40:208, 40:212, 40:215	+	+																+											+		+		+																		
*40:01:06	+	+																+	+										+		+		+																		
*40:01:11	+	+																+											+		+		+																+		
*40:02:01-40:02:03, 40:02:05-40:02:15, 40:104, 40:143, 40:169, 40:173, 40:181, 40:189, 40:202-40:203, 40:205-40:206, 40:211, 40:214		+	+	+																					+						+		+																		
*40:02:04		+	+	+																				+							+		+																		
*40:03		+	+			+	+						+											+										+		+															
*40:04		+	+																	+														+		+		+	+												
*40:05, 40:174		+	+	+				+																					+		+		+		+	+															
*40:06:01:01-40:06:04, 40:165, 40:167		+	+						+																								+		+																
*40:06:05		+	+																														+		+																
*40:07	+									+									+												+		+		+																
*40:08			+	+						+		+																						+		+		+										+			
*40:09		+	+	+									+											+										+		+															
*40:10:01	+												+						+												+		+													+					
*40:10:02	+												+						+														+		+											+					
*40:11:01		+	+																												+		+		+																
*40:11:02		+	+																												+		+		+																
*40:12	+						+												+												+		+													+					
*40:13			+	+						+					+									+									+		+																
*40:14:01, 40:121, 40:136 ⁵		+													+	+		+				+								+		+		+																	
*40:14:02-40:14:03		+													+			+				+									+		+		+																
*40:15		+		+											+							+			+								+		+																
*40:16		+		+																	+			+					+				+		+																
*40:18		+	+	+	+								+										+											+		+															
*40:19		+	+	+											+								+											+		+															
*40:20		+	+	+	+																		+										+		+																
*40:21	+																		+				+										+		+																
*40:22N	+	+																	+				+										+		+		+														
*40:23	+	+																					+						+		+		+		+		+														
*40:24		+	+	+	+								+																+					+		+															
*40:25	+	+									+								+				+										+		+		+												+		
*40:26		+	+																														+		+		+														
*40:27:01		+	+	+		+																	+										+		+		+														
*40:27:02				+		+																	+										+		+		+														
*40:28		+	+																																																

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]