

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
HLA-B allele ^{1,2,3}																				
*41:01:01-41:01:02, 41:14, 41:20-41:21, 41:25-41:26, 41:30 ⁵	1					6		8		10	11									
*41:02:01-41:02:05, 41:18, 41:31		2						8		10		12								
*41:03:01			3					8		10		12								
*41:03:02								8		10		12								
*41:04		2		4				8		10		12								
*41:05	1				5	6		8		10	11									
*41:06						6	7	8			11									
*41:07, 41:12 ⁴	1			4		6		8		10	11									
*41:08, 40:136, 40:231, 40:279 ³								8	9			12								
*41:09, 45:08 ³						6	7	8			11		13							
*41:10		2	3					8		10		12								
*41:11		2						8		10		12							19	
*41:13		2						8		10		12		14						
*41:15		2						8				12				16				
*41:16	1					6		8		10	11			14						
*41:17	1					6		8		10										
*41:19		2						8		10		12						17		
*41:22	1		3			6		8		10	11									
*41:23		2			5					10		12								
*41:24		2						8		10		12							18	
*41:27		2			5			8		10		12								
*41:28	1					6		8		10	11								19	
*41:29, 41:32						6		8			11									
*41:33, 08:09, 08:84, 42:04 ³	1					6				10	11									
*07:04, 07:25, 07:146, 42:05:01-42:05:02		2							9			12								
*07:19, 08:01:21, 08:01:25, 08:31, 08:43, 08:103, 08:113		2										12								
*07:33:01, 07:53, 07:60, 07:139, 13:71, 27:07:01-27:07:04, 27:34, 27:43, 35:31, 35:114, 40:02:03, 40:08, 40:13, 40:27:02, 40:40, 40:58, 40:164, 40:200, 40:219-40:220, 40:226, 40:229, 40:232, 48:04, 48:13, 48:24, 51:81, 51:148, 55:04, 55:49, 55:51, 58:27, C*02:60, C*03:05, C*03:25, C*03:27, C*03:167, C*08:08:01-08:08:02																16				
*07:68:01-07:68:03, 35:66, 48:12, 48:14, 48:29																			18	
*07:100, 14:05, 14:13, 15:07:01-15:07:03, 15:45, 15:55, 15:68, 15:126, 15:207, 15:324, 15:329, 27:32, 35:05:01-35:05:03, 35:22, 35:51, 35:58, 35:72, 35:89, 35:97, 35:199, 35:232, 38:19, 39:03, 39:14, 39:24:01-39:24:02, 39:29, 39:37, 39:76, 44:54, 44:135, 44:184, 46:12, 51:64, 53:14, 58:18																16			18	
*07:143, 39:31			3		5															
*08:01:01-08:01:20, 08:01:22-08:01:24, 08:01:26-08:05, 08:07-08:08N, 08:10-08:11, 08:14-08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:26:02, 08:28-08:30N, 08:32, 08:34-08:39, 08:41-08:42, 08:44-08:48, 08:50-08:54, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:72N-08:75, 08:77-08:78, 08:80-08:83, 08:85-08:88, 08:90-08:95, 08:97-08:102, 08:105-08:106, 08:108-08:112, 08:114-08:115, 08:117-08:128, 08:130-08:133, 35:87, 42:01:01-42:01:03, 42:06-42:07, 42:11-42:12, 42:15-42:16, 42:20, 44:166		2								10		12								
*08:12:01-08:12:02, 08:16, 08:89										10		12								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*08:12:03, 08:49, 08:60, 08:129, 42:13										10										
*08:27, 08:33, 08:76, 08:104		2								10										
*08:40, 27:83																	17			
*08:62, 42:02, 42:09, 42:18		2	3		5					10		12								
*08:71, 42:10		2								10		12							18	
*08:96		2		4						10		12								
*08:107				4						10										
*08:116, 08:134, 42:08, 42:14		2										12				16				

Negative Control

[illegible]

[illegible]