

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*41:01:01:01-41:01:07, 41:20, 41:25-41:26, 41:30, 41:34-41:35, 41:37, 41:44, 41:50, 41:53-41:54, 41:57, 41:59-41:61	1					6		8		10	11										21	22		
*41:02:01:01-41:02:03, 41:02:05-41:02:09, 41:18, 41:31, 41:38-41:42, 41:51-41:52, 41:55, 41:58		2						8		10		12										22		
*41:02:04		2						8		10		12												
*41:03:01			3					8		10		12									20		22	
*41:03:02								8		10		12								20			22	
*41:04, 41:36 ⁴		2		4				8		10		12											22	
*41:05	1				5	6		8		10	11										21	22		
*41:06					6	7	8			11												21	22	
*41:07	1			4	6		8		10	11											21	22		
*41:08, 40:136, 40:231, 40:279 ⁵							8	9			12								20			22		
*41:09					6	7	8			11			13								21	22		
*41:10, 41:45N ⁴		2	3					8		10		12										22		
*41:11		2						8		10		12							19				22	
*41:12	1			4	6		8		10	11											21			
*41:13		2						8		10		12		14									22	
*41:14	1				6		8		10	11				15							21	22		
*41:15		2						8			12				16								22	
*41:16	1				6		8		10	11			14								21	22		
*41:17, 41:56	1				6		8		10												21	22		
*41:19		2						8		10		12					17					22		
*41:21	1				6		8		10	11												22		
*41:22	1		3		6		8		10	11											21	22		
*41:23, 42:27 ⁵		2			5					10		12												
*41:24		2						8		10		12					18					22		
*41:27		2			5			8		10		12										22		
*41:28	1				6		8		10	11								19			21	22		
*41:29					6		8			11				15							21	22		
*41:32					6		8			11											21	22		
*41:33	1				6				10	11											21	22		
*41:43		2						8		10		12										22	23	
*41:46-41:47, 44:166 ⁵		2								10		12										22		
*41:48								8		10												22		
*41:49		2						8		10		12		14				17				22		
*41:62, 40:02:20 ⁵		2						8							16							22		
*41:63								8		10		12										22		
*07:02:13, 07:02:35, 07:05:10, 07:22:02, 07:151:02, 07:209, 08:13, 08:20:01-08:20:02, 08:25, 08:55, 08:70, 08:143, 42:19, C*07:294, C*07:526:01, C*07:530, C*07:681		2																						
*07:04:01-07:04:02, 07:25, 07:146, 07:349, 08:187, 42:05:01-42:05:02		2							9			12												
*07:19, 08:01:21, 08:01:25, 08:31, 08:43, 08:103, 08:113, 08:169		2										12												
*07:33:01, 07:53, 07:60, 07:139:01, 13:71, 15:421, 15:524, 27:07:01-27:07:06, 27:34, 27:43, 27:219, 35:31, 35:114, 40:02:03, 40:08, 40:27:02, 40:200, 40:220, 40:226, 40:232, 40:322, 48:04:01-48:04:02, 48:13, 48:24, 51:81, 58:27, C*02:60, C*03:05, C*03:25, C*03:27, C*03:167, C*03:296:02, C*03:335, C*03:386, C*03:407, C*03:482, C*03:494, C*06:264, C*08:08:01-08:08:02															16									
*07:68:01, 07:68:03, 07:237, 48:14, 48:29																		18						
*07:68:02, 07:262, 35:66, 48:12		2																18						
*07:100, 15:07:01-15:07:02, 15:45, 15:55, 15:68, 15:126, 15:197:02, 15:207, 15:324, 15:329, 15:331, 15:360, 15:405, 15:431, 15:450, 27:32, 27:180, 46:12															16			18						
*07:143			3		5																			
*07:335, 15:426, 18:07:01-18:07:02, 18:33, 18:79, 18:136, 18:155, 18:182N, 27:75, 35:162, 37:21, 37:26, 37:57, 37:60, 37:67, 37:79N, C*06:77			3																					
*07:388, 27:83																	17							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]

*40:366							7	8					13						21	22				
*42:17		2	3		5					10		12					18							
*44:02:33, 44:41:01, 44:230												12								22				
*44:15:01:01-44:15:01:02, 44:18, 45:01:01:01-45:04, 45:07, 45:09-45:13, 45:15-45:25						6	7	8					13							22				
*44:20, 44:100						6	7						13							22				
*44:34:02, 44:245, 53:30				4																22				
*44:54, 44:135, 44:184		2													16		18			22				
*44:80, 44:219																	19			22				
*44:106, 44:158, 44:213		2										12			16		18			22				
*44:342										10										22				
*45:05						6		8					13							22				
*45:06					5	6	7						13											
*45:08						6	7	8			11		13							22				
*45:14						6	7	8					13						21	22				
*49:01:12, 50:11								8					13							22				
*49:14, 49:52							7	8												22				
*49:48				4			7	8					13							22				
*50:45, 50:49							7	8			11		13							22				
*50:52					5		7	8					13							22				
*51:08:01:01-51:08:04, 51:20, 51:44N, 51:97, 51:141, 51:263, 51:274-51:275, 52:19, 56:13						6	7						13											
*51:116				3			7						13											
*51:148, 55:04, 55:49, 55:51, 55:86, C*03:296:01		2													16									
*54:02, 55:16					5		7				11		13											
*55:20	1			4		6				10	11													
*55:67, C*07:463, C*07:758											11		13											
*56:67				4			7						13											
C*07:714, C*07:724			3		w																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control