

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele^{1,2,3}																
*45:01:01-45:01:02	1	2	3													
*45:02	1	2			5											
*45:03	1	2		4												
*45:04		2	3			6										
*45:05	1	2	3				7									
*45:06	1	2	3					8								
*45:07	1	2	3	4												
*45:08		2	3			6			9							
*45:09	1	2								10					15	
*45:10		2	3			6					11					
*45:11	1	2	3									12				
*45:12	1	2	3										13			
*45:13	1	2	3											14		
*45:14	1	2	3												15	
*07:20, 07:140, 07:153, 15:14, 15:91, 15:131, 15:161, 35:45, 35:71, 35:169, 44:10, 44:14, 44:31, 44:43:01-44:43:02, 44:45, 44:51, 44:62, 44:77, 44:82, 44:107, 44:123, 44:135, 44:184, 46:17, 57:07, 58:07																15
*07:78, 07:84, 13:02:01-13:02:16, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30-13:35, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:47, 13:49N, 13:54-13:56N, 13:58-13:59, 13:65-13:70, 13:72, 13:74-13:75, 27:14, 27:81, 39:50, 40:93, 40:96, 40:109-40:110, 49:07, 51:10, 51:16, 51:31, 51:34, 51:82, 51:93, 52:08, 54:26, 55:09, 55:22, 55:24, 73:01-73:02						6					11					
*07:133, 40:166, 44:90		2						8								
*08:09, 08:84, 13:46, 13:53, 14:37, 15:42, 15:83, 35:60, 39:06:01-39:06:04, 39:34, 39:57, 39:62, 39:64, 39:83, 39:90, 41:05, 41:26, 42:04, 46:18, 51:21, 51:36, 51:101, 51:143, 51:153, 51:157, 52:37, 54:01:01-54:02, 54:04-54:05N, 54:07-54:08N, 54:10-54:25, 54:27-54:32, 55:01:01-55:01:06, 55:01:08-55:03, 55:05, 55:07, 55:10-55:13, 55:15-55:21, 55:23, 55:25-55:48, 55:50, 55:52-55:55N, 55:57-55:62, 55:64, 56:23, 59:01:01-59:03, 59:05-59:08, 78:07						6			9							
*08:28, 08:35, 08:37, 08:89, 08:107, 15:51, 15:179:01-15:179:02, 15:199, 15:218Q, 35:115, 37:35, 42:09, 57:24, C*07:68, C*07:140, C*07:151, C*07:302										10						
*13:03-13:04, 13:48, 13:62, 15:04:01-15:04:02, 15:16:01-15:16:03, 15:67, 15:95, 15:137, 15:155, 15:222, 15:254, 15:293, 15:303, 15:310, 35:37, 35:235, 39:33, 46:11, 46:32, 49:01:01-49:06, 49:08-49:19N, 49:21-49:31, 50:06, 51:01:01-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:15, 51:17-51:20, 51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:37-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63, 51:65-51:69, 51:71-51:80, 51:83-51:92, 51:94-51:96, 51:98N-51:100, 51:102-51:107, 51:109-51:134, 51:136-51:138, 51:140-51:142, 51:144-51:147, 51:149N-51:152, 51:154-51:156, 51:158-51:173Q, 52:01:01-52:07, 52:09-52:14, 52:16-52:36, 52:38-52:39, 54:03, 56:01:01-56:01:07, 56:05:01-56:08, 56:14-56:17, 56:19N-56:22, 56:24-56:30, 56:33-56:43, 58:08:01-58:08:02, 59:04, 78:01:01-78:06						6										
*15:46, 15:53, 15:106, 15:143, 15:212, 18:48, 35:19, 35:47, 35:63, 35:217, 39:69, 40:01:01-40:05, 40:07-40:11:02, 40:14:01-40:16, 40:18, 40:20, 40:22N-40:40, 40:42-40:43, 40:45, 40:48-40:52, 40:54-40:69, 40:71-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:108, 40:111-40:116, 40:118N-40:126, 40:128-40:130:02, 40:133Q-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:160:01-40:160:02, 40:163-40:164, 40:168-40:176, 40:179-40:184, 40:186-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:210-40:229, 40:231-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:285, 41:02:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23, 41:27, 41:31, 44:09, 44:46, 44:75, 44:129, 44:131, 47:02, 50:12, 50:17		2														
*18:56, 44:17, 53:22										10						15
*35:02:01-35:02:06, 35:81, 35:83, 35:95, 35:129N, 35:146, 35:157, 35:162, 35:172, 35:182-35:184, 35:211, 35:220, 53:19													13			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*35:154		2												13		
*37:25				4												
*40:06:01:01-40:06:10, 40:44, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:86, 40:95, 40:103, 40:127, 40:131, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268, 40:270, 40:275, 50:31		2				6					11					
*40:132, 40:178		2														15
*40:269		2								11						

Negative Control

*41:01, 41:06-41:07, 41:09, 41:12, 41:14, 41:16, 41:20-41:22, 41:25, 41:28-41:29, 41:32		2				6			9								Negative Control
*41:17, 50:01:01-50:01:04, 50:04-50:05, 50:07-50:11, 50:13-50:16, 50:18-50:20, 50:32-50:35, 50:37-50:38		2				6											
*41:24		2							10								
*41:30		2				6			9	10							
*44:15, 44:18	1		3														
*44:20, 44:47, 44:100, 49:20	1																
*44:117, C*05:102, C*08:93, C*15:25						5											
*44:140			3														
*47:03			w														
*50:02	1	2															
*50:36		2				6								13			
*51:23, 51:108, 52:15	1															15	
*51:97						6				10							
*55:56			3			6			9								
*55:63									9								
*56:13			3			6											
*82:01-82:03			3													15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	