

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele^{1,2,3}																								
*46:01:01-46:01:02, 46:01:04-46:01:15, 46:01:17-46:01:23, 46:35-46:37, 46:42, 46:44, 46:46, 46:48, 46:50, 46:52-46:53, 46:60-46:61, 46:63-46:71, 46:73	1				5				9															
*46:01:03, 46:01:16	1				5				9						15									
*46:02	1	2			5				9															
*46:03	1		3		5																			
*46:04				4	5				9															
*46:05	1								9	10														
*46:06	1				5	6		8									17						23	
*46:07N	1				5		7		9															
*46:08	1				5			8																
*46:09	1				5				9		11													
*46:10	1				5				9			12												
*46:11	1				5			8					13		15		17							
*46:12	1				5				9					14										
*46:13:01-46:13:02, 46:33	1				5			8							15		17							
*46:13:03, 46:43	1				5			8									17							
*46:14	1			4	5				9															
*46:15N	1				5				9	10														
*46:16		2			5		7		9															
*46:17	1				5				9							16								
*46:18	1		3		5			8					13		15		17							
*46:19	1				5			8				12												
*46:20	1				5	6			9															
*46:21:01	1		3		5			8									17							
*46:21:02	1		3		5			8							15		17							
*46:22					5				9				13		16									
*46:23, 46:57 ⁵	1				5				9								18							
*46:24	1				5				9									19						
*46:25	1				5			8									17						23	
*46:26	1				5				9												21			
*46:27	1				5				9													22		
*46:28, 46:34 ⁵	1				5				9										20					
*46:29	1		3		5				9															
*46:30	1				5				9														23	
*46:31, 46:47, 46:49, 15:318 ⁴					5				9															
*46:32	1				5				9				13											
*46:38	1				5				9	10								19						
*46:39	1				5			8	9															
*46:40:01-46:40:02				4	5				9			12												
*46:41N					5				9													22		
*46:45, 46:54, 46:58-46:59, 46:62	1				5																			
*46:51Q	1				5				9												21		23	
*46:55N	1				5				9			12			15									
*46:56	1				5		7		9													22		
*46:72	1								9															
*07:09, 07:11, 07:17, 07:162, 07:237, 07:255, 07:287, 18:35, 18:110, 35:66, 40:38, 40:52, 40:59-40:60, 40:158, 40:222, 40:352, 48:14, 48:23, C*07:38:01-07:38:02						6																		
*07:55, 08:70, 48:19, C*02:60														14										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:78, 13:02:01-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:48-13:49N, 13:53-13:56:02N, 13:58, 13:65-13:70, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 14:37, 35:60, 44:15:01-01-44:15:01:02, 44:18, 45:01:01-45:01:03, 45:03-45:08, 45:10-45:21, 49:01:01-01-49:03, 49:06-49:14, 49:16-49:17, 49:19N-49:46, 49:48, 50:01:01-01-50:02, 50:04-50:08, 50:10-50:11, 50:13, 50:15, 50:18-50:19, 50:31-50:32, 50:34-50:40, 50:43-50:46, 50:48-50:50, 50:52-50:54, 51:15, 51:106:02, 51:157, 51:225, 52:25:01, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:16-54:30, 54:32, 54:34-54:37, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:02:10, 55:05, 55:07, 55:09-55:12, 55:15-55:16, 55:18, 55:22, 55:24-55:26, 55:29-55:31, 55:33-55:38, 55:40-55:41, 55:43, 55:45-55:48, 55:50, 55:52, 55:54-55:57, 55:59-55:60, 55:62-55:67, 55:70-55:80, 55:82-55:85, 55:87, 56:01:01-01-56:01:09, 56:07-56:08, 56:13-56:14, 56:16-56:17, 56:20:01-56:20:02, 56:23-56:30, 56:33-56:42, 56:44-56:49, 56:52-56:55, 59:01:01-01-59:01:01:02, 59:04-59:10N, C*12:183													13											

Negative Control

[illegible]

*15:86, 15:188, 15:224			3					8						15		17								2
*15:91								8						16		17								
*15:108, 15:136, 15:235, 15:249								8				12												
*15:117									9															23
*15:125									9	10														
*15:131, 15:161								8						16										
*15:138						6		8																23
*15:179:01														15										
*15:199			3																					
*15:241						6			9															23
*15:289, 15:397					5			8																
*15:292								8																22
*15:297						6		8																
*15:340									9										20					
*15:348			3					8								17								
*15:401									9															22
*15:415, 35:62									9		11													
*18:06, C*03:278				4																				
*18:56, 35:45, 35:71, 44:17, 44:43:01-44:43:02, 44:144, 44:230, 44:256, 45:09, 53:22, 57:93, 58:07, C*02:82, C*06:165, C*14:61														16										
*18:135										10							19							
*18:142, 73:01-73:02				w																				
*35:74, 35:356, 40:73, C*16:86				4	5																			
*35:113, 44:98, 52:31:01-52:31:02, A*02:376, C*03:125, C*03:176																								22
*35:186				4	5							12												
*55:03, 55:69					5									13										
*55:19						6								13										
*56:19N							7							13										
C*01:73	1																							
C*02:28, C*12:180											11													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24