

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele^{1,2,3}																								
*46:01:01-46:01:02, 46:01:04-46:01:15, 46:01:17-46:01:23, 46:01:25, 46:35-46:37, 46:42, 46:44, 46:46, 46:48, 46:50, 46:52-46:53, 46:60-46:61, 46:63-46:71, 46:73-46:76	1				5				9															
*46:01:03, 46:01:16, 46:01:24	1				5				9						15									
*46:02	1	2			5				9															
*46:03	1		3		5																			
*46:04				4	5				9															
*46:05	1								9	10														
*46:06	1				5	6		8									17						23	
*46:07N	1				5		7		9															
*46:08	1				5			8																
*46:09	1				5				9		11													
*46:10	1				5				9			12												
*46:11	1				5			8					13		15		17							
*46:12	1				5				9					14										
*46:13:01-46:13:02, 46:33	1				5			8							15		17							
*46:13:03, 46:43	1				5			8									17							
*46:14	1			4	5				9															
*46:15N	1				5				9	10														
*46:16		2			5		7		9															
*46:17	1				5				9							16								
*46:18	1		3		5			8					13		15		17							
*46:19	1				5			8				12												
*46:20	1				5	6			9															
*46:21:01	1		3		5			8									17							
*46:21:02	1		3		5			8							15		17							
*46:22					5				9				13			16								
*46:23, 46:57 ⁵	1				5				9								18							
*46:24	1				5				9									19						
*46:25	1				5			8									17						23	
*46:26	1				5				9													21		
*46:27	1				5				9														22	
*46:28, 46:34 ⁵	1				5				9										20					
*46:29	1		3		5				9															
*46:30	1				5				9														23	
*46:31, 46:47, 46:49, 15:318 ⁴					5				9															
*46:32	1				5				9				13											
*46:38	1				5				9	10								19						
*46:39	1				5			8	9															
*46:40:01-46:40:02				4	5				9			12												
*46:41N					5				9													22		
*46:45, 46:54, 46:58-46:59, 46:62	1				5																			
*46:51Q	1				5				9												21		23	
*46:55N	1				5				9			12			15									
*46:56	1				5		7		9													22		
*46:72	1								9															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:09:01-07:09:02, 07:11, 07:17, 07:162, 07:237, 07:255, 07:287, 18:35, 18:110, 35:66, 35:392, 35:425, 40:38, 40:52, 40:59-40:60, 40:158, 40:222, 40:352, 40:422, 48:14, 48:23, C*07:38:01-07:38:02						6																		
*07:55, 08:70, 48:19, C*02:60														14										

Negative Control

[illegible]

[illegible]